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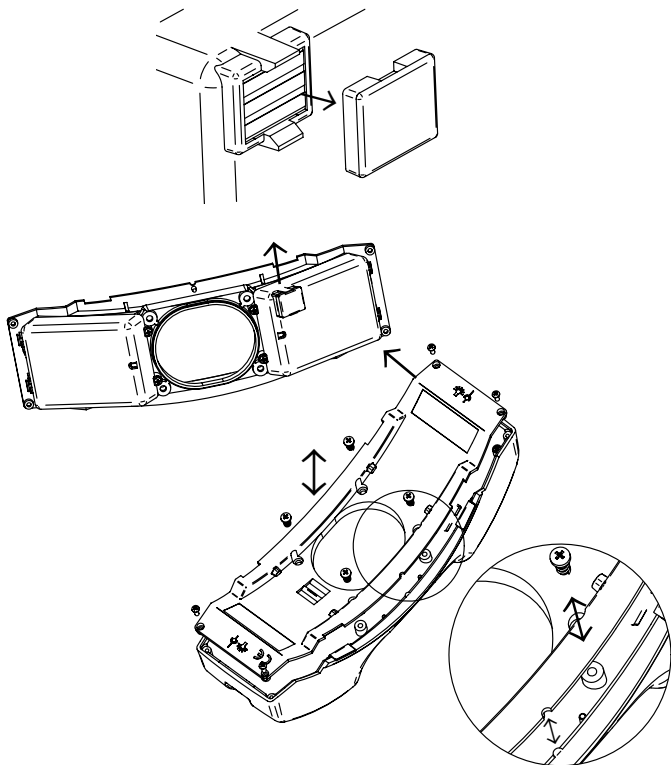
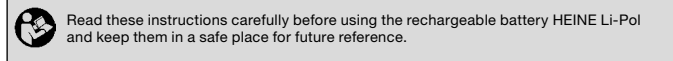
Rechargeable batteries HEINE Li-Pol for HEINE mPack UNPLUGGED

ENGLISH

Read these instructions carefully before using the rechargeable battery HEINE Li-Pol and keep them in a safe place for future reference.

Range of applications

The rechargeable battery HEINE Li-Pol is for use with HEINE mPack UNPLUGGED only (Li-Pol: Lithium Polymer)

General safety instructions

- To ensure optimum performance and safety, please follow these instructions:
- Do not expose to fire (Explosion hazard)
- Ensure that the battery housing is undamaged. Do not use damaged batteries.
- Do not open or modify the Li-Pol battery housing in any way.
- Do not drop or expose to excessive vibration.
- Protect the Li-Pol battery from excessive humidity, direct sunlight and extremes of temperature, which can reduce the charging capacity and life of the battery.
- Do not use in rooms where an explosion hazard is present.
- Do not short-circuit the Li-Pol battery. This can occur when a metal object such as a coin or paper clip connects the two battery contacts together. This can damage the battery.
- Not permitted: Sterilisation, autoclaving, gas sterilisation and Sterrad.
- Keep out of reach of children.
- Store in a cool, dry place.
- Risk of explosion if rechargeable battery HEINE Li-Pol is replaced by an incorrect type.

Safety in use

The ideal ambient temperature during charging is between 10 °C and 35 °C. Do not expose the rechargeable battery HEINE Li-Pol to temperatures over 45 °C. Do not recharge the battery Li-Pol at ambient temperatures below 0 °C. This can destroy the rechargeable battery.

Changing the battery

- Use only the original HEINE Li-Pol battery X-007.99.680.
- The new Li-Pol battery can be inserted simply by opening the lid of the mPack UNPLUGGED:
- Isolate the mPack by removing it from the charger cradle or by unplugging the UNPLUGGED mains transformer.
- Unscrew the eight screws in the lid of the mPack UNPLUGGED with the screwdriver supplied and remove the lid, which contains the Li-Pol battery.
- Make sure that no metal objects such as screws fall into the unit.
- Protect the electronics from electrostatic discharge (ESD), by avoiding contact with electronic components.
- Before replacing the lid, remove the protective cap from the three-pole battery contacts.
- Re-attach the lid with the eight screws, making sure that no wires are damaged when doing so.
- The battery should be disposed of correctly and should be completely discharged before disposal.

Charging the new battery

- Use only the original charger HEINE EN50 UNPLUGGED or HEINE UNPLUGGED mains transformer.
- Read the instructions supplied with HEINE EN50 UNPLUGGED/HEINE mPack UNPLUGGED.
- To enable the LED display of the mPack UNPLUGGED to show the true battery capacity, the unit must first be completely discharged and then recharged. Switch on the instrument connected to the mPack UNPLUGGED until the battery is completely discharged, then recharge the mPack UNPLUGGED until charging is completed.

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