
HEINE SLR Photo Adaptor

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Garching · Germany
E-Mail: info@heine.com · www.heine.com
med 1512 2020-06-02

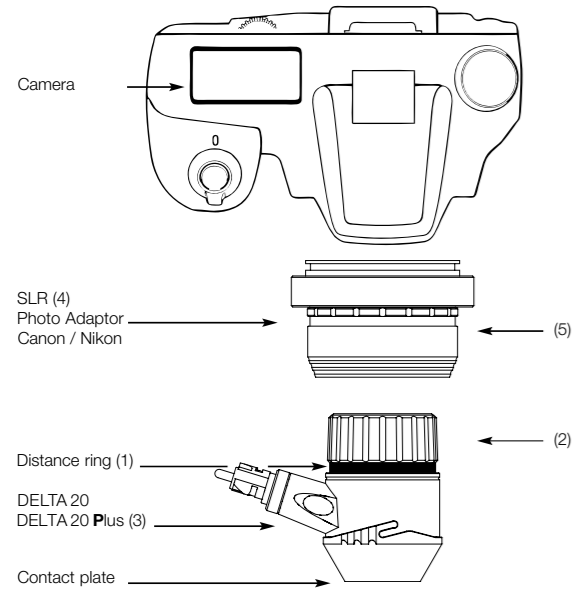HEINE SLR Fotoadapter

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DEUTSCH

HEINE SLR Photo Adaptor

Please read and follow these instructions for use of HEINE SLR photo adaptor and keep them for future reference.

Warnings and Safety Information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

Description

The HEINE SLR photo adaptor with HEINE DELTA 20 Plus and HEINE DELTA 20T has been exclusively developed and optimized for a rapid and systematic documentation of pigmented lesions and does not replace a medical diagnosis. No diagnoses should be made based on a digital image alone.

Assembly of the HEINE SLR photo adaptor

Connect the HEINE SLR photo adaptor to your digital camera. Place the distance ring (1) between the device and the focusing ring (2). Turn the focusing ring (2) in a clockwise direction as far as it will go. You will now have the optimal focus. Next, attach the dermatoscope (3) to the photo adaptor (4): Using your hand, move the sliding ring (5) of the photo adaptor in the direction of the camera and hold it in this position. With your other hand, insert the dermatoscope into the photo adaptor until you audibly hear it click into place. Now release the sliding ring (5) of the photo adaptor; it firmly connects the camera to the dermatoscope. The dermatoscope can now be freely rotated in the photo adaptor, however, it should not be removed. After you have correctly adjusted the camera, you can then start taking dermatoscopic images. In order to remove the dermatoscope from the photo adaptor, move the sliding ring (5) of the photo adaptor towards the camera and pull out the dermatoscope.

Using the HEINE SLR photo adaptor with HEINE DELTA 20 PLUS and DELTA 20T

The recommended camera settings and supported camera models can be found under: (www.heine.com → Products → Dermatoscopes → Documents).

Instructions on the procedure for the digital documentation of birthmarks can be found under: (www.heine.com → Products → Dermatoscopes → Documents)

The size of the image depends on the size of the image sensor used. The depth of field may therefore vary for different digital camera models. In the full-frame camera model (not recommended) for example, the focusing ring is also shown in the image. However, the resolution is sufficient to be able to zoom in on the image.

Cleaning, disinfection and sterilization

A dry cloth is recommended for cleaning the photo adaptor. Disinfection and sterilization are not necessary as the photo adaptor does not come into contact with the patient. Do not spray the device with liquids.

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

Technical data

Bayonet type: NIKON (N), CANON (CN)

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Explanation of utilized symbols

The following symbols are used on the device or on the packaging

	Catalogue- or reference number
	Manufacturer
	Please read and follow these instructions for use and keep them for future reference. (Background: blue, foreground: white)
	Unique Device Identification

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