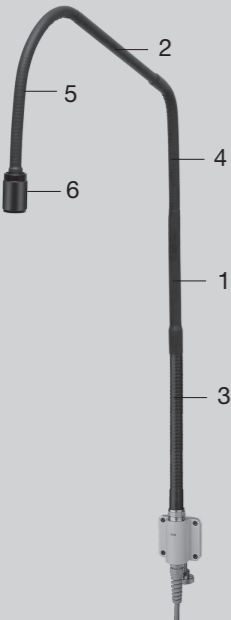
HEINE QUALITY

MADE IN GERMANY

HEINE HKL® Universal F.O. Examination Light

CE 93/42/EWG/CEE/EEC

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 202
E-Mail: info@heine.com · www.heine.com
med 0800 1/1.11

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DEUTSCH

HEINE HKL® Universal-F.O.-Leuchte

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ENGLISH

HEINE HKL® Universal Fiber Optics Examination Light

Read these instructions carefully before using the HEINE HKL® universal fiber optics examination light and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: The HEINE HKL® universal fiber optics examination light is designed exclusively for localized illumination during examination and operations. It should not be used for transillumination by placing it with the illumination head directly on the patient's skin.

Safety in use: Use only light sources which are designed for use in a medical environment.

- Do not light directly into the patient's eyes.
- Ensure that the light is firmly mounted in a safe place.

Assembly/Installation

Installation instructions and materials are included in the packaging.

How to use the HEINE HKL®

The light consists of two rigid sections (1/2), and three flexible lengths (3/4/5). The flexible sections can be bent into position at up to 90°. Extension beyond 180° should be avoided.

The illuminated field can be focused at any distance greater than 5 cm by rotating the focusing head (6).

Cleaning and sterilization

A soft cloth soaked in soapy water or alcohol or any non-aggressive solution can be used to clean the HEINE HKL®.

Gas sterilization can also be used. The HEINE HKL® must not be immersed in sterilizing solution or autoclaved.

Technical Specification

Weight	
HKL with wall-mount:	1415 g
with clamp:	2250 g
Length of flexible section:	120 cm
max. bend of flexible sections:	180°
Light spot at 50 cm:	170 mm dia.
Working time:	continuous

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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