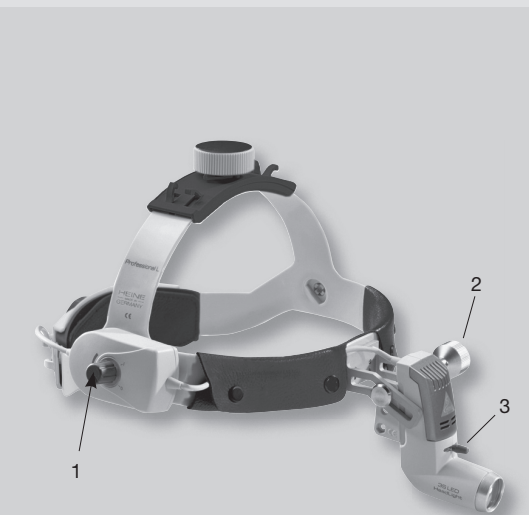
## HEINE 3S LED HeadLight®

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 - 82211 Herrsching - Germany  
Tel. +49 (0) 81 52 / 38 - 0  
Fax +49 (0) 81 52 / 38 - 202  
E-Mail: info@heine.com - www.heine.com  
med 4504 1/2.12

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DEUTSCH

### HEINE 3S LED HeadLight®

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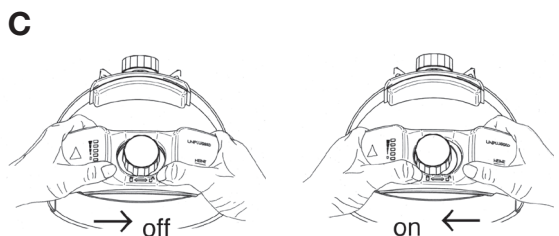
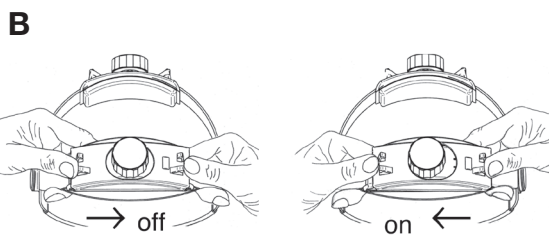
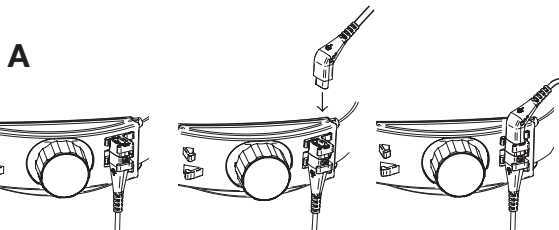
ENGLISH

### HEINE 3S LED HeadLight®

Before using the HEINE 3S LED HeadLight® read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

#### Safety Information

**Range of Application:** The HEINE 3S LED HeadLight® is used for examination and during treatment.

**General Safety:** For applications of the HEINE 3S LED HeadLight® in dental medicine: A yellow filter can be used to avoid premature hardening of composite material.

Do not shine the light directly into the patient's eyes.

Never ever hold magnifying glasses or any other optical system in front of the light.

**Safety in use:** The performance of this unit can only be guaranteed if genuine HEINE Accessories and Spare Parts are used.

Ensure that the headband power cord is located in the clips provided.

The headlight body should not be left covered with a cloth or similar.

**Do not use the HEINE plug-in transformer for 3S LED (X-095.16.320) to power any other bulb-type HEINE instrument.**

#### Installation and operation

##### Attaching the Brightness control:

The brightness control (1) can be mounted on the left or right side of the headband. (Please refer to the enclosed assembly instructions.) The brightness can be continuously controlled by the brightness control (1).

##### Setting up:

To set up the HEINE 3S LED HeadLight® with HEINE mPack® or HEINE EN50®, connect the short cord from the brightness control with the connecting cord mPack UNPLUGGED/Cinch and fix this to the back part of the headband (Fig. A). Connect the instrument to the Cinch-socket of the HEINE mPack® or HEINE EN50® and select the brightness by means of the brightness control.

To use the HEINE 3S LED HeadLight® with the HEINE mPack® UNPLUGGED first remove the rear cover of the headband (Fig.B).

Then attach the mPack UNPLUGGED to the rear of the headband. To do so, hold the headband in your left hand and push the mPack UNPLUGGED to the left, until it clicks into place (Fig.C).

**Note:** We recommend that the mPack brightness control is switched off when used the 3S LED HeadLight.

#### Selecting viewing angle and field of view

##### Without HEINE S-Guard®

Loosen the fixing screw (2) to adjust the light head up and down and backwards or forwards.

After adjustment, re-tighten the screw (2).

##### With HEINE S-Guard®

- Before putting on the loupes, select your inter-pupillary distance (PD) or an average value on the scale. Unscrew the retaining screw and slide the eyepieces to one side symmetrically.
- Unscrew the fixing screw (2) and locate the nosepad on the bridge of the nose.
- Adjust the eyepieces so that they nearly touch the protective lenses.
- Shine the 3S LED onto an object at your normal working distance and adjust the eyepieces so that the object is centred in both eyepieces. After adjustment, re-tighten the screw (2).

The loupes are correctly adjusted when you see a clearly-defined, round, shadow-free field of view.

The illuminated field of view can be selected by means of the slide control (3).

**Fitting HEINE binocular loupes and a video camera:** Use the appropriate adaptor, which is available as an accessory.

#### Cleaning

Use a micro-fiber cloth to clean the lens. The instrument housing can be cleaned with a soft cloth moistened with alcohol or a non-aggressive disinfectant solution. Do not soak or spray with disinfectant.

Read the instructions for cleaning the HEINE i-View®.

#### Technical Specification

Power consumption:	2,6 W, Input Voltage: 6V–9V DC
Length of the cable:	1,6 m
Diameter of the light spot at a distance of 420 mm:	30–80 mm
Vertical direction of illumination:	+/- 20° adjustable
Weight:	70 g
Operation time:	12,5 h with HEINE mPack® 5,2 h with HEINE mPack® UNPLUGGED
Ambient temperature:	In use +10 °C - +40 °C During storage or transport: –40 °C - +70 °C
Classification:	moderate risk, IEC 62471 [2006]

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#### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For repairs, please contact your supplier, who will return the goods to us or our authorized agent.

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