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DEUTSCH

Ladebatterien

HEINE Li-ion mPack®/Li-ion mPack LL®

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- UL 1642** (Standard for Safety of Lithium-ion Batteries)
ISO 7000 – 2004 (Graphical symbols for use on equipment)
EN 980: 1996 (Graphical symbols for use in the labelling on Medical Devices)

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 2 02
E-Mail: info@heine.com · www.heine.com
med 2305 1/4.10

Instructions for use

ENGLISH

Rechargeable batteries

HEINE Li-ion mPack® / Li-ion mPack LL®

Read these instructions carefully before using the rechargeable battery HEINE Li-ion mPack®/Li-ion mPack LL® and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Intended use: The rechargeable battery HEINE Li-ion mPack®/Li-ion mPack LL® is for use with HEINE mPack® and HEINE mPack LL® only.

General safety instructions:

To ensure optimum performance and safety, please follow these instructions:

- Do not expose to fire
- Do not short circuit
- Do not modify or dismantle
- o not drop or expose to excessive vibration
- Not permitted: Sterilisation, autoclaving, gas sterilisation and Sterrad
- Do not expose to direct sunlight
- Keep out of reach of children
- Store in a cool, dry place

Caution: Dispose of used batteries according to the instructions.

Caution: Risk of explosion if HEINE Li-ion mPack®/Li-ion mPack LL® is replaced by an incorrect type.

Safety in use:

The ideal ambient temperature during charging is between 10 °C and 35 °C. Do not recharge the battery Li-ion mPack/Li-ion mPack LL at ambient temperatures below 0 °C. This can destroy the rechargeable battery.

Setting up

- Detach the charging plug from the HEINE mPack® or HEINE mPack LL®.
- Unscrew the screw above the clip with a coin or screwdriver and remove the lid.
- The rechargeable battery HEINE Li-ion mPack®/Li-ion mPack LL® can be removed after detaching the cord. No tools are needed to do this.
- Insert the new battery so that the label is visible and the battery cord is directed towards the front control.
- Plug the plug of the battery into the socket on the circuit board.
- To close, locate the two pins on the lid in the cut-outs in the lower half of the housing. Close the lid and tighten the screw.
- Recharge the HEINE mPack® or HEINE mPack LL® for 2 hours just before use.

Technical Specification

Environmental conditions:	Normal Use: +10 °C to +40 °C Storage: +5 °C to +45 °C Transport: –20 °C to +45 °C
Relative humidity:	10 % to 75 %
Air pressure:	700 hPa to 1060 hPa
Voltage:	7,2 VDC

Applicable standards:

- UL 1642** (Standard for Safety of Lithium-ion Batteries)
ISO 7000 – 2004 (Graphical symbols for use on equipment)
EN 980: 1996 (Graphical symbols for use in the labelling on Medical Devices)

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