HEINE i-View®

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-mail: info@heine.com · www.heine.com
med 3106 1/1.09

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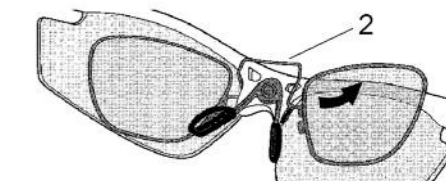
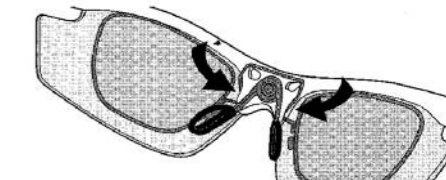
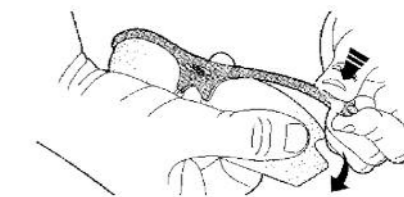
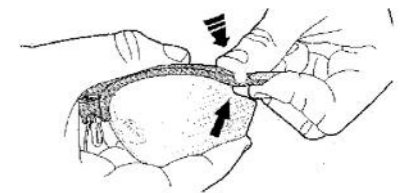
CE 93/42/EWG/CEE/EEC

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from _____

date _____

Instructions for use

ENGLISH

HEINE i-View®

Before using the HEINE i-View®, read these instructions carefully and keep them in a safe place for future reference.

Safety informations

Range of applications: The HEINE i-View® is designed for use exclusively with HEINE instruments and light sources.

General Safety: Use only HEINE original replacement parts and accessories.

Safety in use: To avoid possible damage to the eyes, avoid looking directly at the sun or a bright light! The adaptors in illustrations 1+2 must be assembled as shown. HEINE declines responsibility for any assembly defects caused by the optician. Do not unscrew the set screws. They are factory-set for the appropriate working distance (7).

Description: The HEINE i-View® and the HEINE S-Guard® are already assembled by HEINE completely. Attach the HEINE i-View® to the headlight using the adaptors which are already assembled by HEINE, too. The ease of adjustment of the joints shown in illustrations 1+2 can be adjusted with the screwdriver supplied.

Retrofitting the HEINE i-View®:

Screw your HEINE S-Guard® to the **3S LED HeadLight** as Fig. 1

Screw your HEINE S-Guard® to the **MD 1000 F.O. HeadLight** as Fig. 2

Converting to HEINE i-View®: Mount the HEINE i-View® with the S-Guard to the HEINE 3S LED HeadLight® or the HEINE MD 1000 F.O.* (→ Retrofitting the HEINE i-View®). Undo the knurled screws which attach the optics to the old optics holder (Fig. 3). Mount the optics to the HEINE i-View® as before (Fig. 4). Adjust the eyepieces as described in the instructions for HEINE binocular loupes. Attach the stickers showing the magnification to the HEINE i-View® as in (a) and the working distance as in (b).

Please note: We can only guarantee the performance of HR and HR-C loupes when converted in this way. This is not the case with HRP loupes. The stickers should not influence the mobility of the loupes (4).

HEINE i-View® without S-Guard: If you prefer to use the HEINE i-View® without S-Guard, please consult the attached mounting instructions and use the enclosed mounting-set.

Adjustment and Operation

Adjusting the nose bridge (A): This can be bent into position to suit the anatomy of your nose. By bending the bridge supports you can vary the distance between the lenses and the eyes. This avoids contact between your eyelashes or cheekbones and the frame or corrective lenses.

Changing the nose pads: To change the nose pads (1), unclip them and clip in new pads. The left and right nose pads are identical.

Fitting optional corrective lenses: Please consult an optician before having corrective lenses fitted. Please show the optician the attached "Instructions for specifying corrective lenses". To fit the corrective lenses (B), insert one side of the snap-in frame (2) into the groove in the S-Frame. Next, insert the other side of the snap-in frame in the second groove.

To remove the snap-in frame with corrective lenses, reverse the above procedure, avoiding damage to the lenses.

Changing protective lenses: Hold the S-Frame as in (D) and bend the lens in the direction shown and pull the lens with an audible click out of the frame. To replace, follow the procedure shown in (E) and (F) until the lens clicks into place.

If the standard protective lenses are too close to your cheekbones, you can either fit the small protective lenses or adjust the nose bridge (A).

Swivelling the loupe: The HEINE i-View® can be swivelled upwards with one hand. Use the sterilizable swivel levers to do this (see accessories). This movement makes it possible to observe the illuminated field under investigation directly without touching the setup of the headlight.

Instructions for specifying corrective lenses:

When specifying and fitting corrective lenses, be sure to take into account the following:

- The magnifying optics should be attached to the frame to suit your usual working posture.
- Prismatic errors should be corrected.
- The corrective lenses are mounted at an angle to the optical axis.

Check the correction by means of an eye test with the magnifying optics and the corrective lenses fitted.

The total spherical correction is usually max. +/- 5 dpt. Ask your optician or ophthalmologist!

The corrective lenses should be placed as closely as possible to the eyepieces of the magnifying optics.

Cleaning and Maintenance

General: The plastic protective lenses have an anti-reflex coating. Clean them carefully to avoid mechanical or chemical damage. HEINE recommends the use of only those cleaners or disinfectants which are specifically-recommended by the manufacturer for use with plastic lenses to avoid any risk of material damage.

Sterilisation, machine washing, soaking and ultrasonic cleaning are not permitted.

Manual Cleaning: Clean the binocular loupe carefully with the cloth provided or with a damp microfiber cloth. We recommend HEINE Cleaning Spray.

Disinfectant wipes: Use a HEINE microfiber cloth with Isopropyl alcohol at max 70% concentration.

Spray disinfection: We recommend a spray Isopropyl alcohol at max 70% concentration.

Specification

Protective lenses:
i-View and S-Guard: Plastic, anti-reflex coated
Nickel-free

Accessories and Spare Parts

One pair of protective lenses, large (lg)	C-000.32.306
One pair of protective lenses, small (sm)	C-000.32.307
Snap-in frame for corrective lenses	C-000.32.309
HEINE Cleaning Spray	C-000.32.542
5 Steri-levers	C-000.32.399

Fig. 1

Fig. 2

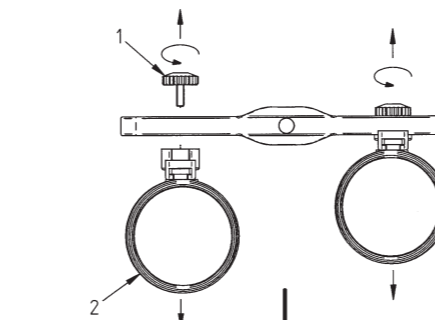
Fig. 3

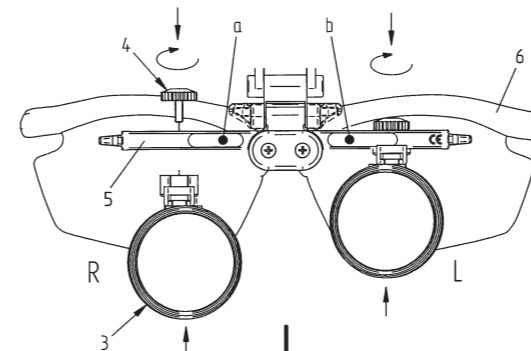
Fig. 4



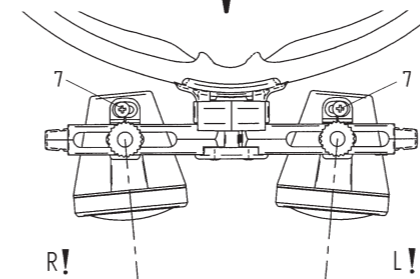
Fig. 5

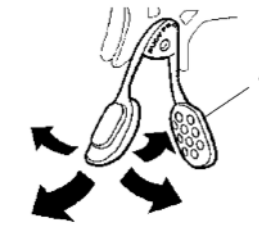
A


B

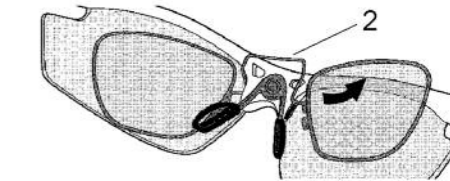
C

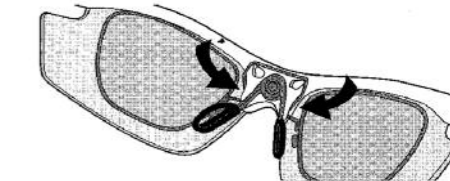D

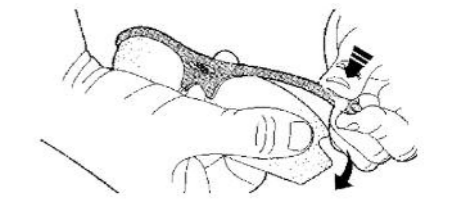E

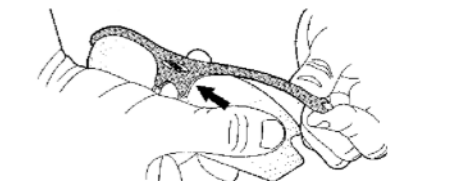F