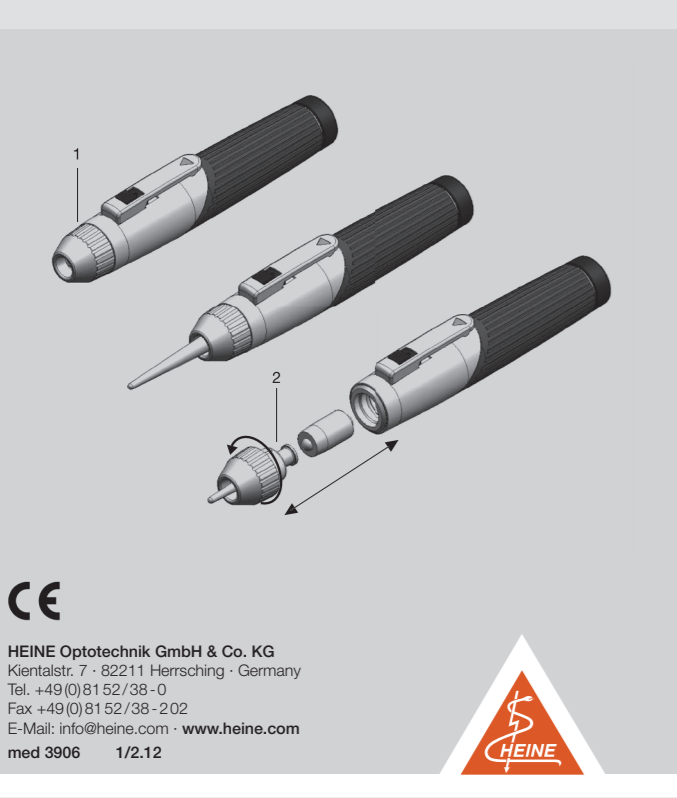
**HEINE** QUALITY  
MADE IN GERMANY

## HEINE mini-c® Cliplamp and Ear Light

**HEINE** Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0) 81 52/ 38 - 0  
Fax +49 (0) 81 52/38 - 202  
E-Mail: info@heine.com · [www.heine.com](http://www.heine.com)  
**med 3906**    **1/2.12**

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### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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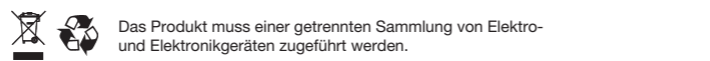
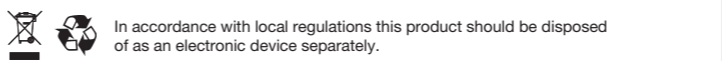
## Instructions for use

### HEINE mini-c® Cliplamp and Ear Light

<p>Read these instructions carefully before using the HEINE mini-c® Cliplamp and keep them in a safe place for future reference.</p>
--

#### Range of applications

- HEINE mini-c® Cliplamp (1): The HEINE mini-c® Cliplamp is designed solely for the examination of the surface of the skin and the anterior segment of the eye.
- HEINE Ear Light (2): The HEINE Ear Light in combination with the HEINE mini-c® Cliplamp is exclusively designed for the lighting of the transition area from the auricle (external ear) to the external acoustic meatus (ear canal).

#### Safety in use

- The HEINE Ear Light is used by hearing aid specialists or trained personnel as a device for the anatomical adjustment of the geometry of hearing aids.
- The HEINE Ear Light is not suitable for the examination of the eardrum and may not be introduced too deeply into the acoustic meatus.
- Do not use alkali manganese batteries containing lithium as bulb life will be reduced.
- The performance of this instrument can only by guaranteed if genuine HEINE Accessories are used.

#### Cleaning and Maintenance

The HEINE mini-c® Cliplamp is not suitable for treatment in the ultrasonic bath, for sterilisation, spray disinfection or treatment by immersion in liquids. The surface of the HEINE Ear Light may be damaged by alcoholic cleaning agents.

- Manual Cleaning and Disinfectant wipes: Use a damp cloth for cleaning the HEINE mini-c® Cliplamp and HEINE Ear Light. For cleaning and disinfection, a disinfectant agent suitable for plastic medical products must be used. Follow the instructions of the manufacturer of the disinfectant for this purpose. Disinfection by spraying or immersion as well as sterilisation is not allowed.

- Inspection / Functional testing: The HEINE Ear Light must be checked for its functionality for its intended use on a regular basis. The item must be discarded, if any damage is visible or suspected.

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