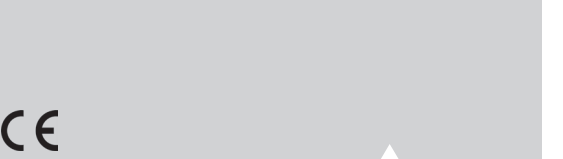
HEINE Standard F.O. L®

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DIN EN ISO 7376

(Anaesthetic and respiratory equipment – Laryngoscopes for tracheal intubation)

Instructions

HEINE Standard F.O. L® Laryngoscope

Rechargeable Handle

Read these instructions carefully before using the HEINE Standard F.O. L® Laryngoscope Rechargeable Handle and keep them in a safe place for future reference.

Safety instructions

Symbols

In accordance with local regulations this product should be disposed of as an electronic device separately.

Range of applications: HEINE Standard F.O. L® Laryngoscope rechargeable handles are designed for use with F.O. Laryngoscopes for intubation only. The performance of the handles can only be guaranteed if genuine HEINE rechargeable batteries (Li-Ion L, see separate instruction leaflet for HEINE rechargeable batteries) and other spare parts are used.

General safety:

- Do not open the bottom insert.
- All repairs other than a change of batteries should be carried out by qualified persons.

Safety in use:

- Use only blades and handles with a green mark together.
- Do not use in the presence of strong magnetic fields.
- Charge rechargeable handles before use.
- The handles should not be used in the presence of explosive substances.
- Rechargeable handles should not be used while they are being recharged in the charger.
- The bulb brightness can reduce during use if the rechargeable battery was not charged sufficiently before use. We recommend a minimum recharge time of 10 minutes for HEINE rechargeable batteries in HEINE chargers.
- Bring the blade into the normal working position and check that the light switches on.
- After some minutes' use the laryngoscope bulb will be hot.
- The laryngoscopes may only be used by trained personnel.

Application

1. HEINE Standard F.O. L® Laryngoscope Rechargeable Handle:

HEINE Standard F.O. L® Laryngoscope rechargeable handles consist of four basic components:

- Handle shell (1)
- Battery insert (2)
- Li-Ion L rechargeable battery (3)
- Bottom insert (4)

Changing the rechargeable battery:

- Uncrew the battery insert from the handle shell and remove it.
- Uncrew the bottom insert from the battery insert and remove it complete with the rechargeable battery.
- Pull the rechargeable battery out of the bottom insert.
- Insert the new rechargeable battery into the bottom insert with an audible „click“.
- Screw the bottom insert complete with the rechargeable battery back into the battery insert.
- Screw the battery insert back into the handle shell.

Changing the ampoule:

- Uncrew the battery insert from the handle shell and remove it.
- Uncrew the bulb from the battery insert.
- Replace the new bulb into the battery insert carefully and screw it in.
- Screw the battery insert back into the handle shell.

2. Angled F.O. L handle (not shown)

The angled F.O. L handle consists of the following components:

- Handle head
- Outer sleeve
- Set of batteries or rechargeable battery
- Adaptor rechargeable battery
- Bottom insert

Changing the rechargeable battery:

- Uncrew the bottom insert from the outer sleeve, remove the battery.
- Insert the new battery into the outer sleeve.
- Screw the bottom insert back into the outer sleeve.

Changing the bulb:

- Uncrew the handle head.
- Uncrew the nut with the key supplied. Alternatively, use a coin inserted into the slits in the nut and the surrounding part and press down.
- Uncrew the bulb completely and remove the bulbholder and bulb.
- Place the new bulb in the opening and place the bulbholder over it.
- Screw in the nut.
- Fit the handle head so that both marks coincide and screw the handle head firmly in.

The rechargeable battery:

This handle has been developed specifically for use with the HEINE Li-Ion L battery (see the instruction leaflet HEINE Rechargeables batteries) and for fast-charging in the NT300 charger (for details see the instruction leaflet NT300).

The NT200 charger can also be used, but does not offer the same performance, as it does not offer fast-charging, (a full charge can take up to 30 hours). The handle can be modified by simply exchanging the bottom cap and the rechargeable battery.

The bottom insert incorporates a charge status display which shows the state of charge of the battery. This activates automatically when the handle is removed from the charger or when it is switched on. After 10 seconds the display switches off.

The display gives the following information:

- Orange flashing: <10%
- Orange: 10–33%
- Orange/green: 33–66%
- Orange/green/green: 66–100%

Important Notes: When using the F.O. L angled handle with a Li-Ion battery, the F.O. L rechargeable battery adaptor (1) must be used. The adaptor must be fitted so that the coil spring faces the rechargeable battery (2).

Cleaning / Disinfection / Sterilising

The following instructions must be followed according to national standards, laws and guidelines. Any such measures must be carried out by qualified personnel. Before cleaning/disinfecting remove the batteries. See the instructions for the appropriate handle type.

For cleaning and disinfection of components, only those products which are approved for use on **medical devices with plated surfaces and plastics** (handles contain plastic components, e.g. bottom cap) may be used. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid reactions with following preparation steps and material damage. The manufacturer's recommendations for use must be followed.

F.O. L Handle

Cleaning / Disinfection:

The handle sleeve can be cleaned and disinfected manually (wiping/soaking) or by machine (thermal disinfection up to 93 °C).

Battery insert can be cleaned and disinfected manually (wiping) only.

Sterilisation: Steam Sterilisation at 134 °C is allowed for the handle sleeve only.

F.O. L Angled Handle

Cleaning / Disinfection:

Handle sleeve and handle head can be cleaned and disinfected manually (wiping/ soaking) or by machine (thermal disinfection up to 93 °C).

Sterilisation: Steam Sterilisation at 134 °C is allowed for the handle sleeve and handle head only.

Applicable standards:

DIN EN ISO 7376

(Anaesthetic and respiratory equipment – Laryngoscopes for tracheal intubation)

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