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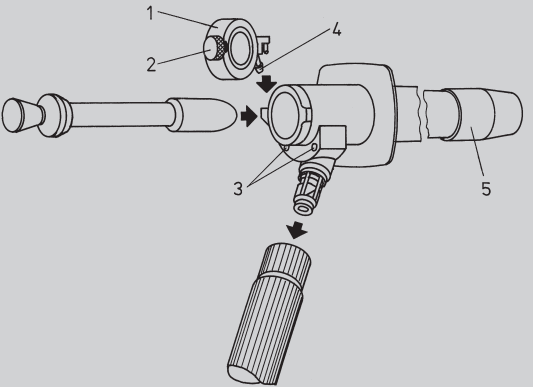
Instructions for use

ENGLISH

HEINE reusable anoscope / proctoscope and sigmoidoscope with fiber optic illumination

Before using the HEINE anoscope / proctoscope or sigmoidoscope, please read these instructions carefully and keep them in a safe place for future reference.
--

Safety information
Range of applications: HEINE anoscopies / proctoscopes and sigmoidoscopes with fiber optic illumination are designed only for examination and treatment of the anal passage.
Safety in use: Use only light sources which are approved for use in a medical environment. Tubes with straight direction of view should be inserted when an obturator is fitted.

1. Anoscope

Attach the instrument to the handle of the light source. Turn the obturator clockwise to lock in position in the tube. To unlock, turn anticlockwise. The cone of the obturator and the tube should be lubricated with a suitable lubricant.

2. Proctoscope and sigmoidoscope

Attach the instrument to the handle of the light source. Make sure that the end-piece (5) is firmly screwed-in. If required, connect an insufflation bulb to the window of the instrument (4).

Loosen the sealing screw (2) and open the window (1). If the window in not required during the examination, it can be removed by extracting it upwards out of the instrument head.

Insert the obturator. Apply a suitable lubricant to the tip of the obturator and the tube.

The plane glass window (1) can be opened or closed at any time. The instrument can be made airtight by gently tightening the screw (2). If an insufflation bulb without a valve is used, the insufflated air can be regulated by gently loosening or tightening the screw.

If magnification of the object is required, insert the loupe into one of the two holes provided (3). Theoupes are marked P for proctoscope and R for recto- or sigmoidoscope. Both swivel to allow the window to be opened or closed. The telescope is inserted into both of the location holes (3) and clamped by means of a clamping screw. During instrumentation, the telescope can be folded down wards.

The vacuum tube accessory can also be located in the holes (3) by means of the attachment. It can then be moved backwards or forwards along the tube and remains steady in any given position.

This does not limit the use of a loupe in any way.

Cleaning / Disinfection and Sterilisation
Preparation at the point of use/Initial Cleaning: Residues and other deposits must be removed immediately after use to avoid any residues drying on to the instrument.
Disassembly: The instrument must be removed from the light source for further preparation steps. The article must be disassembled (see picture) to ensure a satisfactory cleaning result.
TUBES AND OBTURATOR
Manual disinfection and cleaning
For manual cleaning and disinfection, only those cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastics may be used. The manufacturer's recommendations for use must be followed to avoid material damage / corrosion. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid surface residues.

Machine cleaning and disinfection/Thermal Disinfection
The article can be machine-cleaned and disinfected at up to ≤ 93 °C. Only cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastic may be used. The instructions of the manufacturer of both machines and detergents must be followed. The machine should be loaded correctly so that the cleaning procedure is successful. Rinsing procedures must ensure that no deposits are left. The final rinse should be carried out using de-mineralised water.
Ultrasonic Cleaning is not allowed for articles with fiber optics.

Sterilization

We recommend sterilization with saturated steam at 134 °C for 5 minutes. Cycles of 18 minutes are allowed. There may be a reduction in light transmission with frequent/repeated long-term steam sterilization, particularly if the steam quality is not ideal.

Telescopo/Rubber-Insufflations Bulb

Telescope and Rubber-Bulb can be treated by surface disinfection. Low temperature Gas-Plasma-Sterilization and Gas Sterilization with Ethylene oxide are permitted for the Telescope.

Warning: Non-suitable or incorrectly-used chemicals or preparation procedures can lead to irreparable damages to the article.

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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