# HEINE XENON 1000®

## Projektor

	93/42/EWG/CEE/EEC
<b>HEINE</b>	
HEINE Optotechnik GmbH & Co. KG Kientalstr. 7 · 82211 Herrsching · Germany Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202 E-mail: info@heine.com · <a href="http://www.heine.com">www.heine.com</a> med 0202 1/10.08	

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#### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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## Instructions

### HEINE XENON 1000® Projektor

Before using the HEINE XENON 1000® Projektor, please read these instructions carefully and keep them in a safe place for future reference.

-  In accordance with local regulations this product should be disposed of as an electronic device separately.

**Contents**  
Projektor XENON 1000  
Mains cord  
2 Fuses  
Instructions

#### Safety Information

**Range of applications :** The XENON 1000 Projektor is used only as a light source for medical lights by means of a Fiber Optics cable. The unit is designed for continuous use.

Do not use for transillumination of tissue as skin burns could result.

**General Safety:** This unit needs no maintenance. Do not open the housing. Any repairs including, if possible, bulb changing should be carried out by a qualified technician.

**Safe installation:** Ensure that the projektor is free-standing and that there is an unobstructed flow of air from all sides. Do not place on soft furniture, cloth or carpets and do not place any inflammable objects in the light exit (1).

**Safety in use:** The mains voltage on the identification plate must correspond to the local mains voltage. Operating position: Horizontal ±15°. Do not use in rooms where an explosion hazard or a strong magnetic field is present.

#### Assembly

For wall or wheeled-stand mounting, see instructions S 209.50.02 supplied.

#### Setting up

- Connect the unit with the mains cord to the power supply.
- Plug the Fiber optics cable into the light exit (1) until it clicks into place.
- Switch on at (3). After about 60 sec. the XENON lamp produces maximum brightness.
- Please note: Frequent on/off switching can lead to a reduction in bulb life.
- Adjust the brightness with control (2) independent of colour temperature.

#### Changing a bulb module

**Important Note:** High-performance XENON bubs must be handled with great care. We can only guarantee the performance of this device if an original HEINE bulb module is fitted.

#### Safety Instructions

In cold state XENON arc lamps have a high internal pressure. We recommend the use of protective gloves and safety glasses during a bulb change. Before changing the bulb module, remove the protective cover ( f ) over the reflector. When the bulb has been changed, replace the protective cover. Follow the safety instructions from OSRAM which are supplied with the spare bulb module.

- Allow the unit to cool down completely and disconnect from the mains.
- Open the bulb lid ( a ) underneath the unit with a coin or similar and remove. To disconnect the bulb, operate the two security levers ( c ) to remove the plug ( b ) and pull it out as shown by the arrow.
- Hold the bulb module at ( d ) and pull it out carefully and place it on an even surface. Under no circumstances should you touch the glass ( e ) !
- Remove the new bulb module from its packaging and remove the protective cover ( f ) .
- Insert the bulb module into the guide rails until it clicks into place.
- Plug in ( b ) into the connector ( g ) . Make sure that the plug clicks into place.
- Replace the bulb lid ( a ) in the direction shown by the arrow using gentle pressure and screw in by means of a coin or similar.
- Re-connect the mains cord.

#### Changing a Fuse

- Disconnect the mains cord.
- The fuses are in the fuse compartment of the integral socket. Use only fuses as specified in the technical specification.

#### Cleaning and Sterilizing

Disconnect the mains cord ! Clean only when the unit has cooled down completely. Use only a soft cloth with a mild soap solution or other non-aggressive cleaning agent.

#### HEINE Fiber Optics cable

Gas sterilize or wipe clean with alcohol or soapy water. Do not soak or autoclave. The optical end-surfaces can be cleaned by using a cotton-wool bud soaked in alcohol.

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