HEINE Operations-Otoskop

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-mail: info@heine.com · www.heine.com
med 12496 1/1.09

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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ENGLISH

HEINE Operating Oscope

Read these instructions carefully before using the HEINE operating otoscope and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: The HEINE operating otoscope is used only for examination and operation of the ear.

Safety in use: Use only handles which are specifically designed for use in medical applications. If the tip-adaptor is fitted, the otoscope must be fitted with a tip before it is inserted into the ear.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Specula

Reusable ear-specula are attached to the otoscope so that the peg engages in the bayonet slit. A twist to the right ensures that they are firmly located.

Disposable ear-specula can be push-fitted to the otoscope.

Tip-adaptor

By means of the tip-adaptor, **AllSpec disposable tips** can be used with the operating-otoscope:

- Fit the adaptor to the otoscope so that the pin locates in the bayonet connector. Rotate clockwise to lock in position.
- Fit the AllSpec tip to the adaptor, so that the internal cam locates in the slit in the body of the adaptor and secure by gently rotating the tip clockwise.

Magnification Optics

The swivel lens which is attached to the otoscope gives approx. 2.5 x magnification. The lens can be swivelled to allow the use of an instrument.

Swivelling Device

The shaft below the bulb can be rotated. The knurled sleeve can be unscrewed to allow the speculum mount to be swung aside to leave room for an instrument.

Changing the bulb

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

- Unscrew the bulb.
- Ensure that a bulb of the correct voltage with regard to your handle or transformer is screwed into place.

Sterilization and Cleaning

Instruments can be gas-sterilized, although occasional cleaning with alcohol or soapy water is usually sufficient.

Plastic specula made of SANALON S can be cleaned and disinfected with the usual solutions and can be boiled. They can also be sterilized as often as desired up to 140 °C.

AllSpec disposable tips and UniSpec disposable specula are designed for single use only. Do not attempt to clean them as the tip may be damaged and could injure the patient.

Accessories and Replacement Parts

HEINE XHL® Xenon Halogen bulb 2,5 V	X-001.88.037
HEINE XHL® Xenon Halogen bulb 3,5 V	X-002.88.049
SANALON S specula (reusable) 1 complete set = 6 pcs.	B-000.11.221
Spreadable nasal speculum	B-000.11.231
Spreadable universal speculum (symmetrical)	B-000.11.239
UniSpec disposable ear-specula: 1 box = 1000 pcs., 2,5 mm distal dia.	B-000.11.242
same, 4,0 mm distal dia.	B-000.11.241
AllSpec disposable tips 1 box = 1000 pcs., 2,5 mm distal dia.	B-000.11.128
same, 4,0 mm distal dia.	B-000.11.127
Tip-adaptor	B-000.11.306