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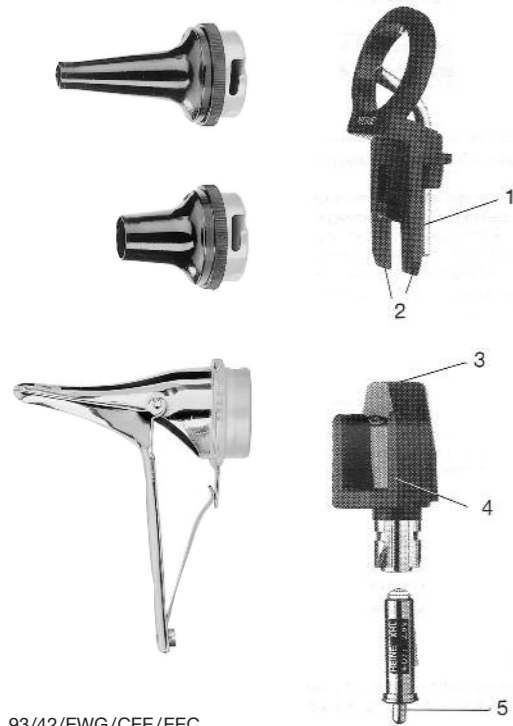
# Instructions for use HEINE multi-purpose illuminator

ENGLISH

Read these instructions carefully before using the HEINE multi-purpose illuminator and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

### Safety information

**Range of applications:** The HEINE multi-purpose illuminator is used only for the following examinations:

- as a diagnostic light for general examination
- with a tongue-blade fitted for examination of the mouth and tonsils
- with the fiber optics nasal illuminator and:
  - ear specula for ear examination
  - nose specula for nasal examination

**Safety in use:** Use only handles which are specifically designed for use in medical applications.

**Please note:** The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

### Instructions

**Attaching a tongue-blade:** The HEINE disposable tongue-blade should be pushed in as far as the stop. Wooden tongue-blades are a simple push-fit in the guide (4).

The **Fiber optics nasal illuminator** should be inserted into the multi-purpose illuminator until it clicks into place so that the two legs (2) locate in the guides (4) and the light guide (1) locates in the light exit (3). Specula can be attached to the illuminator by the bayonet connector.

### Changing the bulb

**Please note:** The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

Remove the illuminator from the handle and pull out the bulb by grasping the collar (5).

### Cleaning and Sterilization

#### Cleaning and Maintenance

Do not use alcohol at high concentration. Sterilisation, machine washing, soaking and ultrasonic cleaning are not permitted.

Use only disinfectant products, which are recommended by the manufacturer for use plastics.

**Manual Cleaning:** Clean the instrument carefully with the cloth provided or with a damp microfiber cloth. We recommend HEINE Cleaning Spray.

**Disinfectant wipes:** Use a HEINE microfiber cloth with Isopropyl alcohol at max 70% concentration.

**Remove the bulb from the illuminator before sterilization.**

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### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories.

(Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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