HEINE QUALITY
MADE IN GERMANY

➤ **HEINE BETA L® Griffe**
HEINE BETA L® Handles

93/42/EWG/CEE/EEC

HEINE Optotechnik GmbH & Co. KG
 Kientalstr. 7 · 82211 Herrsching · Germany
 Tel. +49 (0) 81 52 / 38 - 0
 Fax +49 (0) 81 52 / 38 - 202
 E-Mail: info@heine.com · www.heine.com
 med 1009 1/5.10

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DEUTSCH

HEINE BETA L® Griffe

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ENGLISH

HEINE BETA L® Handles

Read these instructions carefully before using the HEINE BETA L® handle and keep them in a safe place for future reference.

Safety Information

Symbols

In accordance with local regulations this product should be disposed of as an electronic device separately.

Range of applications:

HEINE BETA L® handles are designed exclusively for use with medical diagnostic instruments.

General safety:

- Do not open the bottom insert.
- All repairs other than a change of batteries should be carried out by qualified persons.

Safety in use:

- The handles should not be used in rooms where an explosion hazard is present.
- Rechargeable handles should not be used while they are being recharged.
- The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries are used.

Application

This handle has been developed specifically for use with the HEINE Li-ion L battery (see the instruction leaflet HEINE Rechargeables batteries) and for fast-charging in the NT300 charger (for details see the instruction leaflet NT300). The NT200 charger can also be used, but does not offer the same performance, as it does not offer fast-charging. (a full charge can take up to 30 hours). The handle can be modified by exchanging the bottom insert and the rechargeable battery.

HEINE BETA L® handles consist of three basic components:

- Handle module (1)
- Li-ion L rechargeable battery (2)
- Bottom insert (3)

The bottom insert incorporates a charge indicator which shows the state of charge of the battery. This activates automatically when the handle is removed from the charger or when it is switched on. After 10 seconds the indicator switches off.

The indicator gives the following information:

- Orange flashing: <10%
- Orange: 10–33%
- Orange/green: 33–66%
- Orange/green/green: 66–100%

Automatic Connector (AV)

- Attaching the instrument:
 - Attach an instrument head by inserting it from above so that the connector lugs mate with the groves in the handle. Press the head into the handle until it engages with an audible click.
- Removing the instrument:
 - Remove the head by twisting the knurled ring with thumb and forefinger of the hand holding the handle. Pull off the head with the other hand.

Twist-Lock Connector (TL)

- Attaching the instrument:
 - To connect an instrument head, hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.
- Removing the instrument:
 - To remove the head rotate it clockwise and pull it off.

ON/OFF switch and brightness control:

- Press the button and rotate the black ring clockwise.

Changing the rechargeable battery:

- Unscrew the the bottom insert and remove the battery together with the bottom insert.
- Remove the rechargeable battery from the bottom insert.
- Insert the new rechargeable battery into the bottom insert with an audible „click“.
- Screw the bottom insert back into the handle module.

Cleaning / Disinfection / Sterilising

- The components can be wiped with a cloth moistened with water, cleaning solution or disinfectant.
- Moisture must not be allowed to enter the rechargeable battery or the bottom insert.

Accessories and spare parts

Handle module	X-000.99.010
Rechargeable batterie Li-ion L	X-007.99.383
Bottom insert	X-002.99.396

Applicable standards:

- IEC 60601-1** (Electrical Medical Devices – Part 1: General conditions for safety including the relevant performance data.)

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from	
date	

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