# Mode d'emploi

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## HEINE Ophthalmoscope Trainer

### General Information

The HEINE Ophthalmoscope Trainer is not intended for use on patients and is not a medical product according to directive 93/42 EWG/CEE/EEC.

The device is used to learn and practice ophthalmoscope examination with the direct or indirect ophthalmoscope. It is **not recommended for retinoscopy**, instead, we recommend the HEINE Skia/Retinoscope trainer C-000.33.011.

We are unable to give a complete introduction to Ophthalmoscopy and its application in refraction in these instructions. We recommend the published literature on this subject.

### Technical Description

The optics of the trainer include an achromatic lens with a pupil aperture and an orange-coloured curved shell to mimic the retina which can be moved along the optical axis.

### Technical data

|                      |                     |
|----------------------|---------------------|
| Weight:              | 600 g               |
| Achromat:            | f = 18 mm           |
| Range of refraction: | −10 to +10 dpt      |
| Accuracy:            | +/- 0,5 dpt         |
| Pupil size:          | 2, 3, 4, 5, 6, 8 mm |

### Using the instrument

**Setting up the trainer:** Adjust the body of the trainer so that the optical axis corresponds to your viewing axis, angle can be adjusted by loosening screw (1).

**Selecting refractive error:** Rotate the housing to select a value as shown on the scale (2).

**Selecting Pupil Size:** Six different pupil sizes can be selected by rotating the selector wheel (3). **Ensure that the desired value clicks into place.** The selected pupil size is indicated (4).

**Using trial lenses:** A slot is provided in front of the lens to accommodate two trial lenses.

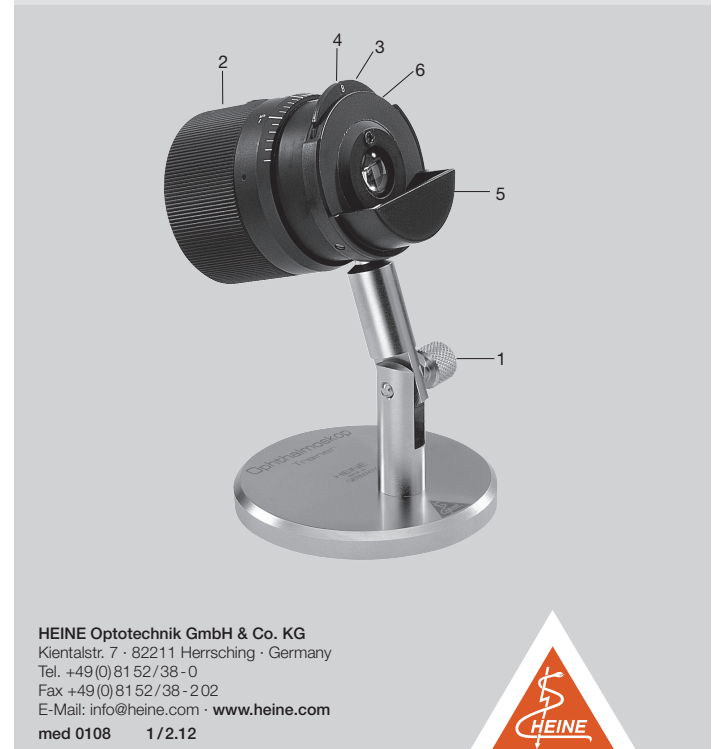
**Ophthalmoscope examination:** Select the desired refractive error and pupil size. Choose the required aperture of your ophthalmoscope and switch the instrument on. Approach the ophthalmoscope trainer up to about 10mm until you can see the red retinal reflex. Rotate the lens wheel of your ophthalmoscope until you see a focused image.

When using an indirect ophthalmoscope, approach the trainer with the hand-held ophthalmoscope loupe so that an enlarged image of the aperture becomes visible. Continue towards the trainer to view the retinal image. To view different sectors of the retina, move your head and the loupe in co-ordination around the trainer. The aperture serves as the axis of rotation. The smaller the aperture selected, the stronger the resulting corneal reflex. Viewing through the smallest 2mm aperture is only possible with a good-quality ophthalmoscope.

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### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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**HEINE** Optotechnik GmbH & Co. KG