## HEINE Hand-help indirect ophthalmoscope

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### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories.

(Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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## Instructions

### HEINE Hand-help indirect ophthalmoscope

<p>Before using the ophthalmoscope read these instructions carefully and keep them in a safe place for future reference.</p>	
------------------------------------------------------------------------------------------------------------------------------	--

<p><b>Safety information</b></p> <p><b>Range of applications:</b> The HEINE Hand-help indirect ophthalmoscope is designed solely for examination of the eye, particularly the retina.</p> <p><b>Safety in use:</b> Use only those devices as power supplies which are designed for medical applications. Use as little light as possible and always switch off the power supply after use.</p>	
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

<p><b>Using the instrument</b></p> <ul style="list-style-type: none"><li>Attach the instrument to a handle and switch on.</li> <li>Depress the sleeve (5) down as far as the stop and project a beam of light onto a light-coloured surface. The size of the spot of light can be adjusted by rotating the sleeve (5).</li> <li>The red-free filter is selected by pushing the sleeve upwards as far as the green symbol (4).</li></ul>	
-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

#### Changing the bulb

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.**

- Remove the instrument from the handle and pull the bulb out of the socket (6).
- Clean the glass of the new bulb with a soft cloth.
- Replace the bulb in the socket so that the guide (7) engages in the cutout of the socket.

#### Monocular examination

Attaching the loupe:  
The loupe makes examination easier for presbyopes and gives a magnified fundus image:

- Slide the holder (1) onto the cylindrical upper part of the instrument.
- The loupe should be fitted behind and parallel to the vertical face of the prism. The beam should not traverse the loupe.

Using the orbital support:  
The orbital support (3) helps the user to steady the instrument during the examination. Spectacle-wearers can use the extension piece which is delivered with the instrument.

- The extension piece (8) should be inserted sideways into the orbital support so that the guides (9) engage in the slit provided.

<p><b>Monocular examination</b></p> <p>Attaching the loupe: The loupe makes examination easier for presbyopes and gives a magnified fundus image:</p> <ul style="list-style-type: none"><li>Slide the holder (1) onto the cylindrical upper part of the instrument.</li> <li>The loupe should be fitted behind and parallel to the vertical face of the prism. The beam should not traverse the loupe.</li></ul> <p>Using the orbital support: The orbital support (3) helps the user to steady the instrument during the examination. Spectacle-wearers can use the extension piece which is delivered with the instrument.</p> <ul style="list-style-type: none"><li>The extension piece (8) should be inserted sideways into the orbital support so that the guides (9) engage in the slit provided.</li></ul>	
---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

#### Binocular examination

<p>Preparation:</p> <ul style="list-style-type: none"><li>Release the friction clutch (10) on the binocular attachment.</li> <li>Remove the loupe from the instrument and attach the binocular attachment as illustrated.</li> <li>Tighten the friction clutch by rotating the lever provided until the attachment sits firmly on the instrument.</li></ul>	
-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

<p>Adjusting the optics:</p> <ul style="list-style-type: none"><li>Turn the knob (12) to adjust the P.D. to the appropriate value shown on the scale (11).</li> <li>With the instrument switched on, select the desired spot size.</li> <li>Look through the binocular attachment without accomodating at a light-coloured surface at a distance of 30–40 cm. A small object, such as a pencil, must be in focus. (+2 diopter lenses are built into the viewing apertures (13). Spectacle wearers should view through the "long-range" part of bi-focal spectacles).</li> <li>The fine adjustment of the P.D. can then be made by centring the light-spot in each viewing aperture by closing one eye at a time.</li></ul>	
----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

Two replacement plain lenses are included with the instrument. If required the eyepieces can be unscrewed and 2D lenses exchanged for the above.

<p><b>Binocular examination</b></p> <p>Preparation:</p> <ul style="list-style-type: none"><li>Release the friction clutch (10) on the binocular attachment.</li> <li>Remove the loupe from the instrument and attach the binocular attachment as illustrated.</li> <li>Tighten the friction clutch by rotating the lever provided until the attachment sits firmly on the instrument.</li></ul>	
-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

<p>Adjusting the optics:</p> <ul style="list-style-type: none"><li>Turn the knob (12) to adjust the P.D. to the appropriate value shown on the scale (11).</li> <li>With the instrument switched on, select the desired spot size.</li> <li>Look through the binocular attachment without accomodating at a light-coloured surface at a distance of 30–40 cm. A small object, such as a pencil, must be in focus. (+2 diopter lenses are built into the viewing apertures (13). Spectacle wearers should view through the "long-range" part of bi-focal spectacles).</li> <li>The fine adjustment of the P.D. can then be made by centring the light-spot in each viewing aperture by closing one eye at a time.</li></ul>	
----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

All external glass surfaces should be cleaned regularly with a lens cloth. The instrument body and spectacle-protectors can be cleaned with an alcohol wipe. Do not use aggressive cleaning fluids. All parts may be gas sterilised. No part of the instrument should be autoclaved.

<p><b>Cleaning and maintenance</b></p> <p>All external glass surfaces should be cleaned regularly with a lens cloth. The instrument body and spectacle-protectors can be cleaned with an alcohol wipe. Do not use aggressive cleaning fluids. All parts may be gas sterilised. No part of the instrument should be autoclaved.</p>	
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

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