**HEINE S-Frame®**  
**HEINE S-Guard®**

**S-Frame**

**S-Guard**

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0)81 52 / 38 - 0  
Fax +49 (0)81 52 / 38 - 202  
E-Mail: info@heine.com · www.heine.com  
med 2106 1/4.10

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## Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

**For repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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## Instructions for use

### HEINE S-Frame® and HEINE S-Guard®

Before using the HEINE S-Frame® and HEINE S-Guard®, read these instructions carefully and keep them in a safe place for future reference.

#### Safety informations

**Range of applications:** The HEINE S-Frame® and HEINE S-Guard® are designed for use exclusively with HEINE instruments and light sources.

**General safety:** Use only HEINE original replacement parts and accessories.

**Safety in use:** To avoid possible damage to the eyes, avoid looking directly at the sun or a bright light! HEINE declines responsibility for any assembly defects caused by the optician. To avoid possible damage to the frame, avoid frequent bending of the temples.

#### Description:

- a) As a frame with mounting bracket for HEINE Instruments:
- HEINE SIGMA 150® Binocular indirect ophthalmoscope
  - HEINE HR® Binocular loupes
  - HEINE HRP® Binocular loupes
  - HEINE HR-C® Binocular loupes
- b) As eye-protection (Frame without temples), when mounted on the HEINE Professional L Headband fitted with the HEINE mounting bracket for HEINE Instruments:
- HEINE MD 1000® Fiber Optic Headlight
  - HEINE 3S LED HeadLight®
  - HEINE HR®, HRP®, HR-C® Binocular loupes

**How to fit the HEINE SIGMA 150® to the HEINE S-Frame®:** Please follow the instructions supplied with the HEINE SIGMA 150®.

**How to fit HEINE Binocular loupes with the HEINE i-View® to the HEINE S-Frame®:** Screw the HEINE i-View® with your HEINE Binocular loupe to the fixing plate (Fig. 3). Please follow the instructions for adjusting the HEINE Binocular loupes and HEINE LED LoupeLight according to the instructions supplied with these units.

**How to fit HEINE Binocular loupes (HR-C and HR 2,0 x) without HEINE i-View® to the HEINE S-Frame®:** Attach the loupe adaptor to the S-Frame with the screw provided. Screw the binocular loupes to the swivel-mount (Fig.1). Please note that the loupes can be mounted at different heights and that the angle of view can be varied by fitting the HEINE dental adapter (only HR-C and HR 2,0 x). Please follow the instructions for adjusting the HEINE Binocular loupes and HEINE LED LoupeLight according to the instructions supplied with these units.

**Assembly of the S-Guard with other HEINE instruments:** Screw your HEINE S-Guard® to the **Professional L Headband** as in Fig. 2. Screw your HEINE S-Guard® to the **3S LED HeadLight** as Fig. 4. Screw your HEINE S-Guard® to the **MD 1000 F.O. HeadLight** as Fig. 5.

#### Adjustment and Operation

**Retaining Cord:** This stabilises your HEINE S-Frame® and prevents slipping. Push the ends of the cord firmly over the ends of the temples. Adjust the length of the cord to suit your needs.

**Adjusting the nose bridge:** [A] This can be bent into position to suit the anatomy of your nose. By bending the bridge supports you can vary the distance between the lenses and the eyes. This avoids contact between your eyelashes or cheekbones and the frame or corrective lenses.

**Changing the nose pad:** The nose pad is attached to the S-Frame by a screw. To change the nose pad, unscrew the screw and remove the old nose pad.

**Adjusting the temples:** [B] The temple ends can be bent into shape in any direction. Adjust them until they fit comfortably.

**Replacing a temple:** To replace a temple, simply unscrew the hinge screw, insert the new temple and screw in the hinge screw.

**Fitting optional corrective lenses:** Please consult an optician before having corrective lenses fitted. Please show the optician the attached "Instructions for specifying corrective lenses". To fit the corrective lenses (C), insert one side of the snap-in frame (2) into the groove in the S-Frame. (D) Next, insert the other side of the snap-in frame in the second groove.

To remove the snap-in frame with corrective lenses, reverse the above procedure, avoiding damage to the lenses.

**Changing protective lenses:** Hold the S-Frame as in (E) and bend the lens in the direction shown and pull the lens with an audible click out of the frame. To replace, follow the procedure shown in (F) and (G) until the lens clicks into place. If the standard protective lenses are too close to your cheekbones, you can either fit the small protective lenses or adjust the nose bridge (A).

#### Instructions for specifying corrective lenses:

When specifying and fitting corrective lenses, be sure to take into account the following:

- The magnifying optics should be attached to the frame to suit your usual working posture.
- Prismatic errors should be corrected.
- The corrective lenses are mounted at an angle to the optical axis.

Check the correction by means of an eye test with the magnifying optics and the corrective lenses fitted.

The HEINE SIGMA 150® eyepieces include 2 diopter lenses. This enables normal-sighted, emmetropic patients to view the retinal image without effort.

The optical spherical correction is usually max. +/- 5 dpt. Ask your optician or ophthalmologist!

The corrective lenses should be placed as closely as possible to the eyepieces of the magnifying optics.

## Cleaning and Maintenance

Generall: The plastic protective lenses have an anti-reflex coating. Clean them carefully to avoid mechanical or chemical damage.

**Manual Cleaning:** Clean the S-Frame and the S-Guard with the enclosed microfibre cloth. We recommend the HEINE cleaning spray C-000.32.542 as a cleaning agent. **Disinfection wipes/Spray disinfection:** Apply disinfectants that have been approved for medical devices made from synthetic material and glass.

We recommend the application of isopropyl alcohol at maximum 70% concentration.

**Warning:** Immersion disinfection, machine cleaning/disinfection, ultrasound cleaning and sterilisation are not permitted.

## Technical Specification

Weight: 25 g  
Protective lenses: Plastic, anti-reflex coated  
Frame: Nickel-free

## Accessories and Spare Parts

One pair of protective lenses, large (lg)	<b>C-000.32.306</b>
One pair of protective lenses, small (sm)	<b>C-000.32.307</b>
Retaining cord	<b>C-000.32.305</b>
Dental adaptor	<b>C-000.32.528</b>
Snap-in frame for corrective lenses	<b>C-000.32.309</b>
HEINE Cleaning Spray	<b>C-000.32.542</b>
S-Frame replacement part set with side rubbers, nose pads and screw	<b>C-000.32.308</b>

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