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CE 93/42/EWG/CEE/EEC

## HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0) 81 52 / 38 - 0 · Fax +49 (0) 81 52 / 38 - 202  
E-mail: info@heine.com · www.heine.com  
med 2196 1 / 11.06

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## Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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## Instructions

### HEINE Lamp handle

Before using the HEINE lamp handle read these instructions carefully and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

### Safety information

**Range of applications:** The HEINE lamp handle is designed exclusively for use with HEINE F.O. diagnostic instruments.

**General safety:** Repairs may only be carried out by qualified personnel.

**Safety in use:** Do not use in rooms where an explosion hazard is present.

### Installation and use

– Plug the lamp handle plug (1) into the 6 V transformer socket and switch on.

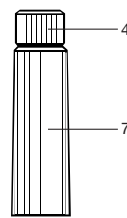
If the connecting cord includes a switch (3), this can be used to switch on and off, leaving the transformer permanently switched on.

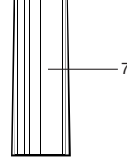
– To disconnect the plug (1) from the transformer pull and rotate gently at the same time.

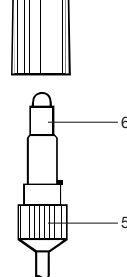
– **To connect an instrument:** Hold the handle in one hand and use the other to insert the the 3 male prongs of the connector into the guides of the handle head. Press the instrument into the handle until it clicks into place.

– **To remove an instrument:** hold the handle in one hand and rotate the knurled upper ring (4) anti-clockwise. With the other hand remove the instrument by pulling gently.

### Changing the bulb

 4 – Press the bulbholder (5) into the outer handle (7). Twist the bayonet connector to the left and pull out the bulbholder.

 6 – Unscrew the bulb (6) from the bulbholder and replace with a new bulb of same type. We can only guarantee the safety and performance of this unit if the correct HEINE bulb is used.

 5 – Press the bulbholder into the outer handle and turn to the right until the bayonet connector engages.

### Cleaning

Before cleaning, always disconnect the unit from the mains supply. Wipe clean with a damp cloth and soap solution.

### Technical specification

#### Halogen bulb of bulb handle:

Nominal voltage: 6 V  
Current consumption: 0.7 A  
Working life: approx. 100 hours  
Colour temperature: approx. 2900 K

### Accessories and spare parts

Lamp handle with on/off switch and 6 V halogen bulb	<b>X-04.99.607</b>
same, without switch	<b>X-04.99.606</b>
HEINE XHL® Xenon Halogen bulb	<b>X-04.88.060</b>

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