
HEINE Mader Proctoscope

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 13696 1/2.12

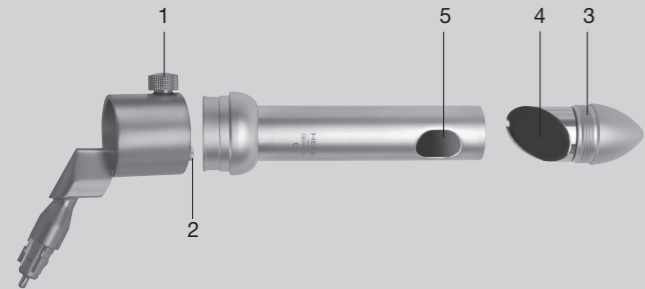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DEUTSCH

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ENGLISH

HEINE Mader proctoscope

Read these instructions carefully before using the HEINE Mader proctoscope and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: The HEINE Mader proctoscope is designed only for examination and treatment of the anal passage.

Safety in use: Use only light sources which are approved for use in a medical environment.

Using the instrument

- Connect the instrument head to a handle.
- Insert the tube into the head as far as the stop.
- Rotate the tube until the viewing window is in the desired position. Secure the tube with the screw (1).
- The cone of the obturator and the tube should be lubricated with a suitable lubricant.

Adjusting the mirror assembly

Insert the mirror assembly (3) into the tube so that the groove (4) coincides with the pin (5). Screw in the mirror assembly as far as possible.

Changing the bulb

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

Unscrew the bulb from the head (2). Screw in a new bulb.

Cleaning and sterilization

Do not soak the head in cleaning solutions.

Avoid scratching the mirror.

1. Cleaning

- Clean immediately after use to avoid drying-on of deposits. Remove the tube from the head and unscrew the mirror assembly (3).
- Wipe the head with a cloth soaked in alcohol or detergent.
 - Clean the tube and mirror assembly with a soft brush and water or detergent solution.

2. Disinfection

After cleaning as in 1. above;

- Wipe with a disinfectant solution.
- Soak the tube and mirror assembly in disinfectant solution.
- Dry before further use.

3. Sterilization

- Gas sterilize with ethylene oxide.
- The tube and mirror assembly can be autoclaved.

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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