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CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-Mail: info@heine.com · www.heine.com
med 4997 1/4.09

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DEUTSCH

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Instructions

ENGLISH

HEINE NT 200® charger and rechargeable handles

Before using the HEINE NT 200® read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: The NT 200 is designed exclusively for charging HEINE rechargeable handles.

General safety: Under no circumstances should the mains transformer housing be opened. Repairs may only be carried out by qualified personnel (exception: changing the rechargeable battery).

Safety in use: The mains supply voltage shown on the identification plate of the mains transformer must correspond to the local mains voltage. NT 200 must not be used in rooms where an explosion hazard is present.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

General Information

Installation and use

- Plug the mains transformer into the mains and the charger is ready for use.
- New handles should be charged for at least 12 hours the first time.
- NT 200 assures that batteries are kept in good condition irrespective of handle size.
- Ideally, the rechargeable handle should always be replaced and stored in the charger. Handles cannot be overcharged.
- The green LED in front of each charging compartment indicates that a charging current is flowing.
- If the second charging compartment is not in use it can be sealed off by the dust cap which is supplied with the unit.

Handles for diagnostic instruments

Automatic connector: Hold the handle in one hand and use the other to insert the 3 male prongs of the connector into the guides of the handle head (1). Press the instrument into the handle until it clicks into place. To remove the instrument, hold the handle in one hand and rotate the knurled upper ring (2) anti-clockwise. With the other hand remove the instrument by pulling gently.

Threaded connector: The alpha handle has a threaded connector (3) into which all alpha instruments can be screwed.

Brightness control: To switch on, rotate the control (4) clockwise - with the BETA NT 3.5 V handle press in the button (5) at the same time. When „1“ shows in the window (6) you have selected maximum brightness. Otherwise, you can select any position between „0“ and „1“.

Changing the battery: The battery can be removed and replaced by unscrewing the bottom insert (7).

Using non-rechargeable batteries: With the small battery handle and the alpha handle you can replace the rechargeable battery with 2 dry batteries IEC LR 06

Cleaning and Maintenance

The NT 200 must only be cleaned or disinfected manually. Disinfection by spraying, immersion, cleaning in an ultrasonic bath, or sterilisation are not allowed. Cleaning must be carried out by sufficiently qualified personnel.

Manual Disinfection and Cleaning

For disinfecting cleaning, disinfectant wipes are recommended. The manufacturers of the disinfectant must have approved the substances for use with plastics. The disinfectant manufacturer's instructions must be followed. Disinfectants containing more than 70% isopropyl alcohol must not be used.

Handles for fiber optics laryngoscopes

(please refer to separate instructions)

Technical specification

Input (see nameplate):	Voltage 100 V/50-60 Hz 120 V/60 Hz 240 V/50 Hz
Power consumption:	max. 7 VA
Charging current:	holding charge: approx. 60 mA
fast charge:	approx. 300 mA
Fuse:	thermal fuse in transformer
Operation:	continuous
Classification:	Type BF
Safety class:	double insulated
Dimensions:	129 x 77 x 61 mm
Weight:	400 g
Ambient temperature:	+10 ° to +40 °C

Accessories and spare parts

NiMH Spare battery for all BETA 3,5 V rechargeable handles	X-002.99.382
Li-ion spare battery for all BETA 3,5 V rechargeable handles	X-007.99.381
K3Z NiMH spare battery for short BETA Laryng. Handle 3,5 V	X-000.99.393
Large adaptor ring for NT 200 charger	X-000.99.084
Reduction sleeve and adaptor ring	X-000.99.083

Applicable standards: IEC 60 601-1

Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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