

HEINE GAMMA 20



med 9791 3/2.01

Instructions for use

Important Notice: Do not apply localized pressure to the membrane!



Using the instrument

Press the chest-piece gently on the patient's chest. Total membrane coverage is not essential.

Guarantee Card

This instrument is guaranteed for five years. Any defects which become apparent during this period will be repaired free of charge provided the instrument has been used normally and in accordance with the Instructions for use. Further claims cannot be accepted. The guarantee period begins from the date of purchase.


Bought from _____

HEINE USA LTD.
One Washington Center
Suite 555, One Washington Street
Dover, New Hampshire USA 03820
Tel. (603) 742-7103
Fax (603) 742-7217
Toll Free 8003 67 48 72
e-mail: info@heineweb.com

**HEINE INSTRUMENTS
(CANADA) LTD.**
20 Steckle Place, Unit 3
Kitchener, Ont. N2E 2C3
Tel. 519-895-1020
Fax 519-895-1022
e-mail: info@heineweb.com

Cleaning and maintenance

Changing the ear tips: Slide off the old tips and push on the new set up to the engraved stop.

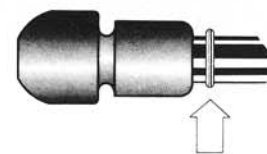

Cleaning: Wipe the instrument with a cloth soaked in alcohol or other non-aggressive cleanser.

Sterilization: Ethylene oxide is recommended. Thermal sterilization and soaking in sterilizing solutions are not recommended.

Accessories

Set of super-soft ear plugs

R-00.09.076