**HEINE** QUALITY  
MADE IN GERMANY

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## HEINE Finoff transilluminator

ENGLISH

 Read these instructions carefully before using the HEINE Finoff transilluminator and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

### Safety information

**Range of applications:** The HEINE Finoff transilluminator is used for general transillumination of tissue and, more specifically, for scleral transillumination. It can be combined with a blue filter for illuminating the eye.

**Safety in use:** Only power sources which are approved for medical use should be used. Apply the transilluminator to the eye only when the protective cap is fitted.

**Please note:** The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

### Instructions for use

We recommend that the examination be carried out in a darkened room. Ensure that the surface of the eye is anaesthetized before scleral transillumination.

- Connect the HEINE Finoff transilluminator to the handle and switch on.
- For scleral transillumination, rest the distal end with its protective cap flat on the bulbus without exerting pressure. Please note that intense light can be uncomfortable for the patient and may make it difficult to recognize areas of thin tumor growth.
- For transillumination of skin tissue use the instrument without the protective cap.
- To illuminate the eye, fit the blue filter and move the instrument as close as necessary to the eye without touching it.

### Changing the bulb

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.**

First remove the instrument from the handle. Grip the used bulb by the collar (1) or the thread (2) and pull it out of the instrument.

### Care and maintenance

After removing the protective cap or blue filter, you can clean the instrument with a cloth soaked in a mild cleansing agent or with a disinfectant solution. Gas sterilization is also possible. Soaking in solutions and autoclaving are not possible. The protective cap and blue filter can be cleaned and sterilized by any conventional means.

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## Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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