
HEINE Focalux® Handleuchte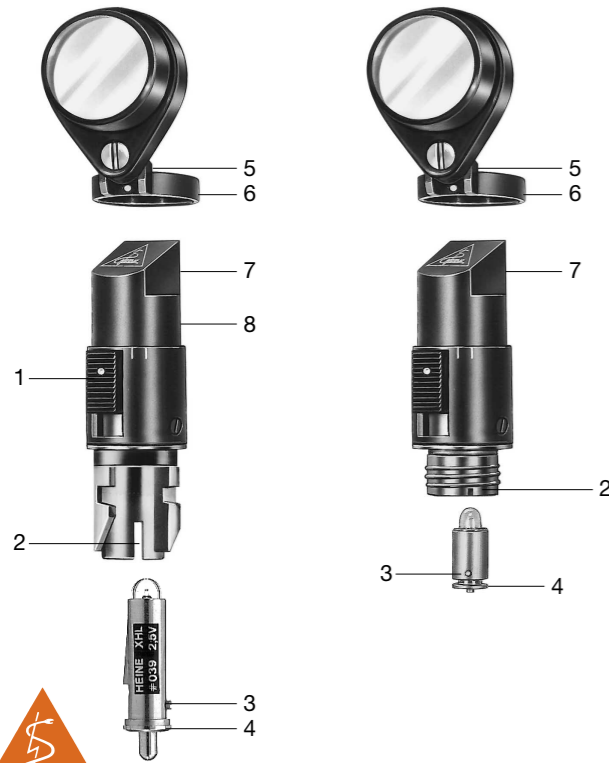

Kientalstrasse 7 · D-82211 Herrsching · Germany
Telefon 0 81 52/38-0 · Telefax 0 81 52/38-202
e-mail: info@heineopto.de · Internet: www.heine.com

CE 93/42/EWG/CEE/EEC

med 6796 1/4.04

DEUTSCH

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HEINE Focalux® Handleuchte

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Instructions

HEINE Focalux® hand lamp

Before using the HEINE Focalux® hand lamp read these instructions carefully and keep them in a safe place for future reference.

Safety information

Range of applications: The Focalux hand lamp is designed for illuminating the oral cavity, the throat, nose, skin surface and for examining the anterior segment of the eye. When combined with an ophthalmoscopy lens it can also be used for indirect ophthalmoscopy.

Safety in use: Use only those devices as power supplies which are designed for medical applications.

Method of operation

Attach the instrument to a handle.

Switch on the illumination and direct the light emerging from the prism (7) onto a white surface.

Using the slide (1) you can adjust the focusing of the light from parallel (slide at bottom) to convergent rays.

Use of magnifying lens +3D

The lens is mounted on the cylindrical part (8) of the hand lamp. Ensure that the shoulder (5) of the lens ring (6) is pointing upward.

The lens enables the presbyopic examiner to carry out examinations in comfort and produces a magnified fundus image. Always position the lens at the center of the hand lamp.

Changing bulbs

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

Remove the instrument from the handle.

Grip the collar (4) of the bulb between your thumb and index finger nails and remove the bulb by pulling it downward.

Wipe the dome of the new bulb on a clean cloth and then insert it so that the adjusting pin (3) fits into the recess (2).

Spare parts and accessories

HEINE XHL® Halogen bulb 2,5 V	X-01.88.039
HEINE XHL® Halogen bulb 3,5 V	X-02.88.091
HEINE XHL® Halogen bulb 2,5 V for alpha Focalux	X-01.88.042
Detachable swivel loupe +3D	C-00.14.109

Warranty

We guarantee this device (except bulbs) for two years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of normal wear and tear, incorrect use, the use of non-original HEINE spare parts, or non-authorized modification or repairs. Further claims are excluded.

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