HEINE QUALITY
MADE IN GERMANY

∴ HEINE visionPRO video laryngoscope

 **HEINE Optotechnik GmbH & Co. KG**
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 235340 2022-11-21

V-200.00.107

HEINE visionPRO video laryngoscope

DEUTSCH	3
ENGLISH	13
FRANÇAIS	23
ESPAÑOL	33
ITALIANO	43
SVENSKA	53
NEDERLANDS	63
DANSK	73
NORSK	83
SUOMI	93
PORTUGUÊS	103

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 Please read and follow these instructions for use and keep them for future reference.

Intended use

visionPRO video laryngoscope

The HEINE visionPRO video laryngoscope with the visionPRO Mac blade supports laryngoscopy and positioning of an endotracheal tube.

The live image data on the product display assists with indirect laryngoscopy.

The internally powered medical device is intended for transient use in a professional healthcare facility. The healthcare professional must have adequate intubation experience.

visionPRO charging station

The HEINE visionPRO CS1 is a charging station designed exclusively for charging the internal power supply of the visionPRO video laryngoscope.

The product must only be used by qualified medical professionals and in medical healthcare facilities.

Warnings and safety information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

Product overview

Video laryngoscope

Fig. 1:
HEINE visionPRO Video laryngoscope

Charging station

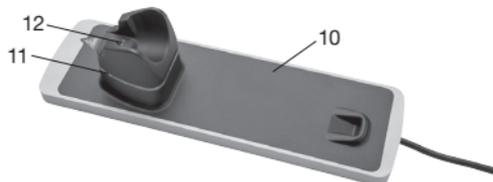
Fig. 2:
visionPRO CS1 Charging station

- 1 Display unit
- 2 Handle
- 3 Camera module with illumination
- 4 Display
- 5 Camera arm
- 6 Disposable blade
- 7 On/Off button
- 8 Fixing clip
- 9 Fixing hook
- 10 visionPRO CS1
- 11 Illuminated ring
- 12 Charging pins

Mounting the visionPRO CS1 charging station

- ⚠ To ensure all-pole disconnection from the mains at all times, the ME device must be installed so that the switch mode power supply is accessible and disconnectable.
- 🔌 Disconnect the mains plug from the mains when assembling or disassembling the ME device.

The visionPRO CS1 is designed for the following mounting options:

- Setup on a table or a similar stable surface (fig. 3)
- Direct wall mounting (Fig. 4)
- Wall mounting using a specific adapter on standard rails

Fig. 3:
visionPRO CS1 with an inserted
visionPRO video laryngoscope
in the table position

Fig. 4:
visionPRO CS1 with an inserted
visionPRO video laryngoscope
in the wall position

Cable routing

Loosen the screws for fastening the identification plate and route the cable of the power supply unit according to the desired fastening option (fig. 5). Tighten the screws again after successfully aligning the cable.

Fig. 5:
Rear of the visionPRO CS1
with the various cable routing options.

Setting up the visionPRO CS1 charging station

Primary plug

The primary country adapter of the power supply is interchangeable, this enables worldwide operation. Fig. 6 and fig. 7 describe the procedure of changing the plug.

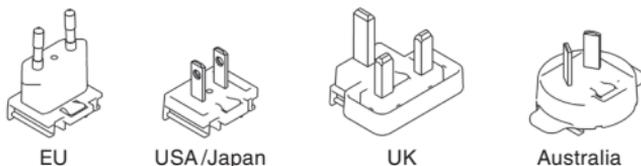
Fig. 6

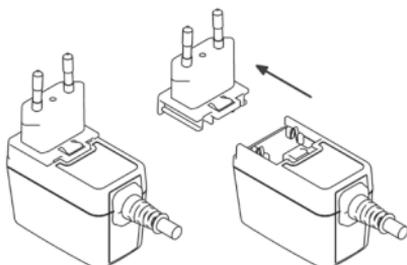
Fig. 7

⚠ After you have checked the mains voltage indicated on the nameplate of the device, connect the mains supply plug to the mains supply.

Set up or mount the visionPRO CS1 in such a way that it is easy to disconnect it from the mains.

The green LED on the mains plug indicates that the power supply unit is ready for operation.

To put the power supply out of operation just disconnect the power supply from the mains supply.

The visionPRO CS1 is intended for setting up and use in medical areas outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see fig. 8).

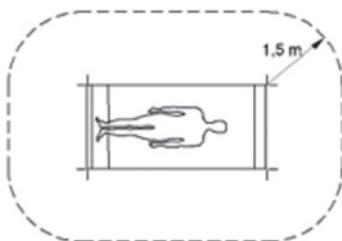
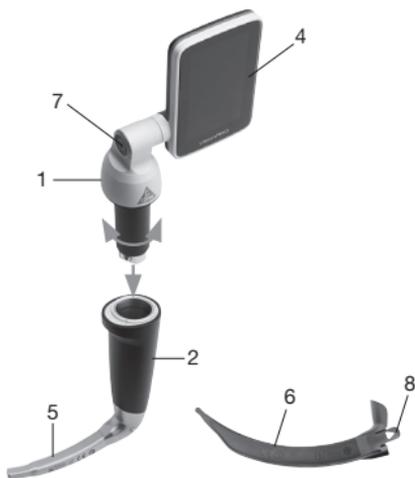
Fig. 8

Setting up the visionPRO video laryngoscope

 Charge the video laryngoscope fully before using it for the first time.

 Wear gloves when using the video laryngoscope.

Insert the display unit (1) into the handle (2) and, applying slight pressure, twist the components until it audibly clicks into place.

Remove the disposable blade (6) from the packaging, slide it completely over the camera arm (5) of the handle (2) and make sure it clicks into place (you will hear it 'click').

Make sure that the blade is firmly and securely in place.

Switch on the device by pressing the On/Off button (7) briefly.

Adjust the angle of the display (4) as desired.

Fig. 9:
Assembling the display unit and handle

Operating the visionPRO video laryngoscope

 Following reprocessing, the device may only be reused when it is completely dry.

Check the blade is positioned both correctly and firmly before using the video laryngoscope.

We strongly recommend having a ready-to-use secondary system of laryngoscopes at hand.

Remove the blade from the video laryngoscope only outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see Fig. 8).

The maximum temperature at the application point at an ambient temperature of 35 °C and in continuous operation is 41.3 °C.

 If the video function fails during intubation, the device can also be used as a direct laryngoscope. Afterwards, return the unit to HEINE Service.

In the upper display section of the live image, the attached blade can be recognized. This provides an orientation.

Insert the video laryngoscope into the patient with the attached disposable blade. Use an appropriate size of blade that suits the patient. Intubate the patient.

After intubation, remove the video laryngoscope from the patient with the disposable blade. Switch off the device following use. Press and hold the On/Off button until the orange countdown has elapsed.

Release the fixing clip (8) and remove the disposable blade from the camera arm.

If you do not switch off the video laryngoscope after use, the device will automatically switch off after 15 minutes. A prompt with a 30-second countdown appears on the display in this regard. Press the On/Off button if you would like to interrupt the automatic switch-off process.

Should a reset be necessary, it can be reset by pressing the on/off button for a period of 10 seconds.

The video laryngoscope's charge level indicator

The charge level indicator is shown by a battery icon in the top part of the display. A distinction is made between the following charge level indicators:

Battery bar	Meaning	Action
Solid green 	The battery bar is green when the remaining battery capacity is between 21 % and 100 %.	No compulsory action is necessary.
Solid yellow 	The battery bar is yellow when the remaining battery capacity is between 11 % and 20 %.	The device should be charged promptly.
Solid red 	The battery bar is red when the remaining battery capacity is between 6 % and 10 %.	The device must be charged.
Flashing with a red exclamation mark 	The battery bar flashes when the remaining battery capacity is between 5 % and 0 %.	The device must be charged. Further intubation is not possible.

Operating the visionPRO CS1 charging station

To charge, place or hang the visionPRO video laryngoscope or display unit DS1 in the visionPRO CS1 as shown in Fig. 3 or 4. The blue illuminated ring (11) on the visionPRO CS1 (10) indicates that the visionPRO video laryngoscope is inserted or hooked in and the charging station's output is switched on. To ensure that the device is being charged, the illuminated ring on the On/Off button (7) of the visionPRO video laryngoscope must light up in addition to the charging station, and the display (4) must briefly show the charging state.

During the charging process, the illuminated ring on the On/Off button of the visionPRO video laryngoscope flashes blue. The illuminated ring on the On/Off button lights up solid blue as soon as the visionPRO video laryngoscope is fully charged. The charge level in percent is shown for a few seconds on the visionPRO video laryngoscope's display when:

- the visionPRO video laryngoscope is inserted or hooked into the visionPRO CS1
- the On/Off button of the visionPRO video laryngoscope is pressed
- the visionPRO video laryngoscope is removed from the visionPRO CS1.

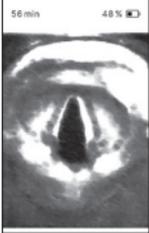
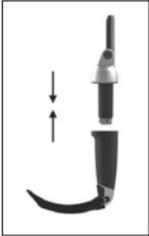
Troubleshooting

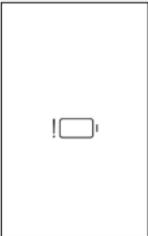
If the visionPRO video laryngoscope is not charging, check that the mains plug is connected to the mains and that the visionPRO video laryngoscope is correctly inserted. If necessary, reset the charging process by removing the visionPRO video laryngoscope from the visionPRO CS1 for a few seconds, unplugging the visionPRO CS1 and plugging it back in, and inserting the visionPRO video laryngoscope. If this error persists, check the electrical contacts for soiling and, if necessary, clean them carefully with a clean and lint-free cleaning cloth.

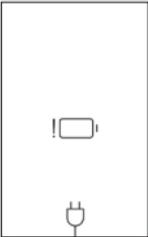
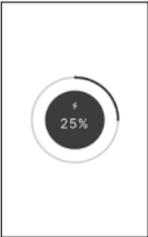
If you do not notice any improvement, please contact the manufacturer.

Screen displays

Video laryngoscope

Screen	Meaning	Action
<p>Start screen</p> 	<p>The device switches on.</p>	–
<p>Live image</p> 	<p>The device displays the video image in real time.</p> <p>The remaining operating time is shown in minutes next to the charging indicator in the top display area.</p>	–
<p>Connection screen</p> 	<p>This animation indicates that the handle and the display unit are not connected with one another.</p>	<p>Connect the handle and the display unit.</p>
<p>Video error screen</p> 	<p>This screen shows a video error.</p>	<p>Disconnect the handle and the display unit and reconnect the two components. The live image is then displayed again.</p> <p>If this error persists, check the electrical contacts for soiling and, if necessary, clean them carefully with a clean and lint-free cleaning cloth.</p> <p>If you do not notice any improvement, take the device out of operation and contact the manufacturer.</p>

Screen	Meaning	Action
<p data-bbox="52 93 225 113">Switch-off screen</p> 	<p data-bbox="299 93 495 136">The device switches off.</p>	<p data-bbox="555 93 961 136">Press and hold the On/Off button until the device switches off completely.</p>
<p data-bbox="52 394 273 438">Switch-off screen when the battery is flat</p> 	<p data-bbox="299 394 534 493">The device switches off. When the battery is low, a battery symbol is also displayed.</p>	<p data-bbox="555 394 926 464">Press and hold the On/Off button until the device switches off completely. The device should be charged.</p>
<p data-bbox="52 738 256 781">Automatic switch-off screen</p> 	<p data-bbox="299 738 515 808">The device switches off automatically after 15 minutes if not used.</p>	<p data-bbox="555 738 971 781">Press the On/Off button if you would like to interrupt the automatic switch-off process.</p>
<p data-bbox="52 1058 273 1102">Switch-off screen when the battery is flat</p> 	<p data-bbox="299 1058 530 1102">Switch-off screen when the battery is low.</p>	<p data-bbox="555 1058 958 1102">The device must be charged before it can be used again.</p>

Screen	Meaning	Action
Switch-on screen when the battery is flat 	The battery is flat and the device cannot be switched on.	The device must be charged before it can be used again.
Charging screen 	The device displays the charge level.	–

Hygienic reprocessing

The instruction is available:

- online at www.heine.com
- in a paper version which you can request from the address listed

After each use, carry out hygienic reprocessing.

 The blade is a disposable product. It must not be reused under any circumstances as serious mechanical damage may occur.

Maintenance

There is no regular maintenance required.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 5 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Charging station: The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

The disposable blade has a limited shelf life. Check this on the disposable blade packaging. Charge your device during longer storage periods to protect the battery from discharge or store the battery and the device separate from each other.

Let the device accommodate to ambient conditions when it is cold.

The device may steam up if the environment is very warm or the device cools down.

If the device is steamed up, wait until the device is ready for use or wipe the affected areas.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Please check the display of the video laryngoscope during the charging process to see if the device is charging.

General Warnings

 Only trained personnel may use the video laryngoscope for intubation.

Check that the device is working properly before each use. In particular, check the illumination and the live image. Do not use the device if you notice moisture in it or damage.

Do not use the device in fire- or explosive risk area.

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not use the device including its accessories outdoors.

Please ensure that a HEINE disposable blade is on the handle before using the video laryngoscope.

Do not use the device for transillumination of tissue.

Do not shine directly in the eyes to avoid dazzle, especially if the pupils are dilated.

Do not use the laryngoscope for eye reflex testing.

Check the disposable blade for sharp corners, edges and rough surfaces before use.

Switch off the device following use.

Do not short-circuit the device!

Never use a power supply with damaged housing or cable

Heating during operation is normal and harmless.

All load-bearing elements of the device are to be loaded only in the manner intended.

Dropping the unit or mounting accessories can cause injury and damage to property.

Check at regular intervals whether the wall mounting and the mounting accessories are in proper condition. In particular, make sure that all screws are firmly tightened.

Danger of fatal electric shock!

Use power supply in dry environment only. Avoid splashing water. Do not immerse in liquids.

Never modify a power supply or open its housing.

Connect the power supply to a regular outlet only.

Protect connectors from moisture.

Never carry the power supply by its electrical cable or pull at the cable.

For separating the connection always pull on the plug, never on the cable.

Cables must be installed safely to avoid tripping over.

Disposal

Carry out reprocessing of the video laryngoscope before disposal.

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.	
Statement for the operational environments	Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
	The supply voltage quality should be that of a typical hospital environment.
	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
	NOTE: The EMISSIONS characteristics of this equipment (visionPRO) make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR 11	Group 1 Class B (charging station) Class A (video laryngoscope)
Conducted emissions (EN 55011/CISPR 11)	Passed
Radiated emissions (EN 55011/CISPR 11)	
Harmonic current emissions (IEC 61000-3-2)	
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)	
Immunity	See attached immunity test levels

Immunity test levels

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ± 8 kV Air Discharge: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80–2700 MHz 80 % AM at 1 kHz	
Electrical fast transients / bursts (IEC 61000-4-4)	± 2 kV 100 kHz repetition frequency	
Surges (IEC 61000-4-5)	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)	3 V 0.15 MHz – 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: ± 5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50 Hz or 60 Hz	30 A/m; 60 Hz
Voltage dips (IEC 61000-4-11)	0 % U _T ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U _T ; 1 cycle and 70 % U _T ; 25/30 cycles Single phase: at 0°	
Short interruptions (IEC 61000-4-11)	0 % U _T ; 250/300 cycles	

Technical specification

Environmental conditions for operation	+5 °C to +35 °C 10 % to 80 % non-condensing 700 hPa to 1060 hPa
Environmental conditions for storage	-20 °C to +50 °C 0 % to 95 % non-condensing 700 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 0 % to 95 % non-condensing 500 hPa to 1060 hPa
Power supply (video laryngoscope)	Internally powered (3,7 dc, 2.300 mAh)
Power supply (charging station)	100 – 240 Vac; 50–60 Hz
Power consumption (charging station)	300 mA – 150 mA

Accessories (List of compatible blades)

Article	Article number	Overall length	Blade length	Distal width	Material
visionPRO Mac 3 (10x)	F-000.22.243	135 mm	117 mm	14,5 mm	ECO ABS
visionPRO Mac 3 (50x)	F-000.22.253				
visionPRO Mac 4 (10x)	F-000.22.244	151 mm	132 mm	15,5 mm	
visionPRO Mac 4 (50x)	F-000.22.254				

Erläuterung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati
Förklaring av symboler som används
Verklaring van de gebruikte symbolen
Forklaring af de anvendte symboler
Symbolforklaring
Käytettyjen symbolien selitys
Explicação dos símbolos utilizados

<p>CE</p>	<p>CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizinprodukteverordnung (EU) 2017/745. The CE mark indicates that the product complies with the medical device regulation (EU) 2017/745. Le marquage CE indique que le produit est conforme ou au règlement (UE) 2017/745 relatif aux dispositifs médicaux. La marca CE indica que el producto cumple el Reglamento europeo sobre productos sanitarios (UE) 2017/745. Il marchio CE indica la conformità con il regolamento sui dispositivi medici (UE) 2017/745. CE-märkning markerar en överensstämmelse med förordningen om medicinska produkter (EU) 2017/745. CE-markering duidt de overeenstemming aan met de verordening betreffende medische hulpmiddelen (EU) 2017/745. CE-mærkningen angiver overensstemmelse med forordningen om medicinsk udstyr (EU) 2017/745. CE-merket angir at produktet er i samsvar med forskriften om medisinsk utstyr (EU) 2017/745. CE-merkintä tarkoittaa, että laite lääkinnällisiä laitteita koskevan asetuksen (EU) 2017/745 kanssa. O símbolo CE identifica a concordância com o regulamento de Dispositivos Médicos (EU) 2017/74.</p>
<p>REF</p>	<p>Katalog- oder Bestellnummer Catalogue- or order number Numéro de catalogue ou de commande Número de catálogo o de pedido Codice catalogo e di dell'ordine numero Katalog- eller Beställningsnummer Catalogus- of Bestelnummer Katalog- eller Ordrenummer Katalog- eller bestillingsnummer Luettelo- tai viitenumero Número de catálogo ou pedido</p>
<p>LOT</p>	<p>Losnummer (entspricht dem Herstellungsdatum) Lot number (corresponds to the date of manufacture) Numéro de lot (correspond à la date de fabrication) Número de lote (corresponde a la fecha de fabricación) Numero di lotto (corrisponde alla data di produzione) Individuellt nummer (motsvarar tillverkningsdatumet) Lotnummer (komt overeen met de productiedatum) Partnummer (komt overeen met de productiedatum) Lösnummer (tilsvarende produktionsdatoen) Eränumero (vastaa valmistuspäivää) Número do lote (corresponde à data de fabricação)</p>

	<p>Hersteller Manufacturer Fabricant Fabricante Produttore Tillverkare Fabrikant Producent Produsent Valmistaja Fabricante</p>
	<p>Hersteldatum Date of manufacture Date de fabrication Fecha de fabricación Data di produzione Tillverkningsdatum Productiedatum Produktionsdato Produksjonsdato Valmistuspäivä Data de fabricação</p>
	<p>Getrennte Sammlung von Elektro- und Elektronikgeräten (Europäische WEEE Richtlinie) Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE) Tri sélectif des appareils électriques et électroniques (Directive européenne DEEE) Desechado separado de aparatos eléctricos y electrónicos (Directiva Europea RAEE) Raccolta differenziata di apparecchi elettrici ed elettronici (direttiva europea RAEE) Separat insamling av elektriska och elektroniska apparater (det europeiska WEEE-direktivet) Gescheiden inzameling van elektrische en elektronische apparaten (Europese AEEA richtlijn). Separat indsamling af elektrisk og elektronisk udstyr (det europæiske WEEE-direktiv) Produkter med dette symbolet skal ikke avhendes sammen med vanlig husholdningsavfall, men krever separat kassering i henhold til lokale bestemmelser. (European Waste Electrical and Electronic Equipment Directive, WEEE) Sähkö- ja elektroniikkalaitteille tarkoitettu erillinen keräyspiste (eurooppalainen WEEE-standardi) Coleção separada de aparelhos elétricos e eletrônicos (Diretrizes Europeias WEEE)</p>

	<p>Zulässiger Temperaturbereich in °C für Lagerung und Transport Temperature limits in °C for storage and transport Plage de température admise en °C pour le stockage et le transport Rango de temperatura permitida en °C para almacenar y transportar el producto Temperatura ammessa in °C per conservazione e trasporto Tillåtet temperaturintervall i °C för lagring och transport Toegestane temperaturen in °C voor opslag en transport Tilladt temperaturområde i °C ved opbevaring og transport Temperaturbegrensning i °C for oppbevaring og transport Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°C) Limite de Temperatura em °C para armazenamento e transporte</p>
	<p>Zulässiger Temperaturbereich in °F für Lagerung und Transport Temperature limits in °F for storage and transport Plage de température admise en °F pour le stockage et le transport Rango de temperatura permitida en °F para almacenar y transportar el producto Temperatura ammessa in °F per conservazione e trasporto Tillåtet temperaturintervall i °F för lagring och transport Toegestane temperaturen in °F voor opslag en transport Tilladt temperaturområde i °F ved opbevaring og transport Temperaturbegrensning i °F for oppbevaring og transport Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°F) Limite de Temperatura permitida em °F para armazenamento e transporte</p>
	<p>Zulässige Luftfeuchtigkeit für Lagerung und Transport Humidity limitation for storage and transport Humidité admise pour le stockage et le transport Humedad del aire permitida para almacenar y transportar el producto Umidità atmosferica ammessa durante il trasporto e la conservazione Tillåten luftfuktighet för transport och lagring Toegestane luchtvochtigheid voor opslag en transport Tilladt luftfugtighed ved opbevaring og transport Fuktighetsbegrensning for oppbevaring og transport. Sallittu ilmankosteus kuljetuksen ja varastoinnin aikana Umidade do ar admissível para o armazenamento e transporte</p>
	<p>Zulässiger Luftdruck für Lagerung und Transport Pressure limitation for storage and transport Pression atmosphérique admise pendant le transport et le stockage Presión de aire permitida para almacenar y transportar el producto Pressione atmosferica ammessa durante il trasporto e la conservazione Tillåten lufttryk för lagring och transport Toegestane luchtdruk voor opslag en transport Tilladt lufttryk ved opbevaring og transport Trykkgrensning for oppbevaring og transport Sallittu ilmanpaine kuljetuksen ja varastoinnin aikana Pressão do ar admissível para o armazenamento e transporte</p>

	<p>Vorsicht Bruchgefahr! Fragile, handle with care! Fragile ! Manipuler avec soin Atención. Frágil. Attenzione: pericolo di rottura! Försiktigt! Risk för brott Voorzichtig, kans op breuk! Forsigtig, risiko for brud! Ømtålig, behandles forsigtig! Varo särkymisvaaraa! Perigo de quebra!</p>
	<p>Trocken lagern! Keep dry! Conserver au sec ! Conservar en un lugar seco! Evitare ambienti umidi! Förvaras torr! Droog bewaren! Opbevares tørt! Hold tørt! Säilytetään kuivassa paikassa! Armazenar em ambiente seco!</p>
	<p>Gebrauchsanweisung verbindlich befolgen. (Hintergrundfarbe: blau, Vordergrundfarbe: weiß) Follow instructions for use! (Background color: blue, foreground color: white.) Suivre le mode d'emploi. (Couleur de fond : bleu ; couleur du premier plan : blanc) Seguir obligatoriamente las instrucciones de uso. (Color de fondo: azul, color de primer plano: blanco) Attenersi obbligatoriamente alle istruzioni per l'uso. (Colore dello sfondo: blu, colore in primo piano: bianco) Bruksanvisningen ska alltid följas. (Bakgrundsfärg: blå, förgrundsfärg: vit) De gebruiksaanwijzing is bindend en dient gevolgd te worden. (achtergrondkleur: blauw, voorgrondkleur: wit) Følg altid brugsanvisningen. (Baggrundsfarve: Blå; forgrundsfarve: Hvid) Følg bruksanvisningen! (Bakgrunnsfarge: blå, forgrunnsfarge: hvit.) Käyttöohjetta on noudatettava tarkasti. (Taustaväri: sininen, etualan väri: valkoinen) Siga as instruções de uso! Cor de fundo: azul, cor de primeiro plano: branco)</p>
	<p>Unique Device Identification</p>
	<p>Medical Device</p>

	<p>Nur zu Verwendung im Innenbereich. For indoor use only. Utiliser uniquement dans des locaux fermés. Sólo utilizar en espacios cerrados. Utilizzare solo in ambienti chiusi. Får endast användas i slutna rum. Uitsluitend in afgesloten ruimten toepassen. Må udelukkende benyttes i lukkede rum. Skal kun brukes i lukkede rom. Käyttö on sallittu ainoastaan suljetuissa tiloissa. Utilizar apenas em espaços fechados.</p>
	<p>Laryngoskop Laryngoscope Laryngoscope Laringoscopio Laringoscopio Laryngoskop Laryngoscoop Laryngoskop Laryngoskop Laryngoskooppi Laringoscópio</p>
	<p>Nicht wiederverwenden Single use only Usage unique Un solo uso Monouso För engångsbruk Eenmalig gebruik Engangsbrug Må ikke gjenbrukes Kertakäyttöinen Uso única</p>
	<p>Achtung Caution Avertissement Advertencia Attenzione Varning Waarschuwing Advarsel Forsiktig Varoitus Advertência</p>
	<p>Seriennummer Serial number Numéro de série Número de serie Numero di serie Seriennummer Seriennummer Seriennummer Seriennummer Sarjanumero Número de série</p>

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