

Cleanliness of your HEINE device's optical parts

Contamination of the outer optical surfaces, e.g. dust and fingerprints, may make cleaning necessary.

Note: Cleaning agents containing solvents can degrade plastic lenses or the lens adhesive.

Procedure:

- First, carefully remove loose particles with compressed air or a special optics brush.
- Moisten a clean cotton swab or a special tissue for cleaning optics (e.g. WHATMAN Lens Cleaning Tissue 105 or Tiffen Lens Cleaning Paper) and ab off excess moisture using a clean absorbent material:
 - For the first cleaning step: Use water.
 - For oily and greasy contaminates: Use a mild solution of detergent and distilled water (e.g. 5 drops in 10ml).
- Always wipe with light pressure out from the center in a spiral motion.
- Repeat the spiral wipe with freshly moistened cotton swabs or special tissues until the desired result is achieved.
- Remove any cleaning liquid residue with a dry cotton swab or special tissue.

