Instruction for use
HEINE F.O.4 NT/F.O.
Laryngoscope Handles

Instruction for use


HEINE F.O.4 NT/F.O.

Laryngoscope Handles


DEUTSCH	4
ENGLISH	8
FRANÇAIS	12
ESPAÑOL	16
ITALIANO	20
SVENSKA	24
NEDERLANDS	28
DANSK	32
NORSK	36
SUOMI	40
PORTUGUÊS	44

HEINE handle types

- | | |
|--------------------------|--------------|
| ❶ Standard F.O. 4 LED NT | F-008.22.894 |
| ❷ Standard F.O. 4 NT | F-007.22.894 |
| ❸ Standard F.O. LED | F-008.22.860 |
| ❹ Standard F.O. | F-001.22.860 |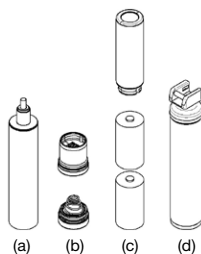

- | | |
|----------------------|--------------|
| ❺ F.O. 4 SLIM LED NT | F-008.22.804 |
| ❻ F.O. 4 SLIM NT | F-007.22.804 |
| ❼ Small F.O. LED | F-008.22.800 |
| ❽ Small F.O. | F-001.22.800 |

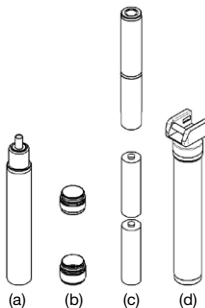- | | |
|-----------------------|--------------|
| ❾ F.O. 4 SHORT LED NT | F-008.22.424 |
| ❿ F.O. 4 SHORT NT | F-007.22.424 |
| ⓫ Short F.O. LED | F-008.22.812 |
| ⓬ Short F.O. | F-001.22.812 |

HEINE F.O. 4 NT/F.O.

Laryngoskopgriffe

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HEINE F.O. 4 NT/F.O.

Laryngoscope Handles

Please read and follow these instructions for use and keep them for future reference.

Intended Use

The internally powered HEINE laryngoscope handles are intended for the transient use with laryngoscope blades according to the "green standard" ISO 7376 for tracheal intubation. It should only be operated by qualified medical personnel within a professional healthcare facility.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Setting up

1 2 Standard F.O. 4

3 4 Standard F.O. battery handle

Connect the Li-ion L rechargeable battery (c) with the bottom insert (b), so that you hear a distinct clicking sound. Insert the batteries (2 x C LR14) (c) into the casing (a) leading with the positive terminal. Screw the bottom insert (b) into casing (a). Finally, screw the casing (a) into the handle shell (d).

If your handle is delivered preassembled, remove the insulation film between the base unit and the rechargeable battery and remove the information band from the handle.

5 6 F.O. 4 SLIM

7 8 Small F.O. battery handle

Insert the rechargeable battery (c) in direction of the arrow into the casing (a) and screw on the bottom insert (b). Finally, screw the casing (a) into the handle shell (d).


9 10 F.O. 4 SHORT

Insert the bulb unit (a) into the handle shell (d). Screw the rechargeable battery (c) into the handle shell.

11 12 Short F.O. battery handle

Insert the batteries (c), regarding the markings on the plastic, into the casing (a). Insert the casing with the bulb first into the handle shell (d) and screw on the bottom insert (b).

Operation

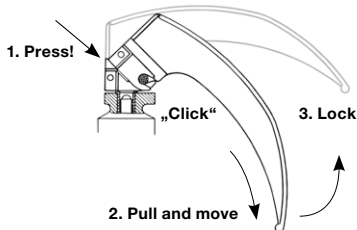⚠ Only use a laryngoscope handle – blade combinations with green markings to ensure proper function.

The HEINE laryngoscope handles correspond to the "green standard" according to the ISO 7376. If non HEINE laryngoscope blades are used, component compatibility and functionality are to be checked before each use!

Only trained personnel are allowed to use a laryngoscope for intubation purposes.

HEINE laryngoscopes are designed for use < 5 minutes.

Before commencing a clinical application, the state of the internal electrical power source must be tested by turning on the lamp.

We strongly recommend a functional backup of laryngoscopes in an emergency situation.

Charging of the battery

If rechargeable batteries are not sufficiently charged before use, the light intensity may be reduced or cut out during operation. We recommend to charge HEINE rechargeable batteries for at least 15 minutes before use in a HEINE charger.

Rechargeable handles should not be switched on during the charging process.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The LED lights cannot be changed.

Check the correct function of the device after each battery or bulb change.

Routinely inspect the condition of the battery and XHL bulb and replace them if necessary.

⚠ The XHL lamp is subject to wear and tear. To ensure safe illumination, we recommend to proactively change the lamp annually.

Before each use, check the condition of the connector for damage and wear and tear. If defects are present, do not use the laryngoscope and replace it with one that is in working order.

Changing the batteries

1 – 8 Standard F.O. and F.O. 4 SLIM / Small F.O.

Unscrew the bottom insert from the handle shell and then out of the casing and replace the (charging) battery.

9 10 F.O. 4 SHORT

Unscrew the charging unit from the handle shell and replace it.

11 12 Short F.O.

Unscrew the handle head from the handle shell and replace the battery.

Changing the bulb

2 4 6 8 10 12

To replace the XHL lamp, unscrew it and screw in a new lamp to the end stop.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

If you don't use the device for a longer period of time, please remove the batteries in advance.

Ensure that moisture does not penetrate the area around the rechargeable battery and the bottom insert.

Check the rechargeable batteries regularly. In case of any abnormalities in the operating or recharging times, replace the rechargeable batteries.

General Warnings

 Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not shine light directly into the eyes to avoid dazzle, especially if the pupils are dilated.

Do not use the laryngoscope for eye reflex testing.

Warning notice for (rechargeable) batteries

 Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight.

Dispose of batteries in the event of visible or presumed damage (e. g. leaking liquid, mechanical damage).

The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water.

For more information on handle types and components please visit www.heine.com.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

 Dispose at your local collection point.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Option
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the laryngoscope handle, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Not applicable
Voltage Fluctuations/Flicker	Not applicable

Technical specification handles

	2 Standard F.O. 4	6 F.O. 4 SLIM	10 F.O. 4 SHORT	1 Standard F.O. 4 LED NT	5 F.O. 4 SLIM LED NT	9 F.O. 4 SHORT LED NT
Nominal load voltage [V]	5					
Input load current [mA]	max. 700	max. 400		max. 700	max. 400	
Nominal voltage light source [V]	3.5				2.5	
Current draw light source [mA]	typ. 770			max. 300		
Class	Internally powered					
Classification according to IEC 62471	LED, exempt group					
Protection class	IP40					

	3 4 Standard F.O.	7 8 Small F.O.	11 12 Short F.O.
Nominal load voltage [V]	N / A		
Input load current [mA]	N / A		
Nominal voltage light source [V]	2.5		
Current draw light source [mA]	LED: max. 300 XHL: typ. 760		
Class	Internally powered		
Classification according to IEC 62471	XHL, exempt group		
Protection class	IP40		

Environmental conditions

Operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa bis 1060 hPa
Storage and transport (without charging-) battery)	-40 °C to +70 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa

Technical specifications rechargeable batteries

	Li-ion L	M2Z 4 NT Li-ion	M3Z 4 NT Li-ion	S2Z 4 NT Li-ion	K3Z 4 NT Li-ion
Cell type	Lithium-ion technology				
Rated voltage [V]	3.7	2.5	3.5	2.5	3.5
Capacity [mAh]	2300	800	800	800	800
Typ. Charging time NT4 [h]	2	2.75	2.75	2.75	2.75
Environmental conditions	30 % to 75 % rel. humidity 700 hPa to 1060 hPa				
Operation	+10 °C to +35 °C				
Storage	+5 °C to +45 °C				
Transport	-20 °C to +45 °C				

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