Instruction for use **HEINE OMEGA® 500 |** **OMEGA 500 UNPLUGGED**

with LED and XHL illumination

Instruction for use

HEINE OMEGA® 500 |

OMEGA 500 UNPLUGGED

with LED and XHL illumination

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
DANSK	24
NORSK	27
SUOMI	30
PORTUGUÊS	33

HEINE OMEGA 500

HEINE OMEGA 500 UNPLUGGED

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HEINE OMEGA 500 UNPLUGGED

with LED and XHL illumination

Please read and follow these instructions for use of and keep them for future reference.

Intended use

The indirect Ophthalmoscope HEINE OMEGA 500 is an AC-powered or battery powered device, containing illumination and viewing optics intended to examine the media (cornea, aqueous, lens and vitreous) and the retina of the eye.

The product may only be used by qualified medical professionals and in professional healthcare facilities.

For U.S. only:

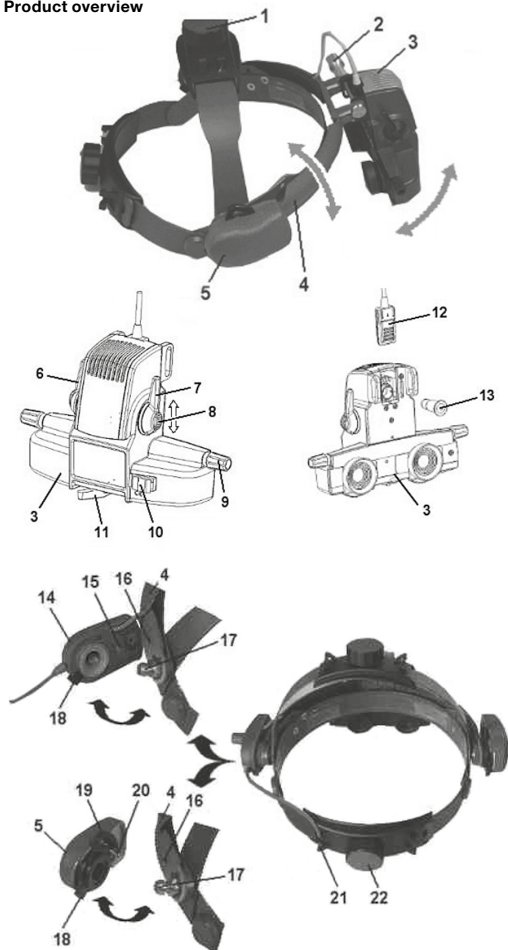
⚠ Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and safety information

⚠ CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

💡 NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview

- 1 Headband height adjustment knob
- 2 Adjustment knob
- 3 Optics unit
- 4 Optics overband
- 5 Position control
- 6 Filter Control
- 7 Aperture Control
- 8 Control Locks
- 9 Rotary knob
- 10 Mount for Teaching Mirror
- 11 Operating lever
- 12 Bulb cover
- 13 Lamp
- 14 Brightness control HC 50 L
- 15 Guide pin
- 16 Hole for guide pin
- 17 Pin
- 18 Release
- 19 Detent plate
- 20 Guide pin
- 21 Cable guide
- 22 Headband size adjustment knob
- 23 Segment cover
- 24 HC 50 L connecting cable with UNPLUGGED socket

Setting up

💡 The HC 50 L brightness controller can be used with XHL lamps and the LED module.

OMEGA 500 can be used with HEINE LED or 5 Watt XHL bulbs. The setup and operation of the HEINE power sources are described in a separate instruction document.

Operation with the HEINE mPack or EN50

To operate the OMEGA 500 with the mPack or EN50, connect the short cable (24) from the HC 50 L brightness control (14) to the connecting cable (Cinch) and secure it in the cable guide (21). Connect the connecting cable (Cinch) to the mPack or the EN50 (Fig. 1).

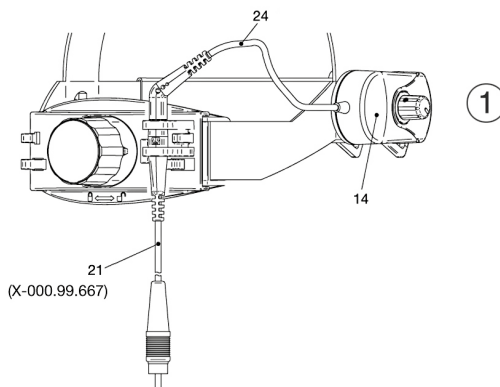
Operation with the mPack UNPLUGGED

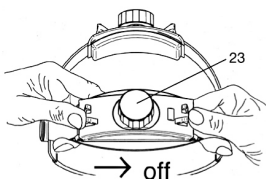
To operate the OMEGA 500 with the mPack UNPLUGGED, first remove the cover from the rear segment (23) on the headband. To do this, slide the segment cover from the headband in the direction of the arrow (Fig. 2).

Next, fit the mPack UNPLUGGED to the headband. To do this, hold the headband with your left hand and using your right thumb, slide the mPack UNPLUGGED to the left until it latches audibly. Connect the short cable (24) from the HC 50 L brightness control (14) to the mPack UNPLUGGED (Fig. 3).



Operation with Power supply for HC 50 L brightness controller

To operate the OMEGA 500 with power supply, please connect the short cable (24) of the brightness controller HC 50 L (14) with the connection cable (UNPLUGGED connector) secure it in the cable guide (21). Connect the connecting cable (Cinch plug) with the power supply (Fig. 1).


②

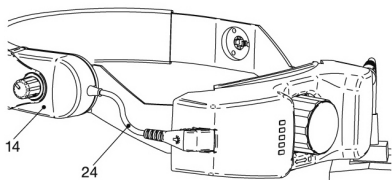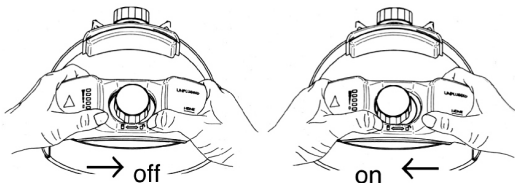③

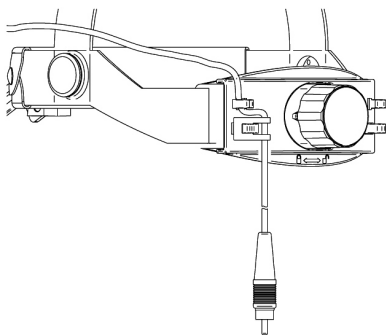④

Operation OMEGA 500 Headband

Selecting the position of the optics overband

The overband (4) swivels and can be fixed into the following positions by means of the position lock (5) (upwards for the rest position, down for working positions 1 or 2). To release the overband, press the position lock control (5) and rotate it. Do not try to force the overband up or down. With practice a comfortable working position can be selected and, when properly adjusted, the optics unit will always return to the desired working position.

Fitting the headband rheostat HC 50 L

When fitting, ensure that the guide pin (15) of the headband rheostat HC 50 L (14) locates correctly in the opening (16) of the overband (4). Locate the connecting cord between the power source and the headband rheostat in the cord clip (21) at the back of the headband.

Fitting the position control

Before fitting the position control (5) ensure that the positioning plate (19) is swiveled upwards as far as possible. To do this, press the positioning plate (19) and swivel it upwards simultaneously. Ensure that the optics overband is in the lower position and that the guide pin (20) locates in the opening (16). Now press the position control (5) on to the guide (17) until it clicks into place.

Removing the headband rheostat HC 50 L and position control

The control of the HC 50 L (14) and the position control (5) can be fitted on the right- or left-hand side of the headband according to preference. To remove, press the release (18) and pull the control off the headband.

Adjusting the optics

Unscrew the control (2) so that the optics unit (3) is free to move. (The position of this control can be reversed for left-handed operation if required). Place the instrument on your head and adjust the height and circumference by means of the adjusters (22) and (1) until a comfortable fit is achieved. The rear part of the headband can be pivoted and is self-latching. Adjust the optics to a position as close as possible to your eyes and view the light spot at a distance of about 30–40 cm without accommodating. A small, pencil-sized object should be sharply-focused (the eyepieces incorporate 2D lenses. Spectacle wearers should examine through the "far" part of the spectacle lens). Adjust the optics unit so that the light spot appears centered vertically in position of the headband or to choose an alternative working position by pressing your view and screw the adjuster knob in (2). (It may be necessary to correct the lever (5) and swiveling the optics overband into a different position. If the light spot needs horizontal adjustment, simply rotate the headband slightly to the side. Adjust and fine-tune the PD setting by closing first one then the other eye and observing an object in the middle of the light spot while adjusting the appropriate eyepiece. Repeat until the object is in the middle of the field of view, and a single image is obtained. Take off the instrument and check that the PD is symmetrically adjusted. If not, repeat the selection procedure as above. Correct adjustment of PD is particularly important when examining through a small pupil. Each user should adjust the setting to match his-own PD. Every instrument includes two plano lenses, which can be exchanged for the 2D lenses which are fitted as standard equipment. Always use the OMEGA 500 with plano or the 2D lenses. Make sure that you change the plano lenses under clean conditions to avoid that particles enter the optics.

Operation OMEGA 500

Basic Adjustments

Switch the instrument on by means of the rotary knob (brightness control) on the HC 50 L brightness controller. Aperture wheel (7) and filter wheel (6) should be rotated upwards to select the large aperture with white light. Control (11) should be in the middle position. The beam control (9) should be in the mid-position. Slide the eyepieces into position to match your P.D. or select the mid-position on the scale.

Setting the brightness

Adjust the brightness using the HC 50 L (14) brightness controller or the connected power source.

If you use OMEGA 500 with LED, please adjust a slightly higher brightness initially and reduce the brightness afterwards.

Control Locks

The aperture (7) and filter controls (6) can be locked into position by means of the lock (8) to avoid unintentional operation.

Aperture Control

3 different aperture sizes and a diffuser aperture can be selected by means of the aperture control (7). The choice of aperture depends mainly on the size of the patient's pupil. The diffuser is useful for photophobic patients and the examination of the periphery.

Filter Control

The filter control (6) can be used to select an interference red-free filter, a blue or a yellow filter which can be introduced into the illumination beam.

Synchronized adjustment of Convergence and Parallax

This feature ensures the best possible stereopsis and image quality for any pupil size by simultaneous adjustment of the observation and illumination beams. For a dilated pupil, move the control (11) underneath the instrument to the large circle position. For a small pupil, move the control to the small circle position. The control (11) can be positioned anywhere between these two settings to select the optimum parallax and stereopsis (convergence) for any pupil size. The illumination beam can be adjusted vertically by the control (9).

Teaching Mirror

The optional teaching mirror can be slid onto the fixture (10) of the optics.

Hygienic reprocessing

- The instruction is available:
- online at www.heine.com
 - in a paper version which you can request from the address listed

Maintenance

Changing the light source (XHL and LED module)

 First allow the instrument to cool down.

Disconnect the instrument from the power source. Unscrew the control (2) so that the optics unit (3) is free to move. Slide the bulb cover (12) upwards. Do not pull on the cord. Swivel the optics unit (3) to enable the bulb (13) to be pulled out of the bulbholder. Clean the glass dome of the new bulb carefully with a soft cloth. Insert the new bulb so that the locating pin on the bulb body fits into the guide in the bulbholder. Re-fit the bulb cover (12) into the optics unit (3) and slide the bulb cover back down into position.

Service

The device has no components serviceable by the end-user.

General notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.
The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:
All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.
The indirect ophthalmoscope OMEGA 500 in its original packaging meets the requirements for transport according to ISO 15004-1. OMEGA 500 meets the requirements according to EN ISO 10943.

General warnings

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.
- Do not use the device in fire or explosive risk area (e.g. oxygen saturated or anaesthetic environments).
- This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.
- Do not modify the device.
- Use only original HEINE parts, spare parts, accessories and power sources.
- Repairs shall only be carried out by qualified persons.
- Store and use the device in dry and dust-free environments only!
- Use the OMEGA 500 only in combination with the HEINE ophthalmoscopy lenses.
- Use as little light as possible for the examination and ensure that the light is switched off after each examination. Please always consider your medical duty of care.
- Do not look into bright light sources by means of loupes. Hazard of glare.
- Avoid the ophthalmoscope lens to be exposed to direct sunlight. Hazard of fire.
- Only use the OMEGA 500 when the lenses are clean.
- Let OMEGA 500 accommodate to ambient conditions when the instrument is cold.
- Change the ocular lens only in a clean environment in order to avoid dust to enter the OMEGA 500.

Light exposure hazard

Because prolonged intense light exposure can damage the retina, the use of the device for ocular examination should not be unnecessarily prolonged, and the brightness setting should not exceed what is needed to provide clear visualization of the target structures. The retinal exposure dose for a photochemical hazard is a product of the radiance and the exposure time. If the value of radiance were reduced in half, twice the time would be needed to reach the maximum exposure limit.

While no acute optical radiation hazards have been identified for direct or indirect ophthalmoscopes, it is recommended that the intensity of light directed into the patient's eye be limited to the minimum level which is necessary for diagnosis. Infants, aphakes and persons with diseased eyes will be at greater risk. The risk may also be increased if the person being examined has had any exposure with the same instrument or any other ophthalmic instrument using a visible light source during the previous 24 hours. This will apply particularly if the eye has been exposed to retinal photography.

Caution – The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater the risk of ocular damage. An exposure time with this instrument at maximum intensity of longer than (see table below) will lead to the guideline hazard value being exceeded.

Exposure guidelines

Instrument type	Working distance* [mm]	Duration [min]
OMEGA 500 XHL	400	4
OMGEA 500 LED	400	35

*) from instrument to A.R. 16D ophthalmoscopy lens

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

- The appendix contains following tables
- Electromagnetic disturbances – Requirements and tests
 - Technical specification
 - Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the OMEGA 500 including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	See attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed
Voltage Fluctuations Flicker	Passed

*n|a: „Not applicable“ in the internally powered mode

Technical specification	
Power supply	6 V
Power consumption	5 W XHL, 1 W LED
Class	II
Classification according to ISO 15004-2:2007	Group 2
Classification according to ANSI Z.80.36-2016	Group 2
Weight	470 g (Instrument)

Environmental conditions	
Environmental conditions for operation	+5 °C to +35 °C 30 % to 90 % rel. humidity 800 hPa to 1060 hPa
Environmental conditions for storage	-20 °C to +55 °C 10 % to 95 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for transport	-40 °C to +70 °C 10 % to 95 % rel. humidity 500 hPa to 1060 hPa

Options	
HC 50 L Headband Rheostat (without plug-in transformer)	X-095.16.330
Extension Cord Cinch-Cinch, 2 m	C-000.33.510
Connecting Cord Cinch to EN 50 charger mPack plug-in transformer for HC 50 L, 1.6 m	X-000.99.667
Extension cord from plug-in transformer UNPLUGGED to mPack UNPLUGGED, 2 m	X-000.99.668
HC 50 L Headband Rheostat (without plug-in transformer)	X-095.16.325
Teaching mirror	C-000.33.209
Scleral depressor, small	C-000.17.301
Scleral depressor, large	C-000.17.300
Ophthalmoscopy lens with case A.R.16 D Ø 54 mm	C-000.17.225
Ophthalmoscopy lens with case A.R.20 D Ø 50 mm	C-000.17.228
Ophthalmoscopy lens with case A.R.30 D Ø 46 mm	C-000.17.231
Fundus charts Pad with 50 pcs.	C-000.33.208

Light sources	
XHL Xenon Halogen spare bulb 6 V, 5 W	X-004.88.111
Upgrade-Kit OMEGA 500 with LED module and HC 50 L	X-008.16.325
LED module	X-008.87.200

Erläuterung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati
Förklaring av symboler som används
Verklaring van de gebruikte symbolen
Forklaring af de anvendte symboler
Symbolforklaring
Käyttetytjen symbolien selitys
Explicação dos símbolos utilizados

REF

SN

CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizinprodukteverordnung (EU) 2017/745. The CE mark indicates that the product complies with the medical device regulation (EU) 2017/745. Le marquage CE indique que le produit est conforme ou au règlement (UE) 2017/745 relatif aux dispositifs médicaux. La marca CE indica que el producto cumple el Reglamento europeo sobre productos sanitarios (UE) 2017/745. Il marchio CE indica la conformità con il regolamento sui dispositivi medici (UE) 2017/745. CE-märkning markerar en överensstämmelse med förordningen om medicinska produkter (EU) 2017/745. CE-markering duidt de overeenstemming aan met de verordening betreffende medische hulpmiddelen (EU) 2017/745. CE-mærkningen angiver overensstemmelse med forordningen om medicinsk udstyr (EU) 2017/745. CE-merket angir at produktet er i samsvar med forskriften om medisinsk utstyr (EU) 2017/745. CE-merkintä tarkoittaa, että laite lääkinnällisiä laitteita koskevan asetuksen (EU) 2017/745 kanssa. O símbolo CE identifica a concordância com o regulamento de Dispositivos Médicos (EU) 2017/74.

Katalog- oder Bestellnummer
Catalogue- or order number
Numéro de catalogue ou de commande
Número de catálogo o de pedido
Codice catalogo e di dell'ordine numero
Katalog- eller Beställningsnummer
Catalogus- of Bestelnummer
Katalog- eller Ordrenummer
Katalog- eller bestillingsnummer
Luettelo- tai viitenumero
Número de catálogo ou pedido

Serienummer
Serial number
Numéro de série
Número de serie
Numero di serie
Seriennummer
Seriennummer
Seriennummer
Seriennummer
Sarjanumero
Número de série

Hersteller
Manufacturer
Fabricant
Fabricante
Produttore
Tillverkare
Fabrikant
Producent
Produsent
Valmistaja
Fabricante

Herstelldatum
Date of manufacture
Date de fabrication
Fecha de fabricación
Data di produzione
Tillverkningsdatum
Productiedatum
Produktionsdato
Produksjonsdato
Valmistuspäivä
Data de fabricação

Getrennte Sammlung von Elektro- und Elektronikgeräten (Europäische WEEE Richtlinie)
Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE)
Tri sélectif des appareils électriques et électroniques (Directive européenne DEEE)
Desechado separado de aparatos eléctricos y electrónicos Directiva Europea RAEE)
Raccolta differenziata di apparecchi elettrici ed elettronici (direttiva europea RAEE)
Separat insamling av elektriska och elektroniska apparater (det europeiska WEEE-direktivet)
Gescheiden inzameling van elektrische en elektronische apparaten (Europese AEEA richtlijn)
Separat indsamling af elektrisk og elektronisk udstyr (det europæiske WEEE-direktiv)
Produkter med dette symbolet skal ikke kasseres sammen med vanlig husholdningsavfall, men krever separat kassering i henhold til lokale bestemmelser. (European Waste Electrical and Electronic Equipment Directive, WEEE)
Sähkö- ja elektroniikkalaitteille tarkoitettu erillinen keräyspiste (eurooppalainen WEEE-standardi)
Coleção separada de aparelhos elétricos e eletrônicos (Diretrizes Europeias WEEE)

Zulässiger Temperaturbereich in °C für Lagerung und Transport
Temperature limits in °C for storage and transport
Plage de température admise en °C pour le stockage et le transport
Rango de temperatura permitida en °C para almacenar y transportar el producto
Temperatura ammessa in °C per conservazione e trasporto
Tillåtet temperaturintervall i °C för lagring och transport
Toegeestane temperaturen in °C voor opslag en transport
Tilladt temperaturområde i °C ved opbevaring og transport
Temperaturbegrensning i °C for oppbevaring og transport
Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°C)
Limite de Temperatura em °C para armazenamento e transporte

Zulässiger Temperaturbereich in °F für Lagerung und Transport
Temperature limits in °F for storage and transport
Plage de température admise en °F pour le stockage et le transport
Rango de temperatura permitida en °F para almacenar y transportar el producto
Temperatura ammessa in °F per conservazione e trasporto
Tillåtet temperaturintervall i °F för lagring och transport
Toegeestane temperaturen in °F voor opslag en transport
Tilladt temperaturområde i °F ved opbevaring og transport
Temperaturbegrensning i °F for oppbevaring og transport
Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°F)
Limite de Temperatura permitida em °F para armazenamento e transporte

Zulässige Luftfeuchtigkeit für Lagerung und Transport
Humidity limitation for storage and transport
Humidité admise pour le stockage et le transport
Humedad del aire permitida para almacenar y transportar el producto
Umidità atmosferica ammessa durante il trasporto e la conservazione
Tillåten luftfuktighet för transport och lagring
Toegeestane luchtvochtigheid voor opslag en transport
Tilladt luftfugtighed ved opbevaring og transport
Fuktighetsbegrensning for oppbevaring og transport
Sallittu ilmakesteus kuljetuksen ja varastoinnin aikana
Umidade do ar admissível para o armazenamento e transporte

Zulässiger Luftdruck für Lagerung und Transport
Pressure limitation for storage and transport
Pression atmosphérique admise pendant le transport et le stockage
Presión de aire permitida para almacenar y transportar el producto
Pressione atmosferica ammessa durante il trasporto e la conservazione
Tillåten lufttryck för lagring och transport
Toegeestane luchtdruk voor opslag en transport
Tilladt lufttryk ved opbevaring og transport
Trykkbegrensning for oppbevaring og transport
Sallittu ilmanpaine kuljetuksen ja varastoinnin aikana
Pressão do ar admissível para o armazenamento e transporte

	<p>Vorsicht Bruchgefahr! Fragile, handle with care! Fragile ! Manipuler avec soin Atención. Frágil. Attenzione: pericolo di rottura! Försiktigt! Risk för brott Voorzichtig, kans op breuk! Forsigtig, risiko for brud! Ømtålig, behandles forsigtigt! Varo särkymisvaaraa! Frágil, manuseie com cuidado!</p>		<p>Netzteilanschluss (9VDC) Power supply connector (9VDC) Connecteur-adaptateur – Alimentation en tension (9VDC) Conector – Alimentación de tensión (9VDC) Spina di collegamento – tensione di alimentazione (9VDC) Anslutningskontakt – spänningsförsörjning (9VDC) Stekker-aansluiting (9 V DC) Netdeltiöslutning (9VDC) Strömformörsyningskontakt (9 V DC) Liitäntäpistoke – jännitteen syöttö (9 V DC) Conector de energia (9VDC)</p>
	<p>Trocken lagern! Keep dry! Conserver au sec ! Conserver en un lugar seco! Evitare ambienti umidi! Förvaras torrt! Droog bewaren! Opbevares tørt! Hold tørt! Säilytetään kuivassa paikassa! Armazenar em ambiente seco!</p>	<div>UDI</div> <div>MD</div>	<p>Unique Device Identification</p> <p>Medical Device</p>
	<p>Gebruiksaanweisung verbindlich befolgen. (Hintergrundfarbe: blau, Vordergrundfarbe: weiß) Follow instructions for use! (Background color: blue, foreground color: white.) Suivre le mode d'emploi. (Couleur de fond : bleu ; couleur du premier plan : blanc) Seguir obligatoriamente las instrucciones de uso. (Color de fondo: azul, color de primer plano: blanco) Attenersi obbligatoriamente alle istruzioni per l'uso. (Colore dello sfondo: blu, colore in primo piano: bianco) Bruksanvisningen ska alltid följas. (Baggrunds-färg: blå, förgrunds-färg: vit) De gebruiksaanwijzing is bindend en dient gevolgd te worden. (achtergrondkleur: blauw, voorgrondkleur: wit) Følg altid brugsanvisningen. (Baggrunds-farve: Blå; forgrunds-farve: Hvid) Følg bruksanvisningen! (Baggrunns-farge: blå, forgrunns-farge: hvit.) Käyttöohjetta on noudatettava tarkasti. (Täustaväri: sininen, etualan väri: valkoinen) Siga as instruções de uso! (Cor de fundo: azul, cor de primeiro plano: branco)</p>		
	<p>Anwendungsteil Typ BF Applied part Typ BF Partie appliquée de type BF Pieza de aplicación del tipo BF Applicazione di tipo BF Användningsdel för typ BF Gebruiksonderdeel van het type BF Anvendelsesdel type BF Anvendt del type BF Tyyppin BF liityntäosa Parte de aplicação do tipo BF</p>		
	<p>Instrumenten Anschluss Instrument connector Connecteur-adaptateur – Instrument Conector – Instrumento Spina di collegamento strumento Anslutningskontakt – instrument Stekkerverbinding – instrument Instrumentiöslutning Instrumentkontakt Liitäntäpistoke – instrumentti Conector de instrumento</p>		