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HEINE BETA 200 LED retinoscope

HEINE BETA 200 retinoscope

These instructions apply to the following products of the HEINE Retinoscope series: BETA 200 LED, BETA 200.

Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE Retinoscope is a hand-held, non-invasive, internally powered medical device. It is intended for the transient measuring of the refraction (objective refraction) of the eye.

The product may only be used by qualified medical professionals and in professional healthcare facilities.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview

- 1 Connector (only XHL)
- 2 Bulb cover (only XHL)
- 3 Control ring
- 4 ParaStop
- 5 Brow rest
- 6 Brow rest
- 7 Fixation card
- 8 Orange filter (only XHL)

Setting up

To set up the HEINE Retinoscope, plug the instrument head into the HEINE supply handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE 2.5 V bulb
only for use with the HEINE BETA battery handle.

Red ring = HEINE XHL 3.5 V bulb
Black ring = HEINE LED 3.5 V bulb

only for use with the HEINE BETA rechargeable handle and HEINE wall unit.

Operation

Two different-length brow rests (5) and (6) are included with the instrument.

They can be removed and interchanged according to user preference. The control ring (3) is used to rotate the streak image through 360°. The lowest position gives a divergent beam, the highest position focuses the streak at a distance of approx. 25 cm in front of the retinoscope.

The ParaStop (4) limits the upward travel of the control ring to give a parallel beam of light at the top of its travel. To operate, move the button down. To disengage, push either the button or the control ring (3) upwards.

To convert the XHL Streak Retinoscope to a Spot Retinoscope simply change the streak bulb for a spot bulb (see options).

HEINE Retinoscopes are intended for a transient examination of less than < 5 minutes with a 15 minutes break until the next application.

ISO 15004-2: Group 2 (LED)

CAUTION – The light emitted from this instrument is potentially harmful. The risk of damaging your eyes increases with the exposure time. Exposure time with this instrument at maximum intensity for more than 5 minutes will result in an exceeding of the guideline value for hazard.

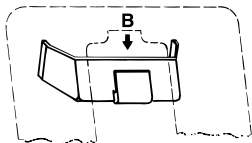
ANSI Z80.36-2016: Group 2 (LED)

CAUTION – The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater is the risk of ocular damage. Exposure to light from this instrument when operated at maximum intensity will exceed the recommended maximum exposure (RME) of 2.2 J/cm², unless additional action is taken by the user to minimize exposure, after 53 sec. The risk of retinal injury at an exposure of 2.2 J/cm² is not high, but because some patients may be more susceptible than others, caution is advised if this radiant exposure value is exceeded. However, because of a significant risk of injury at exposures exceeding 10 J/cm², the user should avoid exposures longer than 4 min. If a longer exposure is needed anyway, the light intensity has to be reduced in an appropriate manner.

If, after switching on the unit, a sharp drop in brightness is noticed, new batteries should be inserted or rechargeable batteries recharged.

The setup and operation of the HEINE handles are described in a separate instruction document.

Fixation Cards

The HEINE Fixation Cards for dynamic retinoscopy are attached to the retinoscope by means of a special mount.

- Slide the mount from the patient's side into the lateral grooves (A).
- Select a fixation card and hook it on from above (B).

When not using fixation cards the mount can be left on the retinoscope or can be removed by pulling it off.

Hygienic reprocessing

The instruction is available:

- online at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

Changing the light source

- Verify that the lamp voltage complies with the supply voltage of the handle.
- Allow the device to cool down before changing the bulb.

BETA 200 LED

With these devices, the LED cannot be changed.

BETA 200

- Unscrew the connector (1) and shake lightly to remove the bulb.
- Pull off the bulb cover (2).
- Wipe the glass body of the new replacement bulb with a clean cloth and replace the bulb cover.
- Insert the bulb into the retinoscope and replace the connector

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE.

For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Store and use the device in dry and dust-free environments only.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments)

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Options
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
	The supply voltage quality should be that of a typical hospital environment.
	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in im-proper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the considered device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.

Test	Compliance
RF emissions CISPR11	Group 1* Class B
Conducted emissions (EN 55011/CISPR 11)*	Passed
Radiated emissions (EN 55011/CISPR 11)	
Harmonic current emissions (IEC 61000-3-2)*	
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)*	
Immunity	See attached immunity test levels

Immunity test levels

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ±8 kV Air Discharge: ±2 kV, ±4 kV, ±8 kV, ±15 kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80-2700 MHz 80 % AM at 1kHz	
Electrical fast transients/ bursts (IEC 61000-4-4)*	± 2 kV 100 kHz repetition frequency	Retinoscope with a 2,5 V bulb: not applicable, because of the internally powered mode. Retinoscope with a 3,5 V bulb: Compliance test levels are identically to the IEC 60601-1-2 test levels
Surges (IEC 61000-4-5)*	± 0,5 kV, ± 1 kV	Retinoscope with a 2,5 V bulb: not applicable, because of the internally powered mode. Retinoscope with a 3,5 V bulb: Compliance test levels are identically to the IEC 60601-1-2 test level
Conducted disturbances induced by RF fields (IEC 61000-4-6)*	3 V 0,15 MHz – 80 MHz 6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	Retinoscope with a 2,5 V bulb: not applicable, because of the internally powered mode. Retinoscope with a 3,5 V bulb: Compliance test levels are identically to the IEC 60601-1-2 test levels
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: ±5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50 Hz or 60 Hz	
Voltage dips (IEC 61000-4-11)*	0 % UT; 0,5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0°	Retinoscope with a 2,5 V bulb: not applicable, because of the internally powered mode. Retinoscope with a 3,5 V bulb: Compliance test levels are identically to the IEC 60601-1-2 test levels
Short interruptions (IEC 61000-4-11)*	0 % UT; 250/300 cycle	

*n/a: "Not applicable" in the internally powered mode

Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 90 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	-10 °C to +55 °C 10 % to 95 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for transport	-40 °C to +70 °C 10 % to 95 % rel. humidity 500 hPa to 1060 hPa
Nominal voltage	3.0 V – 3.7 V
Nominal current	Typ. 390 mA
Class	Internally powered
Device classification according ISO 15004-2	Group 2 (LED) Group 1 (XHL)
Device classification according to IEC 62471	Exempt Group (LED)
Protection class	IP40

Options

Fixation cards	C-000.15.360
Orange filter (only for the XHL bulb)	C-000.15.359

XHL xenon spare bulb

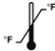
Types	2.5 V	3.5 V
BETA200 Streak-Retinoscope	X-001.88.087	X-002.88.089
BETA200 Spot-Retinoscope	X-001.88.088	X-002.88.090

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