∴ HEINE mPack mini

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 113158 2021-06-25

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DEUTSCH	3
ENGLISH	6
FRANÇAIS	8
ESPAÑOL	11
ITALIANO	14
SVENSKA	17
NEDERLANDS	19
DANSK	22
NORSK	25
SUOMI	27
PORTUGUÊS	30

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Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE mPack mini rechargeable battery is intended for use with HEINE medical devices exclusively. The product may only be used by qualified medical professionals and in professional healthcare facilities.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview

- 1 Micro-USB connector
- 2 Socket
- 3 Controller
- 4 Status indicator

Setting up

Charge the HEINE mPack mini fully before its first use.

Charging

Use a USB cable to connect the USB connector (1) of the mPack mini to a USB power supply (e.g. HEINE E4-USB, PC or Laptop). At the start of the charging process, all three illuminating segments of the status indicator (4) light up briefly. During the charging process, the segments light up progressively to indicate the charging status. As soon as all segments are on permanently, the mPack mini is fully charged. The indicator is switched off after a short period of time.

Installation

Plug the supply cable of the HEINE instrument you wish to operate with the mPack mini into socket (2).

Operation

Rotate the controller (3) to change the brightness of the connected instrument. To switch the mPack mini off, rotate the controller counterclockwise until you hear a „click“.

When the mPack mini is switched on and an instrument is connected to it, the status indicator indicates the charging status as follows:

- All segments are on permanently: fully charged
- Lower two segments are on permanently: sufficiently charged
- Lower segment (orange) is on permanently: almost discharged
- Lower segment (orange) flashes: strongly discharged, please recharge!

If no instrument is connected to the mPack mini when the mPack mini is switched on, the status indicator lights up for approximately 1 second and is then switched off.

 The connected instrument cannot be operated during the charging process! The illumination of the instrument is switched off as soon as the mPack mini is connected to a USB power source. Once the charging connection is disconnected, the instrument illumination is switched on again at the previously set brightness level. If the connected instrument is defective, the mPack mini shuts down automatically and issues an error-indicating sequence of flashes.

Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the Class II and safety requirements from the IEC 60950-1 Information technology or IEC 60601-1 Medical electrical equipment.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Battery replacement

A change of the charging cell is only necessary if it does not take the required charge anymore. With proper care this is usually after around 2 to 3 years. If a replacement is required, please contact your authorized dealer.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority. Do not kink the cable and do not wrap it around the power supply since this reduces the service life of the cable.

General Warnings

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.
- Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)
- Do not modify the device.
- Use only original HEINE parts, spare parts, accessories and power sources.
- The device must not be placed near strong Magnetic fields, e.g. MRI units.
- If the power source is transferred from a cold to a warm environment, let it rest for approximately 1 h to allow it to equilibrate to the ambient conditions.
- The device should never be used or stored outdoors, since this may lead to malfunction.

Disposal

-  The product must be recycled as separated electrical and electronic devices.
-  Please observe the relevant state-specific disposal regulations.
-  Dispose at your local collection point.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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Technical specification

Environmental conditions for operation	+10 °C to +35 °C 10 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa
Recommended conditions for storage for more than 3 months	+5 °C to +20 °C up to 70 % rel. humidity
Input	5 V, 500 mA (Micro-B USB)
Output	max. 450 mA, max. 2 W
Nominal voltage	0 V to 5.5 V
Charging time	typ. 4 h, max. 10 h
Weight	95 g
Dimensions	44 mm x 23 mm x 101 mm
Class	Charging: II operation: internally powered
Housing protection	IP40

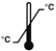
THIS INSTRUMENT DOES NOT WITHSTAND THE TEMPERATURE CONDITIONS FOR STORAGE ACCORDING TO ISO 15004-1. DO NOT STORE THIS INSTRUMENT AT CONDITIONS UNDER WHICH THE TEMPERATURE MAY BE ABOVE 45 °C OR BELOW 5 °C.

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