

HEINE Universal Smartphone Adapter mag for DELTA 30 PRO, DELTA 30 and DELTAone.

Digital dermatoscopic documentation using a smartphone camera.

IT'S JUST THAT EASY!




You can use almost any smartphone with the Universal Smartphone Adapter mag, regardless of the camera positioning.

⚠ The delivery contains two fixing buttons of different depths. Please check which one is suitable for your smartphone's camera height.



Next, attach the adapter to the fixing button.

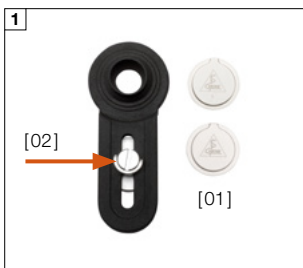

⚠ Keep the fixing button [01] separate from the magnet before attaching it to the phone case!

If both are still stuck together, separate the screw [02] from the adapter arm. Turn the screw directly into the thread of the magnet holder and push the two elements apart.



Loosen the screw slightly.

If your smartphone has just one camera, fix the adapter in the centre in front of this camera.

If your smartphone has multiple cameras, first determine which of the cameras is the main camera: place the adapter arm on the back of the smartphone so that one of the camera lenses is in the centre. Is the field of view on the screen clear in the camera view? If everything fits, use the screw to tighten the adapter arm in front of this camera.



You need a DELTA 30 PRO, DELTA 30 or DELTAone and the Universal Smartphone Adapter mag, consisting of fixing button + adapter, as well as your smartphone and a suitable phone case*.

The phone case should have a hard polycarbonate or acrylic glass back.



Remove the protective film and press the fixing button firmly onto the centre of the phone case on the right below the selected camera lens.



The magnet of the adapter now keeps the mobile phone and case in position on the focusing ring of the dermatoscope.

You can now use the camera function on your smartphone as normal to take photos.



Please find the video tutorial here: www.heine.com/how-to-smartphone-adapter-mag

* The smartphone and phone case are not included in the scope of delivery.

