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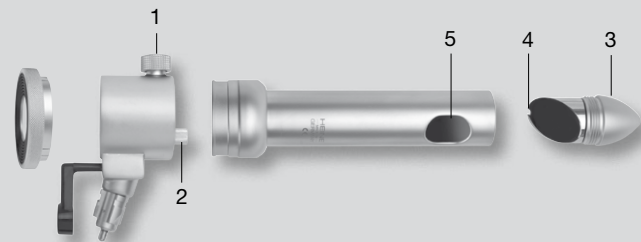
Instructions for use

ENGLISH

HEINE Anoscope / Proctoscope with proximal illumination

Read these instructions carefully before using the HEINE Anoscope/Proctoscope with proximal illumination and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Range of applications: HEINE Anoscopes and proctoscopes with proximal illumination are designed only for examination and treatment of the anal passage.

Safety in use: Use only light- or power- sources which are approved for use in a medical environment. Tubes with straight direction of view should only be inserted when an obturator is fitted.

Using the instrument

- Connect the instrument head to a handle.
- Insert the tube into the head as far as the stop.
- Rotate the tube until the viewing window is in the desired position. Secure the tube with the screw (1).
- Insert the obturator through the open instrument head.
- The cone of the obturator and the tube should be lubricated with a suitable lubricant.

The viewing window and swivel lens magnify by approx 2 x. With a short tube they can be used together to give ca. 3,5 x magnification.

Adjusting the mirror assembly

With tubes with a lateral viewing window, insert the mirror assembly (3) into the tube so that the groove (4) coincides with the pin (5). Screw in the mirror assembly as far as possible.

Changing the bulb

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

Unscrew the bulb from the head (2). Screw in a new bulb.

Use with a cold light source

The F.O. (fiber optics) head can be illuminated by any HEINE cold light source.

Cleaning / Disinfection and Sterilisation

Preparation at the point of use/ Initial Cleaning

Residues and other deposits must be removed immediately after use to avoid any residues drying on to the instrument.

Disassembly

The instrument must be removed from the light source for further preparation steps. The article must be disassembled to ensure a satisfactory cleaning result (see picture).

Tubes and Obturator

Manual disinfection and cleaning

For manual cleaning and disinfection, only those cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastics may be used. The manufacturer's recommendations for use must be followed to avoid material damage / corrosion. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid surface residues.

Machine cleaning and disinfection / Thermal Disinfection

The article can be machine-cleaned and disinfected at up to $\leq 93^{\circ}\text{C}$. Only cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastic may be used. The instructions of the manufacturer of both machines and detergents must be followed. The machine should be loaded correctly so that the cleaning procedure is successful. Rinsing procedures must ensure that no deposits are left. The final rinse should be carried out using de-mineralised water.

Ultrasonic Cleaning is not allowed for articles with fiber optics.

Sterilization

We recommend sterilization with saturated steam at 134°C for 5 minutes. Cycles of 18 minutes are allowed.

Instrument head

Instrument head can be treated by surface disinfection.

Warning: Non-suitable or incorrectly-used chemicals or preparation procedures can lead to irreparable damages to the article.

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