HEINE VET Oscopes

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 202
E-Mail: info@heine.com · www.heine.com
MED 112824 2015-12-15

HEINE VET Oskopse

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HEINE VET Oscopes

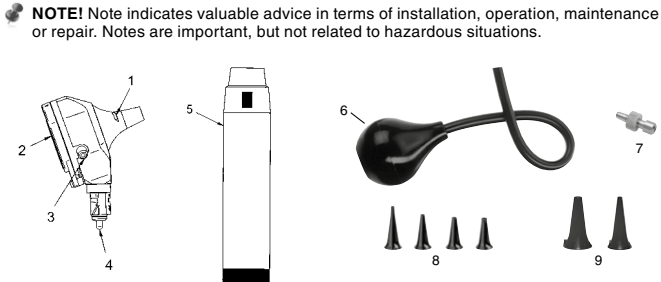
These instructions apply to the following products of the HEINE Oscope series: BETA200 LED VET, BETA200 VET, BETA100 VET

🔦 Please read and follow these instructions for use of and keep them for future reference.

Intended Use
HEINE otoscopes are intended for the examination of the ear, the external auditory canal and the tympanum. The instruments have an optical magnifying equipment and a battery- or rechargeable battery-powered illumination equipment. The instruments may only be used by qualified medical personnel.

⚠️ For U.S. only:
Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information
⚠️ CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).
🔦 NOTE! Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

- Product overview**
- | | |
|---------------------|------------------------------------|
| 1 Slit | 6 Insufflation bulb |
| 2 Viewing window | 7 Insufflation port connector |
| 3 Insufflation port | 8 HEINE SANALON® S tips (re-use) |
| 4 Connector plug | 9 HEINE AllSpec® tips (single use) |
| 5 HEINE Handle | |

Setting up
To set up the instrument, fit the instrument head onto the HEINE battery handle or the HEINE rechargeable handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage.

White ring = HEINE XHL® 2.5 V bulb only for use with the HEINE BETA Battery handle
Red ring = HEINE XHL® 3.5 V bulb
Black ring = HEINE LED illumination only for use with the HEINE BETA Rechargeable Handle

Operation
The HEINE otoscope may only be inserted into the auditory canal when a tip is fitted.

BETA200 LED VET / BETA200 VET
Place the HEINE tip on the otoscope so that the projection inside the tip fits into the bayonet slit (1). Tighten the tip by twisting it slightly to the right.

BETA100 VET
Specula with bayonet connector
Fit the speculum to the head so that the locating pin fits the groove provided. Then twist the profiled ring as far as the stop.
Disposable AllSpec specula can be used if the adaptor B-000.11.306 is fitted.

SANALON S specula
The adapter G-000.21.214 must be fitted if these tips are used. Fit the speculum to the adaptor so that the locating pin fits in the groove provided. Twist the profiled ring as far as the stop.

The magnifying lens is inside the viewing window (2). To facilitate instrumentation, the viewing window can be swivelled to both sides or folded up.
The otoscopes have a connection port (3) for the optionally available insufflation bulb (6, 7). The test of tympanic mobility can be carried out when the viewing window is closed using the insufflation bulb. Apply the pressure carefully with the insufflation bulb.

🔦 Application duration
The otoscopes are intended for short term treatment < 1 min with a break between applications of 10 minutes.
The maximum temperature at the application point at an ambient temperature of 35 °C and in continuous operation is 44.5 °C.

The setting up and operation of the HEINE handles are described in a separate instruction of use.

Hygienic Reprocessing
Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: noncritical
Spaulding Classification USA: noncritical
⚠️ In the event of suspected contamination, carry out hygienic preparation of the instrument.
The reusable tips have to be reprocessed after each use.
The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.
HEINE Optotechnik only approves the resources and procedures named in these instructions for use.
Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.
Observe the instructions of the manufacturer of the reprocessing media.
Do not use spray or immersion disinfection, dripping wet or heavily foaming tissues.
Do not use automated or ultrasonic reprocessing.

Procedure
The otoscope and the insufflation bulb must be cleaned on the outside manually with a damp cloth and inside with a cotton-wool bud (wipe cleaning and wipe disinfection).

Recommended agents
Cleaning agent: Neodisher® MediClean
Disinfectant agent: alcoholic (e.g. Incides® Tissues)
The reusable tips must be cleaned and disinfected manually (in an immersion bath) or by machine.
Recommended agents
Cleaning agent: Neodisher® MediClean
Disinfection agent EU: alcoholic (e.g. rotasept®); USA: Cidex®OPA Solution
The reusable tips can be reprocessed up to 360 cycles.
Disposable tips (HEINE AllSpec®/HEINE UniSpec®) are for single use only.

Changing the light source
⚠️ Verify that the lamp voltage complies with the supply voltage of the handle.
Allow the device to cool down before changing the bulb.

BETA200 LED VET
With these otoscopes, the LED cannot be changed.

BETA200 VET
• Detach the otoscope from the handle and pull the bulb out of the guide tube of the connector plug (4) holding it at the narrow collar.
• Insert the new lamp all the way into the lamp guide.

BETA100 VET
• If necessary, remove the speculum and adaptor from the instrument and pull off the bulb cover.
• Unscrew the bulb from the head.
• Check that the bulb voltage is the same as the voltage of your handle or power supply and screw in the new bulb.
• Push on the bulb cover as far as it will go. The cover reduces stray light in the specula and should always be fitted.

Maintenance and Service
The instruments do not require maintenance or service.

General Warnings
⚠️ Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.
Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).
This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.
Do not modify the device.
Use only original HEINE parts, spare parts, accessories and power sources.
Repairs shall only be carried out by qualified persons.
Do not shine light of the instrument directly into the eyes.
Check the tip before each use for rough surfaces and edges.
In order not to exceed the surface temperature of 41 °C of the application part, keep within the application duration.

General Notes
🔦 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

Disposal
🗑️ The product must be recycled as separated electrical and electronic devices.
Please observe the relevant state-specific disposal regulations.

Electromagnetic Compatibility
Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

⚠️ This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.

The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission and reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices. If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains following tables
- Guidance and manufacturer's declaration – Electromagnetic immunity
- Technical specification
- Explanation of the used symbols

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| Guidance and manufacturer's declaration – electromagnetic emissions | | | |
|--|----------------|---|--|
| The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment. | | | |
| Emission test | Compliance | Electromagnetic environment – Guidelines | |
| RF emissions CISPR11 | Group 1 | The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely that any interference in nearby electronic equipment. | |
| RF emissions CISPR 11 | Class B | The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. Warning: This device is intended only for use by medical professionals. This is a device of class A CISPR 11 in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site. | |
| Harmonic Emissions IEC 61000-3-2 | not applicable | | |
| Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3 | Passed | | |

| Guidance and manufacturer declaration - Electromagnetic immunity | | | |
|--|--|--|---|
| The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment. | | | |
| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment – Guideline |
| Electrostatic discharge (ESD) acc. to IEC 61000-4-2 | ± 6 kV contact discharge ± 8 kV air discharge | ± 6 kV contact discharge ± 8 kV air discharge | Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %. |
| Electrical fast transient/ burst IEC 61000-4-4 | ± 2 kV for mains cables ± 1 kV for input and output lines | ± 2 kV for mains cables ± 1 kV for input and output lines | The supply voltage quality should be that of a typical commercial or hospital environment. |
| Surge IEC 61000-4-5 | ± 1 kV voltage phase – phase, ± 2 kV voltage phase – earth | ± 1 kV voltage phase – phase ± 2 kV voltage phase – earth | Mains power quality should be that of a typical commercial or hospital environment. |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | <5% U _r , (>95% dip in UT) for 12 periods 40% U _r , (60% dip in UT) for 5 periods 70% U _r , (30% dip in UT) for 25 periods <5% U _r , (>95% dip in U _r) for 5 seconds | <5% U _r , (>95% dip in UT) for 1/2 period 40% U _r , (60% dip in UT) for 5 periods 70% U _r , (30% dip in UT) for 25 periods <5% U _r , (>95% dip in U _r) for 5 seconds | Mains power quality should be that of a typical commercial or hospital environment. If the user of the device during requires continued operation under power mains interruptions, it is recommended that |