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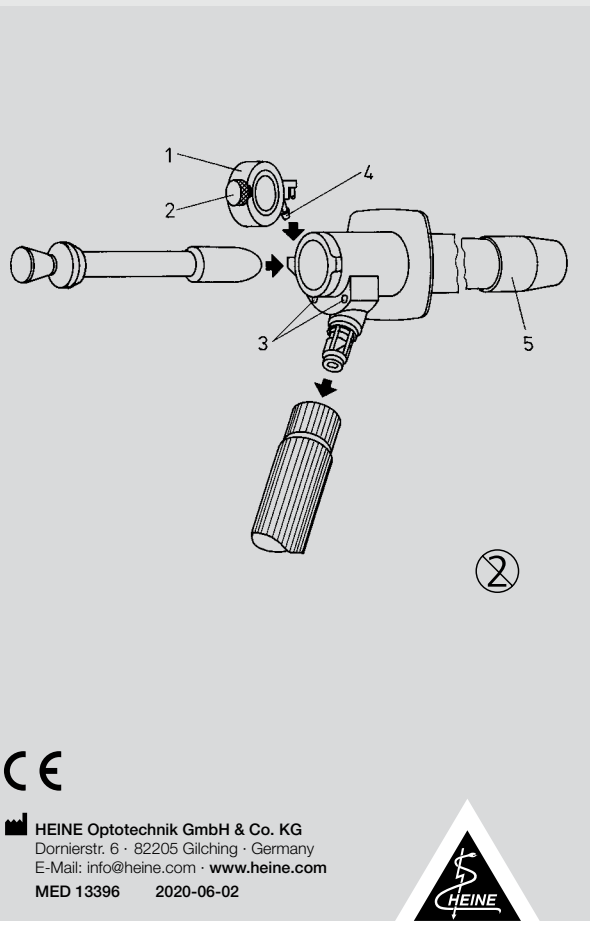
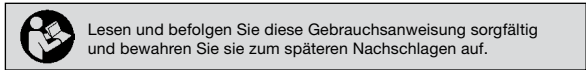
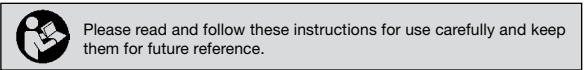
## Instructions for use

Please read and follow these instructions for use carefully and keep them for future reference.

### Safety information

**Range of applications:** HEINE anoscopes/proctoscopes and sigmoidoscope with fiber optic illumination are designed only for examination and treatment of the anal passage.

**Safety in use:** Use only light sources which are approved for use in a medical environment. Tubes with straight direction of view should be inserted when an obturator is fitted.

#### 1. Anoscope

Attach the instrument to the handle of the light source. Turn the obturator clockwise to lock in position in the tube. To unlock, turn anticlockwise. The cone of the obturator and the tube should be lubricated with a suitable lubricant.

#### 2. Proctoscope and sigmoidoscope

Attach the instrument to the handle of the light source. Make sure that the end-piece (5) is firmly screwed-in. If required, connect an insufflation bulb to the window of the instrument (4).

**Loosen the sealing screw (2) and open the window (1).** If the window is not required during the examination, it can be removed by extracting it upwards out of the instrument head.

**Insert the obturator. Apply a suitable lubricant to the tip of the obturator and the tube.**

The plane glass window (1) can be opened or closed at any time. The instrument can be made airtight by gently tightening the screw (2). If an insufflation bulb without a valve is used, the insufflated air can be regulated by gently loosening or tightening the screw.

If magnification of the object is required, insert the loupe into one of the two holes provided (3). The loupes are marked P for proctoscope and R for rector-sigmoidoscope. Both swivel to allow the window to be opened or closed. The telescope is inserted into both of the location holes (3) and clamped by means of a clamping screw. During instrumentation, the telescope can be folded downwards.

The vacuum tube accessory can also be located in the holes (3) by means of the attachment. It can then be moved backwards or forwards along the tube and remains steady in any given position.

**This does not limit the use of a loupe in any way.**

### ⚠ Use of the Rubber Insufflation Bulb:

Always use the HEINE Hygiene Filter (8) when connecting the rubber insufflation (9) bulb to the HEINE Proctoscopes/Sigmoidoscopes (6). These filters are intended for single use only and prevent a contamination of the rubber insufflation bulb. For this, connect the transparent hose (7) to the insufflation port on the HEINE Proctoscopes/Sigmoidoscopes. Control the connection between the transparent hose and the HEINE Hygiene Filter. If it isn't securely attached, fix it more tightly. Then connect the Hygiene Filter to the rubber insufflation bulb.

**Caution!** Infection! The use of the instrument without the HEINE Hygiene Filter may lead to contamination of the rubber insufflation bulb and thereby to an infection of the following patients. Therefore always use the filter. If you have forgotten to use the filter, replace the rubber insufflation bulb.

**LATEX** Risk of allergic shock: The rubber double bellows contain natural rubber latex. Do not use it in patients with a latex allergy.

**Caution!** Infection! The filter is a single use product. If it is reused, the risk of infection increases. The filter is not suitable for being cleaned, disinfected and sterilized.

### Cleaning/Disinfection and Sterilisation

**Preparation at the point of use/Initial Cleaning:** Residues and other deposits must be removed immediately after use to avoid any residues drying on to the instrument.

**Disassembly:** The instrument must be removed from the light source for further preparation steps. The article must be disassembled (see picture) to ensure a satisfactory cleaning result.

### TUBES AND OBTURAR

#### Manual disinfection and cleaning

For manual cleaning and disinfection, only those cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastics may be used. The manufacturer's recommendations for use must be followed to avoid material damage / corrosion. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid surface residues.

#### Machine cleaning and disinfection / Thermal Disinfection

The article can be machine-cleaned and disinfected at up to ≤ 93 °C. Only cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastic may be used. The instructions of the manufacturer of both machines and detergents must be followed. The machine should be loaded correctly so that the cleaning procedure is successful. Rinsing procedures must ensure that no deposits are left. The final rinse should be carried out using de-mineralised water.

**Ultrasonic Cleaning** is not allowed for articles with fiber optics.

#### Sterilization

We recommend sterilization with saturated steam at 134 °C for 5 minutes. Cycles of 18 minutes are allowed. There may be a reduction in light transmission with frequent/repeated long-term steam sterilization, particularly if the steam quality is not ideal.

#### Telescope/Rubber-Insufflations Bulb

Telescope and Rubber-Bulb can be treated by surface disinfection. Low temperature Gas-Plasma-Sterilization is permitted for the Telescope.

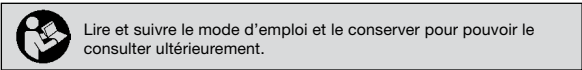
**Warning:** Non-suitable or incorrectly-used chemicals or preparation procedures can lead to irreparable damages to the article.

⊗ : Single use only

### General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

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