HEINE QUALITY
MADE IN GERMANY

HEINE® EasyClean Laryngoscope Handle

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
MED 113856 2018-02-06

HEINE® EasyClean Laryngoscope Handle

DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ESPAÑOL	9
ITALIANO	11
SVENSKA	13
NEDERLANDS	15
DANSK	17
NORSK	19
SUOMI	21
PORTUGUÊS	23

HEINE® EasyClean

Laryngoskopgriff

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HEINE® EasyClean

Laryngoscope Handle

 Please read and follow these instructions for use of and keep them for future reference.

Intended Use

The internally powered HEINE laryngoscope handle is intended for the transient use with laryngoscope blades according to the "green standard" ISO 7376 for tracheal intubation.

It should only be operated by qualified medical personnel within a professional healthcare facility or emergency medical services environment.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

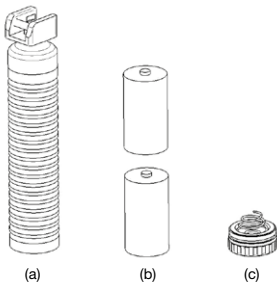
Warnings and Safety Information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation, maintenance or repair, as applicable. Notes are important, but not related to hazardous situations.

Product overview

- (a) Handle
- (b) Batteries
- (c) Bottom insert


Setting up

Insert the batteries (b) into the handle (a) leading with the positive terminal. Screw the bottom insert (c) into handle (a).

Operation

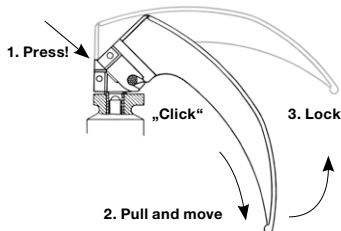Fix the blade into the working position (see fig.).

Before starting a clinical application, the state of the internal electrical power source must be tested by turning on the light source.

 Only use a laryngoscope handle-blade combinations with green markings to ensure proper function.

The HEINE laryngoscope handle corresponds to the "green standard" according to ISO 7376. If non-HEINE laryngoscope blades are used, component compatibility and functionality are to be checked before each use!

Only trained personnel are allowed to use a laryngoscope for intubation purposes.

HEINE laryngoscopes are designed for use < 5 minutes.

Check the correct operation of the device before each use! Do not use the device if you notice any moisture in the device or damage to the device. (e.g. rough surfaces and edges, etc.)

Before each use, check the condition of the connector for damages, wear and tear. If defects exist, do not use the laryngoscope handle and replace it with one that is in working order.

We strongly recommend a functional backup of laryngoscopes in an emergency situation.

Hygienic reprocessing

The instruction is available:

- online at www.heine.com

- in a paper version which you can request from the address listed

Maintenance

Routinely inspect the condition of the batteries and replace them if necessary.

Changing the batteries

Unscrew the bottom insert from the handle shell and replace the batteries.

In order to ensure a secure electrical connection, use only batteries from renowned manufacturers with the size C, LR14.

Check the correct function of the device after each battery change.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

If you don't use the device for a longer period of time please remove the batteries in advance.

General Warnings

 Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not shine directly in the eyes to avoid dazzle, especially if the pupils are dilated. Do not use the laryngoscope for eye reflex testing.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the laryngoscope handle, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Not applicable
Voltage Fluctuations/Flicker	Not applicable

Technical specification

Environmental conditions for operation	+10 °C to + 35 °C 30 % to 80 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage (without batteries)	-40 °C to +70 °C up to 95 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport (without batteries)	-40 °C to +70 °C up to 95 % rel. humidity 500 hPa to 1060 hPa
Nominal voltage	2 x Battery 1.5 V (Size C/LR14)
Nominal current	max. 250 mA
Class	Internally powered
Classification according to IEC 62471	Group 2
Protection class	IP68
Applied Part	Type BF

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