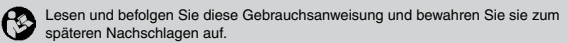
HEINE Ladebatterien

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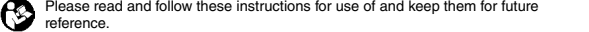
HEINE Rechargeable Batteries

These instructions apply to the following products of the HEINE Rechargeable batteries/ battery insert series: NiMH, Li-ion, Li-ion L, M3Z, mini2Z, LED S2Z, LED M2Z, K3Z, M3Z 4 NT Li-ion, M2Z 4 NT Li-ion, K3Z 4 NT Li-ion, S2Z 4 NT Li-ion.

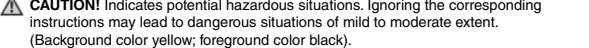
- Please read and follow these instructions for use of and keep them for future reference.

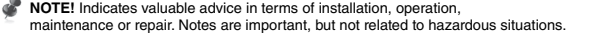
Intended Use

The HEINE rechargeable batteries are intended solely for HEINE rechargeable handles.

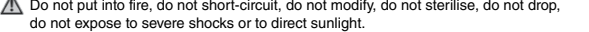
-  **For U.S. only: Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

General Warnings

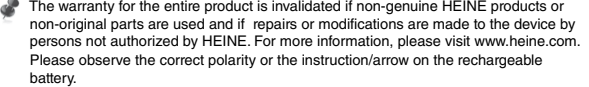
Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight.

Dispose of batteries in the event of visible or presumed damage (e.g. leaking liquid, mechanical damage).

The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water.

After hygienic reprocessing of the base caps, allow 15 min for the cleaning agent to evaporate before using the handle.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com. Please observe the correct polarity or the instruction/arrow on the rechargeable battery.

Check the rechargeable batteries regularly. In case of any abnormalities in the operating or recharging times, replace the rechargeable batteries.

If the rechargeable battery has not been recharged for a long period it may discharge automatically. If this occurs a longer charging time will be required when recharging.

Disposal

- Dispose at your local collection point.

The appendix contains following tables:

- Technical specification
- Technical specifications of the rechargeable batteries
- Explanation of the used symbols

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Technical specification				
Environmental conditions for operation	+10 °C to +35 °C <p>30 % to 75 % rel. humidity <p>700 hPa to 1060 hPa</p></p>			
Environmental conditions for storage	+5 °C to +45 °C <p>45 % to 80 % rel. humidity <p>500 hPa to 1060 hPa</p></p>			
Environmental conditions for transport	-20 °C to +50 °C <p>45 % to 80 % rel. humidity <p>500 hPa to 1060 hPa</p></p>			
Recommended charging temperature of the rechargeable batteries	+10 °C to +35 °C			

	HEINE Li-ion L	HEINE Li-ion	HEINE NiMH	HEINE mini2Z
	(X-007.99.383)	(X-007.99.381)	(X-002.99.382)	(X-001.99.487)
Cell type	Lithium-ion technology		Nickel-metal hydride	
Rated voltage [V]	3.7		3.6	2.4
Capacity [mAh]	2300		1000	750
Charging time NT300/miniNT [h]	approx. 2 (fast charging)		approx. 4 (fast charging)	
Charging time NT200 [h]	24		14	–

	M2Z 4 NT Li-ion	M3Z 4 NT Li-ion	S2Z 4 NT Li-ion	K3Z 4 NT Li-ion
	(X-007.99.104)	(X-007.99.380)	(X-007.99.304)	(X-007.99.394)
Cell type	Lithium-ion technology			
Rated voltage [V]	2.5	3.5	2.5	3.5
Capacity [mAh]	800			
Charging time NT4 [h]	approx. 2.75			

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Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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**HEINE Optotechnik GmbH & Co. KG**