

HEINE OMEGA 600

A quick look at the most important features

We thank you for choosing HEINE OMEGA 600 and wish you a lot of success and fun working with your new instrument.

If you want to share your experiences or discoveries, we would be highly pleased to see it: #heine #iuseheine #OMEGA600



USB-C Charging socket
Charge with USB-C cord

Charge status indicator

● ● ● ●
66–100 %

● ● ● ●
33–66 %

● ● ● ●
10–33 %


● ● ● ● * flashes
< 10 %

Brightness Control
On/off, stepless adjustment of the light intensity from 3–100 %, and visionBOOST from 100–245 % for up to 20 % better view in media opacities.
Can be positioned for right and left-handed users.

Filter selection lever



Red-free, yellow and cobalt blue filters

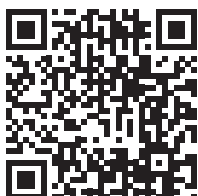

The video “How to set up your OMEGA 600” explains how to adjust the instrument to your personal needs.



HEINE OMEGA 600

A quick look at the most important features



Flip-up function of the optics unit
For more convenient patient interactions



Eyepieces
To best accommodate your view, exchange the initially mounted eyepieces +2D with the neutral lenses 0D (supplied with the instrument)



Teaching mirror TM1



Dust cover for front glass



Width adjustment

