**HEINE** QUALITY  
MADE IN GERMANY

## ∴ HEINE E4-USB Power Supply

 HEINE Optotechnik GmbH & Co. KG  
Dornierstr. 6 · 82205 Gilching · Germany  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
MED 112712      2020-08-10

V-200.00.137

# HEINE E4-USB Power Supply

<b>DEUTSCH</b>	<b>3</b>
<b>ENGLISH</b>	<b>6</b>
<b>FRANÇAIS</b>	<b>9</b>
<b>ESPAÑOL</b>	<b>12</b>
<b>ITALIANO</b>	<b>15</b>
<b>SVENSKA</b>	<b>18</b>
<b>NEDERLANDS</b>	<b>21</b>
<b>DANSK</b>	<b>24</b>
<b>SUOMI</b>	<b>27</b>
<b>PORTUGUÊS</b>	<b>30</b>

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# HEINE E4-USB

 Please read and follow these instructions for use of and keep them for future reference.

## Intended Use

The HEINE E4 USB is intended for charging exclusively with the HEINE USB power supplies. It should only be operated by qualified medical personnel within a professional healthcare facility.

 **For U.S. only:**  
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

## Warnings and Safety Information

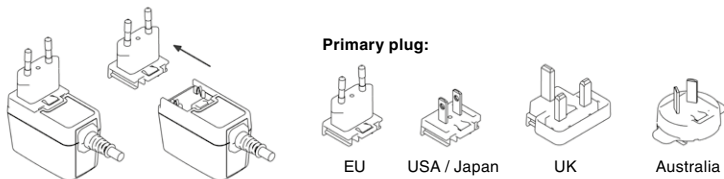
 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

## Product overview

## Setting up


The mains socket adapter of the power supply is interchangeable, this enables worldwide operation. The pictograms describe the procedure of changing the adapters.

Connect the USB cable to the HEINE instrument.

Place the power supply in such a position where it is accessible and easily unplugged.

 After you checked the mains voltage, connect the switch mode power supply to the mains supply. When taking the instrument out of service, proceed in reverse order.

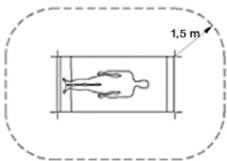The HEINE Charger is intended for installation in medical areas outside the patient environment (at least 1.5 metres from the patient or patient support pursuant to IEC 60601-1, see figure).

### Hygienic Reprocessing

The instruction is available:

- at [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

### Maintenance

The device is maintenance free.

### Service

The device has no components serviceable by the end-user.

### General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

### General Warnings

 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

The device must not be placed near strong Magnetic fields, e.g. MRI units.

Do not use the power supply outdoors.

The device must not be used with appliance outlets of extension leads.

The power supply is only approved for the use with ungrounded loads (Class II).

Before operating the device, verify that the mains voltage corresponds to the value indicated on the power supply.

Heating of the housing during operation is normal and harmless. Make sure that the device isn't covered during operation, that it isn't placed in close proximity to a heating and that isn't exposed to direct sunlight.

Use this power supply only for the supply of the devices it is intended for.

## Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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## Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Nominal input voltage	100 – 240 V AC / 50 – 60 Hz
Power consumption	< 15 W
Output voltage	5 V
Output current	max. 1.2 A
Protection class	II
Protection provided by enclosures (IP code)	IP 20

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