## HEINE VET Otoscopes

HEINE Optotechnik GmbH & Co. KG  
Dornierstr. 6 - 82205 Gilching - Germany  
E-Mail: info@heine.com - www.heine.com  
MED 11284 2020-10-02

### HEINE VET Otoscopes

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## HEINE VET Otoscopes

**ENGLISH**  
These instructions apply to the following products of the HEINE Otoscope series: BETA200 LED VET, BETA200 VET, BETA100 VET

Please read and follow these instructions for use and keep them for future reference.

#### Intended Use

HEINE otoscopes are intended for the examination of the ear, the external auditory canal and the tympanum. The instruments have an optical magnifying equipment and a battery-rechargeable battery-powered illumination equipment. The instruments may only be used by qualified medical personnel.

#### For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

#### Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

#### Product overview

- |                     |                                   |
|---------------------|-----------------------------------|
| 1 Slit              | 6 Insufflation bulb               |
| 2 Viewing window    | 7 Insufflation port connector     |
| 3 Insufflation port | 8 HEINE SANALON S tips (re-use)   |
| 4 Connector plug    | 9 HEINE AllSpec tips (single use) |
| 5 HEINE Handle      |                                   |

#### Setting up

To set up the instrument, fit the instrument head onto the HEINE battery handle or the HEINE rechargeable handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE XHL 2.5 V bulb

only for use with the HEINE BETA Battery handle

Red ring = HEINE XHL 3.5 V bulb

Black ring = HEINE LED illumination

only for use with the HEINE BETA Rechargeable Handle

#### Operation

The HEINE otoscope may only be inserted into the auditory canal when a tip is fitted.

#### BETA200 LED VET / BETA200 VET

Place the HEINE tip on the otoscope so that the projection inside the tip fits into the bayonet slit (1). Tighten the tip by twisting it slightly to the right.

#### BETA100 VET

Specula with bayonet connector

Fit the speculum to the head so that the locating pin fits the groove provided. Then twist the profiled ring as far as the stop.

Disposable AllSpec specula can be used if the adaptor B-000.11.306 is fitted.

SANALON S specula

The adapter G-000.21.214 must be fitted if these tips are used. Fit the speculum to the adaptor so that the locating pin fits in the groove provided. Twist the profiled ring as far as the stop.

The magnifying lens is inside the viewing window (2). To facilitate instrumentation, the viewing window can be swivelled to both sides or folded up.

The otoscopes have a connection port (3) for the optionally available insufflation bulb (6, 7).

The test of tympanic mobility can be carried out when the viewing window is closed using the insufflation bulb. Apply the pressure carefully with the insufflation bulb.

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**Statement for the operational environments**

Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.

**Performance features of the ME system that have been determined to be essential to the performance**

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Spaulding Classification USA: noncritical

**Technical specification**

Environmental conditions for operation +10 °C to +35 °C  
30 % to 75 % rel. humidity  
700 hPa to 1060 hPa

Environmental conditions for storage +5 °C to +45 °C  
45 % to 80 % rel. humidity  
500 hPa to 1060 hPa

Environmental conditions for transport -20 °C to +50 °C  
45 % to 80 % rel. humidity  
500 hPa to 1060 hPa

Nominal voltage 3.0 V - 3.7 V

Nominal current XHL: max. 760 mA  
LED: max. 350 mA

Protection class Internal power supply

Device classification according to IEC 62471 free group

Application part Type BF

**XHL Xenon Halogen Bulbs**

BETA200 VET #077 (2.5 V), #078 (3.5 V)  
BETA100 VET #037 (2.5 V), #049 (3.5 V)

## Electromagnetic disturbances – Requirements and tests

&lt;p