HEINE® iC2 Funduscope Quick Start Guide

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1

Download on the
App Store

Download the HEINE iC2 App in the App Store. Open the App, follow Quick Start.

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4

The HEINE iC2 switches off automatically after 30 seconds of inactivity. Switch off the iPhone* to save energy.

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2

Place the iPhone* in the appropriate mobile phone case until it engages. Please make sure the camera is in the correct position.

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5a

Charging the HEINE iC2 with the supplied USB cord. Completely charged after typ. 5.5h. The iPhone* has to be charged separately.

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3

Attach the mobile phone case to the HEINE iC2 by sliding it upwards until it engages.

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5b

Set it down and charge in one step. The new iC2 charging station is the world's first inductive charging station to receive approval for a medical product. Say goodbye to charging cables!

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*Apple, the Apple logo and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

HEINE® iC2 Funduscope application situation

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1 Before the examination, adjust a refractive error of the patient via the diopter wheel.

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2 Start the app and activate the image capture mode, push the trigger button on the device, the light has to be on. Position the device with a distance of approx. 10 cm to the patient's eye and monitor the eye to be examined on the display. A red reflex should be visible already. Approach the eye slowly until approx. 3 cm until fundus structures are identifiable on the display. The left hand stabilizes the instrument at the patient's head and may lift the upper eyelid.

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3 Activate the autofocus by tapping on the display or pushing the trigger button to the pressure point.

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4 Take a foto by tapping on the trigger button in the display or pushing the trigger button on the device completely.

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5 In order to view the images, activate the display mode.

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