



HEINE CHANGE System Module: Smartphone Adapter

For use with the HEINE BETA  Otoscope


IT'S REALLY QUITE SIMPLE!

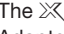


The  CHANGE System Smartphone Adapter allows you to use any smartphone. It is optimised for the Apple iPhone* in conjunction with the HEINE Auris app.


 The adapter is supplied with two fixing buttons at different depths. Please check which one is suitable for your smartphone.



 Keep the fixing button [01] separate from the magnetic holder before attaching it to the phone case!

Should both components still end up sticking together, separate the screw [02] from the adapter arm. Turn the screw directly into the magnetic holder's thread and push the two elements apart.

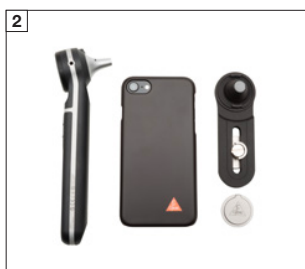

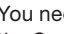You need the BETA  Otoscope and the Smartphone Adapter (consisting of fixing button + adapter), as well as your smartphone and a suitable phone case**.

The phone case should have a hard polycarbonate or acrylic glass back.



Now remove the protective film and press the fixing button firmly onto the phone case on the right below the selected camera lens.



Next, attach the adapter to the fixing button.




Loosen the screw slightly.

If your smartphone has just one camera, fix the adapter in the centre in front of this camera.

If your smartphone has multiple cameras, first determine which of the cameras is the main camera: Place the adapter arm on the back of the smartphone so that one of the camera lenses is in the centre.

Tighten the screw only slightly so that fine adjustment is still possible later.



Release the  CHANGE System Magnifier Module with a twisting motion and remove it from the otoscope.



In place of the Magnifier Module, position the smartphone adapter so that it is flush, and turn it clockwise to secure it. Check that the adapter is sitting completely flat against the entire surface and is fully engaged.



To make fine adjustments, adjust the smartphone camera with the otoscope so that a clear image is visible on the switched-on display. Tighten the screw on the adapter.

You can now use the camera function on your smartphone as normal to take photos.

*iPhone is a trademark of Apple Inc..

**The smartphone and phone case are not included in the scope of delivery.

