

HEINE QUALITY
MADE IN GERMANY

HEINE EASYCLEAN LED LARYNGOSCOPE HANDLE

In situations, where
you need 100 %
reliable performance.
Easy reprocessing to
ensure patient safety –
whatever national
standards require.



RELIABLE PERFORMANCE. EASY REPROCESSING. FOR PATIENT SAFETY.



Simple, fast and safe cleaning

- ▶ Reliable wipe disinfection of the handle after use ('Intermediate Disinfection') thanks to non-slip "Wave Design".
- ▶ 100% waterproof. In the case of immersion disinfection ('High-Level Disinfection') or low temperature sterilisation processes (Sterrad®, VHP® (Steris)), neither the batteries nor the LED light unit need to be removed.
- ▶ Autoclavable LED technology: No removal of the light unit required for use in autoclaves, just batteries.

Fail-safe

- ▶ Typ. 10 hours of operating time (EasyClean LED laryngoscope handle in continuous operation).
- ▶ Fade-Out Feature: brightness reduces slowly as battery operating time diminishes, for even more efficient battery use.
- ▶ Compatible with all laryngoscope blades according to ISO 7376 (Green Standard).

Hygienic reprocessing options:*

Reprocessing process →	Wipe disinfection	Manual immersion disinfection	Low temperature gas plasma sterilisation (STERRAD® /VHP® (Steris))	Mechanical cleaning + disinfection (thermal disinfectant)	Steam sterilisation / autoclave
Temperature	Cold	< 60 °C	< 60 °C	max. 93 °C	max. 134 °C / 4 min
with batteries	✓	✓	✓	Batteries removed previously	Batteries removed previously
with HEINE LED	✓	✓	✓	✓	✓

* Detailed instructions can be found on heine.com

Perfect view

- ▶ There's LED. And there's LED^{HO}. HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible. That's what we call LED in HEINE Quality – or LED^{HO}.
- ▶ Maximum brightness: 3x brighter than conventional halogen light.
- ▶ Virtually unlimited hours of working life.

Laryngoscope handles – Ordering Information

EasyClean LED – STANDARD	F-008.22.820
EasyClean LED – SLIM	F-008.22.821
EasyClean LED – SHORT	F-008.22.822

- ▶ [Download Book of Truth for more information on our laryngoscope blades: www.heine.com/BoT](http://www.heine.com/BoT)