⚡ Switch Mode Power Supply HEINE® E7

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 2412 1/2.13

Switch Mode Power Supply HEINE® E7

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
SUOMI	24
PORTUGUÊS	27

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Please read and follow these instructions for use and keep them for future reference.

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries).

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criterions: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

Intended Use

The HEINE® E7 Switch Mode Power Supply is exclusively intended for the power supply for HEINE diagnostic instruments.

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Product overview

Primary plug:

UK

USA/Japan

EU

Australia

The primary plug is interchangeable. This allows a world wide use.

Initiation and handling of HEINE® E7

 The power supply may only be used with the HEINE Lamp handle and the suitable 6 V instruments.

- Plug the connector (RCA plug) into the HEINE diagnostic instrument.
- After you checked the mains voltage, connect the switch mode power supply to the mains supply.
- When taking the instrument out of service, proceed in reverse order.
- For disconnection pull the plug of the instrument cord out of the connector socket while turning it slightly.

When the device is ready for use, the operation indicator on top of the housing lights up green. If the display remains dark despite of existing mains supply, the power supply is defect and has to be send in for repair or the load is causing an output short-circuit. In this case the load has to be checked externally.

Cleaning

-  Disconnect the mains plug before cleaning. Do not clean with chemical detergents.
-  Clean only with a dry cloth. Do not use solvents.
-  In order to avoid damages, do not immerse, sterilize or disinfect the device with spray.

Maintenance

The device does not require any regular maintenance.

Service

The device does not require regular service.

-  Do not open or modify the device. Repairs are only to be performed by qualified personnel.

Disposal

-  The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Technical specification

Nominal input voltage	100-240V ~
Power consumption	<15W
Output voltage	6.5V
Output current	1.7A
Protection class	II
<i>Operating conditions:</i>	
Permissible Ambient temperature	+0°C to +40°C
Rel. humidity:	10% to 90% (noncondensing)
<i>Transport and storage conditions:</i>	
Permissible Ambient temperature	-30°C to +70°C
Rel. humidity:	10% to 95% (noncondensing)

General Notes and Warnings

This product is a high-quality precision instrument. Handle it with care and always store it in a clean, dry place.

-  Check the correct operation of the product at regular intervals. In the case of visible damages on the housing or on the cord do not use the power supply.
-  Do not use the device near strong magnetic fields like MRI scanners.
-  Do not use the power supply outdoors.
-  The device must not be used with appliance outlets of extension leads.
-  The power supply is only approved for the use with ungrounded loads (Class II).
-  Before operating the device, verify that the mains voltage corresponds to the value indicated on the power supply.
-  Heating of the housing during operation is normal and harmless. Make sure that the device isn't covered during operation, that it isn't placed in close proximity to a heating and that isn't exposed to direct sunlight.

-  Use this power supply only for the supply of the devices it is intended for.
-  Do not use the device in hyperbaric chambers, in explosive or oxygen loaded environments!
-  To put the device into service, first connect the cord to the load, and then plug the power supply into the socket. When taking the instrument out of service, proceed in reverse order.
-  The device must not come into contact with liquids.
-  The performance and safety of the device can only be guaranteed when fitted with original HEINE accessories and HEINE spare parts. Otherwise the warranty is terminated.
-  A modification of the power supply is not allowed (Loss of warranty).

Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

	The CE marking indicates that the product conforms to the specifications of Directive 93/42/EEC concerning medical devices.
	Catalogue number
	Serial number
	Manufacturer
	Date of manufacture
	Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal.
	Temperature limits in °C for storage and transport
	Temperature limits in °F for storage and transport
	Humidity limitation for storage and transport
	Pressure limits for storage and transport
	Protection Class II
	For indoor use only
	UL approval mark. Valid in the USA and Canada
	ENEC European mark of conformity
	Please read and follow the instructions for use and keep them for future reference (Background: blue; Foreground: white)
	Keep dry!
	"Grüner Punkt" (country-specific)

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