HEINE QUALITY
MADE IN GERMANY

∴ HEINE FlexTip®+ Laryngoscope Blade

CE

 HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 0698 2020-11-06

V-200.00.110

HEINE FlexTip+ Laryngoscope Blade

DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ESPAÑOL	9
ITALIANO	11
SVENSKA	13
NEDERLANDS	15
PORTUGUÊS	17

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 Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE FlexTip+ laryngoscope blade is used exclusively for endo-tracheal intubation. Only persons with training in endo-tracheal intubation are authorised to use it.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

Operate

The movable blade tip can cause injury to the patient if it is not used according to these instructions and with a normal degree of caution.

- Before use, ensure that the instrument is free from damage with e.g. no sharp edges etc.
- Attach the blade to the handle in the usual way and hinge upwards into the working position.
- Check that the light is bright enough (batteries!).
- During intubation hold the laryngoscope by the handle, do not operate the lever.
- When the tip of the blade locates in the vallecula, operate the lever carefully and no further than necessary with the thumb to lift the epiglottis.
- Release the lever before extracting the blade.

Hygienic reprocessing

Cleaning, disinfection and sterilisation

The following instructions must be incorporated into the hospital or practice procedures. National guidelines and regulations (e.g. regarding CJK/vCJK) and the relevant standards and laws must be put into practise.

These procedures must be carried out by suitably-qualified personnel.

Procedure after use

Remove all traces of contamination immediately after use to avoid incrustation.

Manual cleaning / disinfection

Use only cleaning and disinfection products which are recommended by the manufacturer for use with fiber optics and plastics. Follow the manufacturers instructions regarding concentration and duration of use exactly. After this process, rinse the blade with de-mineralised water and ensure that no deposits remain on them.

Warning: Use of the wrong cleaning and disinfection products or methods can cause irreparable damage to the laryngoscope.

Ultrasonic cleaning is not permitted!

Automatic cleaning / disinfection

The laryngoscope can be cleaned or thermally disinfected at up to 93 °C in an automatic machine. Use only cleaning products, which are recommended for use with stainless steel and plastic. The rinsing procedure must ensure that no deposits are left on the product. Use only de-mineralised water for the final rinse.

Sterilization

Laryngoscopes have to be sterilized as required by national regulation and internal institutional standards. We recommend the sterilization with saturated steam at 134 °C for 4 minutes. Cycles of up to 18 minutes or up to 60 minutes in specified cases should not damage the metal part of the blade. There may be a reduction in light transmission with frequent / repeated long-term steam sterilization, particularly if the steam quality is not ideal.

It is the responsibility of the end user to ensure that the cleaning and sterilization is actually performed using appropriate equipment, materials, and personnel to achieve the desired result. This normally requires validation and routine monitoring of the process. Any deviation from these instructions should be evaluated for effectiveness and potential adverse consequences.

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

Disposal

Please observe the relevant state-specific disposal regulations.

Applicable standards: DIN EN ISO 7376

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

This product is a precise optical instrument. Please handle this product with care and store it in a clean place.

General Warnings

 Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.

Use only laryngoscope handles with a green identification mark. The HEINE F.O. angled handle should not be used with HEINE FlexTip+. The lever mechanism and moving parts are not designed to operate under heavy loads. Do not bend or use unnecessary force!

Bring the blade into the normal working position and check that the light switches on.

The bulb brightness can reduce during use if the rechargeable battery was not charged sufficiently before use. We recommend a minimum recharge time of 10 minutes for HEINE rechargeable batteries in HEINE chargers.

After some minutes' use the laryngoscope bulb will be hot.

The laryngoscope may only be used by trained personnel.

To avoid cross-infection, the laryngoscope must be cleaned and disinfected or sterilized after each intubation (see cleaning instructions).

Where the instrument is subject to frequent use we recommend that a spare instrument and a small stock of replacement bulbs and batteries should be permanently available.

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

The appendix contains following tables

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