∴ HEINE XP Disposable Laryngoscope Handle Shell

 **HEINE Optotechnik GmbH & Co. KG**
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 0615 2021-09-02

HEINE XP Disposable Laryngoscope Handle Shell

DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ESPAÑOL	9
ITALIANO	11
SVENSKA	13
NEDERLANDS	15
DANSK	17
SUOMI	19
PORTUGUÊS	21

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HEINE XP Disposable Laryngoscope Handle Shell

ENGLISH

 Please read and follow these instructions for use of and keep them for future reference.

Intended Use

This XP handle shell is designed for single use only for endo-tracheal intubation with a fiber optics laryngoscope blade or XP blade and will only function when fitted with a HEINE Standard (XHL/LED) laryngoscope inserts including bottom insert. It should only be operated by qualified medical personnel within a professional health-care facility or emergency medical services environment.

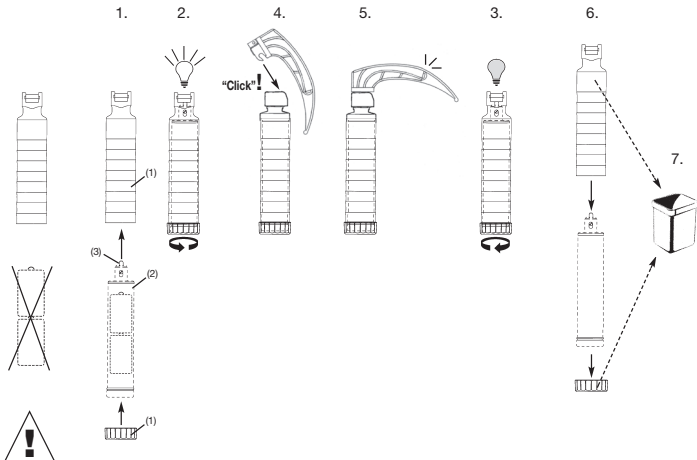
 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Setting up

Operation

We can only guarantee the function of this unit if a HEINE Standard (XHL/LED) laryngoscope insert (2), a HEINE bulb (3) and HEINE spare parts are used.

Bring the blade into the normal working position and check that the light switches on.

The bulb brightness can reduce during use if the rechargeable battery was not charged sufficiently before use. We recommend a minimum recharge time of 10 minutes for HEINE rechargeable batteries in HEINE chargers.

After some minutes' use the laryngoscope bulb will be hot.

The laryngoscope may only be used by trained personnel.

General Notes

- The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Only plastic- and metal blades to the green standard (ISO 7376) may be used.

Cross-contamination can only be avoided if a non-contaminated battery insert is used.

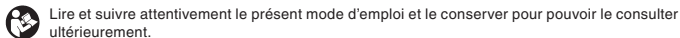
The product can be used up to 5 years from the date of manufacture.

General Warnings

- ⚠ Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.
Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)
Do not modify the device.
Use only original HEINE parts, spare parts, accessories and power sources.
The setup and operation of the HEINE Standard (XHL/LED) laryngoscope insert are described in a separate instruction for use.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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