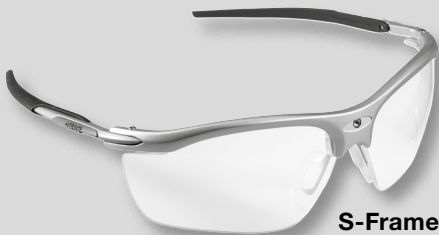
- ∴ **HEINE S-Frame**
- ∴ **HEINE S-Guard**

**S-Frame**

**S-Guard**

 **HEINE Optotechnik GmbH & Co. KG**  
Dornierstr. 6 · 82205 Gilching · Germany  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
MED 2106      2020-08-21

# HEINE S-Frame

## HEINE S-Guard

DEUTSCH	5
ENGLISH	8
FRANÇAIS	11
ESPAÑOL	14
ITALIANO	17
NEDERLANDS	20
PORTUGUÊS	23

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 1

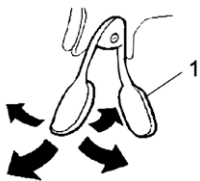
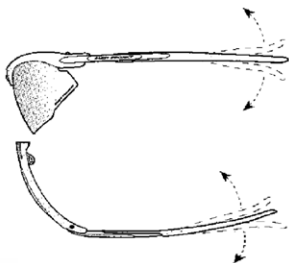
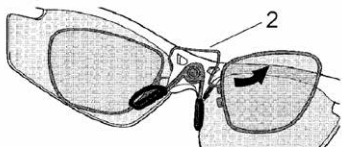
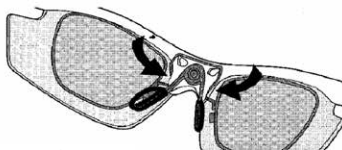
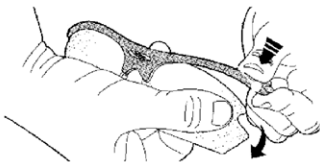
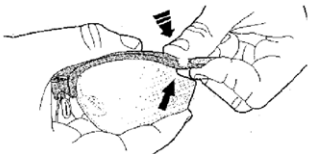
Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 2

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 3

Abb. • Fig. • schéma  
dibujo • Fig. • afb. • Fig. 4

**A****B****C****D****E****F****G**

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 Please read and follow these instructions for use and keep them for future reference.

## Intended Use

The HEINE S-Frame and HEINE S-Guard are designed for use exclusively with HEINE instruments and light sources. It should only be operated by qualified medical personnel within a professional healthcare facility.

 **For U.S. only:**  
**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

## Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

## Operate

**Retaining Cord:** This stabilises your HEINE S-Frame and prevents slipping. Push the ends of the cord firmly over the ends of the temples. Adjust the length of the cord to suit your needs.

**Adjusting the nose bridge:** (A) This can be bent into position to suit the anatomy of your nose. By bending the bridge supports you can vary the distance between the lenses and the eyes.

**Changing the nose pad:** The nose pad is attached to the S-Frame by a screw. To change the nose pad, unscrew the screw and remove the old nose pad.

**Adjusting the temples:** (B) The temple ends can be bent into shape in any direction. Adjust them until they fit comfortably.

**Replacing a temple:** To replace a temple, simply unscrew the hinge screw, insert the new temple and screw in the hinge screw.

**Fitting optional corrective lenses:** Please consult an optician before having corrective lenses fitted. Please show the optician the attached "Instructions for specifying corrective lenses". To fit the corrective lenses (C), insert one side of the snap-in frame (2) into the groove in the S-Frame.

(D) Next, insert the other side of the snap-in frame in the second groove.  
To remove the snap-in frame with corrective lenses, reverse the above procedure, avoiding damage to the lenses.

**Changing protective lenses:** Hold the S-Frame as in (E) and bend the lens in the direction shown and pull the lens with an audible click out of the frame. To replace, follow the procedure shown in (F) and (G) until the lens clicks into place.

If the standard protective lenses are too close to your cheekbones, you can either fit the small protective lenses or adjust the nose bridge (A).

### Instructions for specifying corrective lenses:

When specifying and fitting corrective lenses, be sure to take into account the following:

- The magnifying optics should be attached to the frame to suit your usual working posture.
- Prismatic errors should be corrected.
- The corrective lenses are mounted at an angle to the optical axis.

Check the correction by means of an eye test with the magnifying optics and the corrective lenses fitted. The HEINE SIGMA 250 eyepieces include 2 diopter lenses. This enables normalsighted, emmetropic patients to view the retinal image without effort.

The total spherical correction is usually max. +/- 5 dpt. Ask your optician!

The corrective lenses should be placed as closely as possible to the eyepieces of the magnifying optics.

### Description

a) As a frame with mounting bracket for HEINE Instruments:

- HEINE SIGMA 250 Binocular indirect ophthalmoscope
- HEINE HR Binocular loupes
- HEINE HRP Binocular loupes

b) As eye-protection (Frame without temples), when mounted on the HEINE Professional L Headband fitted with the HEINE mounting bracket for HEINE Instruments:

- HEINE MD 1000 Fiber Optic Headlight
- HEINE ML4 LED HeadLight
- HEINE HR, HRP Binocular loupes

### How to fit the HEINE SIGMA 250 to the HEINE S-Frame

Please follow the instructions supplied with the HEINE SIGMA 250.

### How to fit HEINE Binocular loupes with the HEINE i-View to the HEINE S-Frame

Screw the HEINE i-View with your HEINE Binocular loupe to the fixing plate (Fig. 2). Please follow the instructions for adjusting the HEINE Binocular loupes and HEINE LED LoupeLight according to the instructions supplied with these units.

### Assembly of the S-Guard with other HEINE instruments

Screw your HEINE S-Guard to the

**Professional L Headband** as in Fig. 1.

**ML4 LED HeadLight** as Fig. 3.

**MD 1000 F.O. HeadLight** as Fig. 4.

### Hygienic reprocessing

General: The plastic protective lenses have an anti-reflex coating. Clean them carefully to avoid mechanical or chemical damage.

Cleaning: Clean the S-Frame and S-Guard with the included microfiber cloth.

In addition, we only recommend the use of HEINE Cleaning Spray C-000.32.542.

If other agents are used, for example, 70 % isopropyl alcohol, the durability of the spectacle frame is limited to max. 500 cycles.

**Warning:** Immersion disinfection, machine cleaning/disinfection, ultrasound cleaning and sterilisation are not permitted.

## Maintenance

The device is maintenance free.

## Service

The device has no components serviceable by the end-user.

## General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

This product is a precise optical instrument. Please handle this product with care and store it in a clean place.

## General Warnings

 Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.

To avoid possible damage to the eyes, avoid looking directly at the sun or a bright light!

HEINE declines responsibility for any assembly defects caused by the optician. To avoid possible damage to the frame, avoid frequent bending of the temples.

## Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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