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HEINE Optotechnik GmbH & Co. KG  
Dornierstr. 6 · 82205 Gilching · Germany  
E-Mail: info@heine.com · www.heine.com  
MED 1309 2020-08-20

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DEUTSCH

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## Instructions

ENGLISH

Please read and follow these instructions for use of and keep them for future reference.

### Safety information

#### Symbols:

In accordance with local regulations this product should be disposed of as an electronic device separately.

Double insulated equipment of Protection Class II.

Follow the safety information.

#### For U.S. only:

**Caution: Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

**Range of applications:** The HEINE mini NT is designed exclusively for charging HEINE mini 2000 and mini 3000 rechargeable handles.

**Please note:** The performance of this instrument can only be guaranteed if genuine HEINE instruments and spare parts are used. It should only be operated by qualified medical personnel within a professional healthcare facility.

**General safety:** Under no circumstances should the mains transformer housing or charger be opened. Repairs may only be carried out by qualified personnel (exception: changing the rechargeable battery). Do not use the charger if fit is defective in any way.

**Safe installation:** The charger must be placed on a solid surface. Do not place the mini NT on soft furniture, cloth or carpet. Do not insert any objects in any opening in the housing.

**Safety in use:** The mini NT may only be used at the mains voltage specified on the identification label. Unplug the unit from the mains if not in use over long periods. If liquid enters the housing, ensure that the unit is tested immediately. The mini NT must not be used in rooms where an explosion hazard is present. The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

### Setting up

- The mini NT fulfils the requirements regarding the electromagnetic compatibility. However, its function might be affected by electrical and magnetic fields, such as those which occur in connection with radiological devices or mobile telephones.
- Plug the mains transformer into the mains and the charger is ready for use.
- mini NT ensures that batteries are kept in the ideal state of charge irrespective of handle size.
- When replacing the handles in the charging compartments, ensure that the internal contacts in the compartment are clean and that the instruments are switched off.
- Instruments placed next to each other must not touch each other.
- The HEINE mini NT charger can be used to charge both mini 2000 and mini 3000 handles by means of an adaptor set.
- The adaptors can be replaced simply so that the handles are always correctly positioned. The charger must be disconnected from the mains supply before the adaptors are fitted.
- Ideally, the rechargeable handle should always be replaced and stored in the charger. Handles cannot be overcharged.
- To ensure perfect operation, please allow one second between removal and replacement of the handle.
- The rechargeable battery can only be charged when fitted to the handle.
- The handles will only recharge if the correct mini NT metal bottom inserts are fitted instead of the standard plastic unit normally fitted to the battery handle.
- The pulsating light around the charging compartment shows that current is flowing and that the battery is being recharged.
- When the light is permanently lit, the charging sequence is completed. The battery in the handle is fully-charged and is ready to supply maximum power.
- When the handle is fully-charged, the current to the charger is reduced. This economy status is indicated by the charging compartment ring, which is lit at reduced brightness.
- The light stays off when the charging compartment is empty or of the charge is defective.

### Hygienic reprocessing

- We recommend that external surfaces of the mini NT be wiped clean with a disinfectant cleaning wipe. This should only be carried out by qualified personnel.
- Any disinfectant used must be approved by the disinfectant manufacturer for use on plastics. The manufacturer's instructions must be followed.
- The external surfaces may also be wiped clean with a cloth moistened with water or with a neutral pH or slightly alkaline cleansing agent.
- Isopropyl alcohol may only be used at concentrations up to 70%.
- Spray disinfection and soaking are not permitted and may cause damage.
- The article can be neither machine-cleaned, nor disinfected at high temperature or sterilized. These processes lead to irreparable damage to the product.

### Technical specification

Mains supply:	100 – 240 V / 50 – 60 Hz
Current consumption:	160 – 80 mA
Fuse:	Integral overload protection
Charging current:	Optimised charging cycle CC/CV for mini 2Z NiMH, maximum charging current 0,3 A NiMH 2.5 V = approx. 4 h (680 mAh)
Charging time:	continuous
Operation:	II
Safety class:	IP 20
Protection class:	112 x 50 x 52 mm
Dimensions:	270 g
Weight:	+10 °C to +35 °C
Ambient temperature:	10 % to 75 %
Humidity:	700 hPa to 1060 hPa
Air pressure:	

### General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

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