HEINE C 2.3K Headband Loupes

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 11000 2021-03-15

V-200.00.031

DEUTSCH

HEINE C 2.3K Kopfbandlupe

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ENGLISH

HEINE C 2.3K Headband Loupes

Please read and follow these instructions for use and keep them for future reference.

Safety Information

Range of applications: The HEINE C 2.3K Headband Loupes are used for magnification during examination and operations.

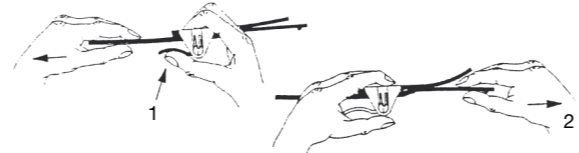
Safety in use: Do not look directly into the sun or at a powerful light source when using the Headband Loupes!
The product must only be operated by qualified medical personnel within a professional healthcare facility.

Adjustment

To obtain the best results you should adjust the instrument as follows:

Adjusting the headband

To adjust the size of the headband, press the lever (1) to increase and pull the free end (2) to decrease the circumference.

Adjusting headband height

The position of the optics is adjusted over a wide range by means of the hinged mount. The ease of movement of the hinges can be adjusted by means of the hinge screws.

Obtaining a clearly-focused single image

- Unscrew both adjuster screws just enough to let you slide the eyepieces in or out.

- Put on the headband and adjust the eyepieces to the correct height and look at a well-lit object at the working distance (a printed text is not suitable as object)

- Adjust the oculars sideways until you see a shadow-free round image which is sharply-focused.

- Lightly tighten both adjuster screws without changing the position of the optics.

- Take off the headband and check that the inter-pupillary distance shown on the scale is symmetrical. Correct this if necessary by adjusting both adjuster screws to give a symmetrical setting.

- Re-tighten the adjuster screws.

Hygienic reprocessing

Protection, Maintenance and Cleaning/Disinfection

The optics of the loupes can be easily damaged and should be protected against scratching, usually caused by dust or debris.

Please attach the detachable protective lenses when using the loupes.

When not in use, store the loupes in the protective case provided.

Cleaning

Before cleaning, remove the protective lenses.

For cleaning or removing fingerprints from the optics use a micro-fiber cloth. The housing can be cleaned by wiping with a damp cloth. Use pure water or a pH-neutral or mild soapy/alkaline solution.

Caution: Do not rinse the loupes under running water. Do not immerse the loupes in liquids. Avoid penetration of liquids into the optics.

Caution: Hard and/or metallic brushes can cause scratching and damage to the optics.

Caution: Do not use any abrasive cleaners or aggressive detergents for cleaning.

Disinfection

Only disinfection by wiping with a damp cloth is recommended.

For disinfection use only detergents which are recommended for use on medical devices made of plastic and glass. Isopropyl alcohol should only be used at a concentration below 70 %.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

General Warnings

- Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.
- Do not look into the sun or into a powerful light source with this loupe.
- This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Technical specification

Magnification*)	2,3 x
Working distance*)	340/450 mm
Inter-Pupillary distance	50 – 74 mm

*) If your refractive error is not corrected, these values may be slightly different. Generally, the working distance is shorter for myopes and longer for hyperopes.

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