Instruction for use
HEINE Charger 30

Instruction for use

HEINE Charger 30

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
DANSK	24
NORSK	27
SUOMI	30
PORTUGUÊS	33

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HEINE Charger 30

 Please read and follow these instructions for use of and keep them for future reference.

Intended Use

The HEINE Charger 30 is intended only for charging the HEINE DELTA 30 dermatoscope.
The product may only be used by qualified medical professionals and in professional healthcare facilities.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

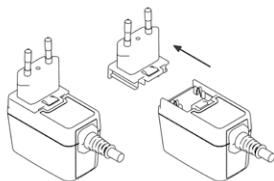
Product overview

1 Charging indicator

Setting up

The primary country adapter of the power supply is interchangeable, this enables worldwide operation. The following picture describes the procedure of changing the plug.

Primary plug:

EU

USA/Japan

UK

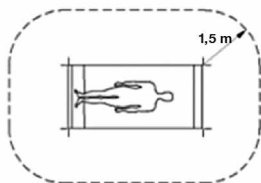
Australia

 After you checked the mains voltage indicated on the nameplate of the device, connect the mains supply plug to the mains supply.

Mount the power supply in such a position where it can be easily unplugged.

To put the power supply out of operation just disconnect the power supply from the mains supply.

The charger is intended for installation in medical areas outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see figure).

The above figure defines the patient environment (minimum radius around the patient).

Operation

For charging the DELTA 30 dermatoscope place it onto the charger as pictured in the product overview. Make sure that there are no objects in the charging cradle.

The blue illuminated charging display on the charger indicates that the DELTA 30 is being charged. The charge level is indicated by the charging display on the DELTA 30 dermatoscope.

If the charging display on the charger or on the DELTA 30 does not light up after placing the instrument in the charger, please check whether the device is plugged into the mains and the instrument is correctly inserted into the charger. Reset the charging process by removing the instrument from the charger for a few seconds and then replacing it. If the situation persists, contact the manufacturer.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.
 - Do not place metallic objects (e.g. paper clips, coins) in the charging cradle.
 - Do not use the device in fire or explosive risk area (e.g. oxygen saturated or anaesthetic environments).
 - This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.
 - Do not modify the device.
 - Use only original HEINE parts, spare parts, accessories and power sources.
 - Repairs shall only be carried out by qualified persons.
 - Do not use the power supply outdoors.
 - Heating during operation is normal and harmless. Make sure that the device isn't covered during operation, that it isn't placed in close proximity to a heating and that isn't exposed to direct sunlight.

Disposal

-  The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the HEINE Charger 30, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed
Voltage Fluctuations/Flicker	Passed

Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Nominal input voltage	100 – 240 V AC / 50 – 60 Hz
Power consumption	< 15 W
Nominal output voltage	5.9 V
Nominal output current	max. 2 A
Class	II
Protection class	IP40

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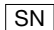
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
Marque réglementaire de conformité (RCM)
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
The Regulatory Compliance Mark (RCM)
Vaatimustenmukaisuusmerkki (RCM)
(RCM)- Marca de Conformidade Regulamentar

UL Mark für Kanada und USA
UL Mark for Canada and the USA
Marque d'homologation UL pour le Canada et les États-Unis
Marca UL para Canadá y EE.UU.
UL Mark per il Canada e gli Stati Uniti d'America
UL Mark för Kanada och USA
UL Mark voor Canada en de VS
UL Mark for Canada og USA
UL Mark for Canada og USA
UL-merkintä Kanadaa ja Yhdysvaltoja varten
UL Mark para Canadá e EUA

Effizienzlevel VI (US)
(Hintergrund: blau, Vordergrund: weiß.)
Efficiency level (US)
(Background color: blue, foreground color: white.)
Niveau d'efficacité VI (États-Unis)
(Couleur de fond : bleu ; couleur du premier plan : blanc)
Estándar de eficiencia nivel VI (EE.UU.)
(Color de fondo: azul, color de primer plano: blanco.)
Livello di efficienza VI (Stati Uniti d'America)
(Colore dello sfondo: blu, colore in primo piano: bianco)
Effektivitetsnivå VI (US)
(Bakgrundsfärg: blå, förgrundsfärg: vit)
Efficiëntieniveau VI (VS)
(achtergrondkleur: blauw, voorgrondkleur: wit)
Energiklasse VI (US)
(Baggrundsfarve: Blå; forgrundsfarve: Hvid)
Effektivitetsnivå VI (US)
(Bakgrunnsfarge: blå, forgrunnsfarge: hvit.)
Tehokkuustaso VI (Yhdysvallat)
(Taustaväri: sininen, etualan väri: valkoinen)
Nível de eficiência VI (US)
(Cor de fundo: azul, cor de primeiro plano: branco)

UDI

Unique Device Identification

MD

Medical Device

 **HEINE Optotechnik GmbH & Co. KG**
Dornierstr. 6, 82205 Gilching, Germany
www.heine.com

V-200.00.157

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Trade mark of the manufacturer