## ⌘ HEINE EN50 UNPLUGGED HEINE mPack UNPLUGGED

# **HEINE EN50 UNPLUGGED**

## **HEINE mPack UNPLUGGED**

<b>DEUTSCH</b>	<b>3</b>
<b>ENGLISH</b>	<b>7</b>
<b>FRANÇAIS</b>	<b>11</b>
<b>ESPAÑOL</b>	<b>15</b>
<b>ITALIANO</b>	<b>19</b>
<b>SVENSKA</b>	<b>23</b>
<b>NEDERLANDS</b>	<b>27</b>
<b>DANSK</b>	<b>31</b>
<b>SUOMI</b>	<b>35</b>
<b>PORTUGUÊS</b>	<b>39</b>

# HEINE EN 50 UNPLUGGED

## HEINE mPack UNPLUGGED

DEUTSCH

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# HEINE EN50 UNPLUGGED

## HEINE mPack UNPLUGGED

 Please read and follow these instructions for use of and keep them for future reference.

### Intended Use

The HEINE mPack UNPLUGGED battery is intended exclusively for supplying power to HEINE instruments on headbands specially designed for the purpose. There are two chargers available for charging the Lithium-polymer battery inside the HEINE mPack UNPLUGGED:

- a) the HEINE EN50 UNPLUGGED, with two charging cradles, is intended for wall mounting.
- b) the HEINE UNPLUGGED plug-in transformer is intended as a charger for mobile operation.

The product may only be used by qualified medical professionals and in professional healthcare facilities.

### For U.S. only:

 Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

### Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

### Product overview


#### HEINE mPack UNPLUGGED

Connector for the instrument or an UNPLUGGED power unit for mobile charging

Capacity indicator

Serial-No.

Charging contacts for charging via the HEINE EN50 UNPLUGGED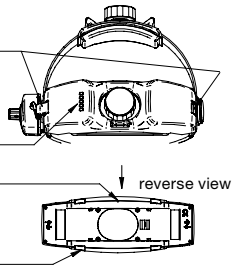


#### HEINE EN 50 UNPLUGGED


Charging cradle II

Contact pins

Charging cradle I


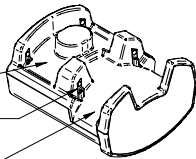
#### HEINE UNPLUGGED power unit (plug-in transformer)
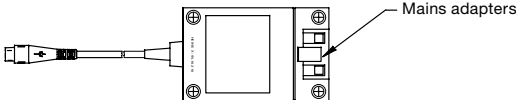
### Setting up

#### Mains adapters

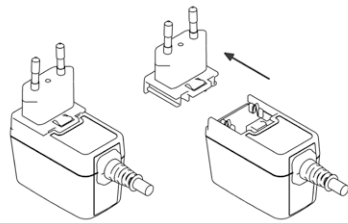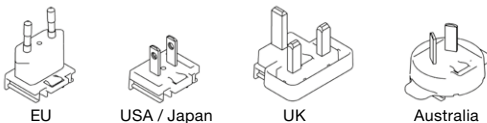The mains adapter is interchangeable. This allows a world wide use. The procedure of changing the plugs is as per the pictures.

 Take care to mount the charging station outside the patient environment. There must be adhered to the distances (1.5m) in accordance with IEC60601-1 (see figure).


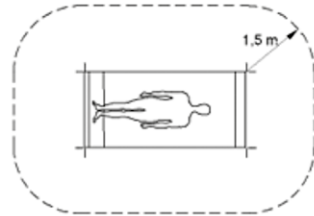
Fig. 1: Minimum dimensions of the patient environment

 The device should never be operated or even stored at places listed below, because this could lead to operating failures:

- a. which are heavily exposed to moisture or where water condensing may occur
- b. which are exposed to special environmental conditions
- c. which are subject to constant vibrations
- d. which are subject to high temperature fluctuations
- e. outdoors

#### Wall mounting the HEINE EN50 UNPLUGGED


 Use the four provided screws to fix the mounting bracket to the wall. Place the EN50 UNPLUGGED on top of the mounting racket and fasten with the two top screws.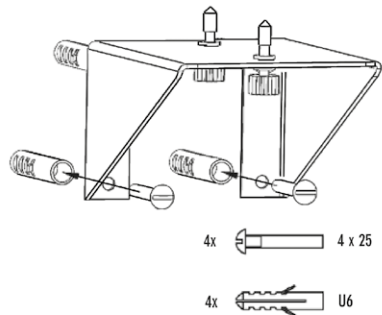


## Setting up the HEINE mPack UNPLUGGED

### Fitting the mPack UNPLUGGED to the instrument headband

First remove the rear segment cover of the headband. To do this, hold the headband in the region of the rear segment cover with your left hand and pull the segment cover to the right with your right thumb.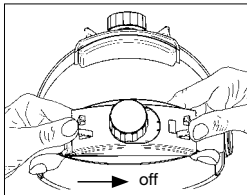

Then fit the mPack UNPLUGGED to the rear part of the headband by supporting the headband with your left hand and placing the mPack UNPLUGGED on the headband so that the LEDs are arranged to the left. The mPack UNPLUGGED will engage securely with a little light pressure from your left thumb. Connect the brightness control to the mPack UNPLUGGED.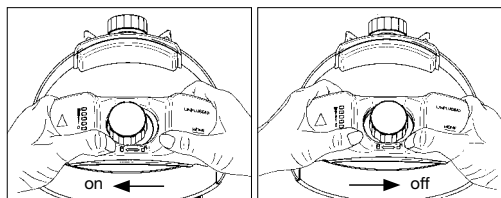

The batteries must be charged before using the mPack UNPLUGGED for the first time.

- You can charge the mPack UNPLUGGED either from the UNPLUGGED plug-in transformer or from the EN50 UNPLUGGED charging station.

When supplied, new batteries are only partially-charged. The mPack UNPLUGGED must be charged before the instrument is used for the first time. The LED indicator corresponds to the actual battery capacity only after a full charge and discharge cycle. (Also applies when changing the battery.)

- To enable the LED display of the mPack UNPLUGGED to show the true battery capacity, the unit must first be completely discharged and then recharged. Switch on the instrument connected to the mPack UNPLUGGED until the battery is completely discharged, then recharge the mPack UNPLUGGED until charging is completed. (Also applies when changing the battery.)

## Operation

### Charging via the HEINE EN50 UNPLUGGED (wall-mounted charger)

Connect the plug-in transformer to a suitable mains outlet (pay attention to the mains supply voltage). The green LED on the plug-in transformer will light. The mPack UNPLUGGED can be charged in charging cradle I or charging cradle II. Using gentle pressure, insert the mPack UNPLUGGED into the charging cradle until it engages securely. Charging cradle I is intended for parking the complete instrument and for charging the mPack UNPLUGGED. The instrument is automatically switched off as soon as the complete instrument is parked in charging cradle I. The mPack UNPLUGGED will engage in the charging cradle with light pressure, thus ensuring safe charging. After approx. 1 second, the LEDs start to light in sequence. During charging, the LEDs blink. (See the "LED display" chapter for further information.) When you remove the instrument from the charging station and switch it on with the brightness control, all five LEDs will light if the battery is fully charged. As the level of charge falls, the LEDs go out in turn. If the orange LED starts to flash, you should recharge the battery. You can park the instrument in the charging

station at any time, even after short periods of operation. To increase the working life of the battery, recharging only occurs if the residual charge is less than 75%.

### Charging by means of the UNPLUGGED power unit

- Please note that charging with the UNPLUGGED mains transformer is only possible when the brightness control is switched off.

Connect the plug-in transformer to a suitable mains outlet (pay attention to the mains supply voltage). The green LED on the plug-in transformer will light. Next connect the cable to the right or left connector on the mPack UNPLUGGED. The charging process begins. All LEDs will go out as soon as the charging process is complete. Disconnect the charger. You can now switch the instrument on by means of the brightness control.

### The following applies to either charging method

The charging process is dependent on the residual charge of the battery and requires a maximum of approximately 2 hours. The charging process switches off automatically when the battery is full, all LEDs go out. Overcharging of the battery is not possible.

### Direct operation

Direct operation is possible if there is no capacity in the mPack UNPLUGGED battery but you want to continue to work with the instrument. To do this, simply connect the UNPLUGGED plug-in transformer to the connector on the left or right hand side and switch the instrument on with the brightness control.

- The battery will not be charged at the same time during "direct operation".

## LED display

### Charging

During charging the LEDs operate sequentially, in use they remain switched on. If all LEDs are out, the charging process is complete and the battery is fully charged.

### Operating the mPack UNPLUGGED

The number of LEDs illuminated indicates the level of charge in the battery. The full operating period is achieved when at least 4 LEDs are lit. If LED is flashing, you should charge the mPack UNPLUGGED. If all the LEDs are out, the battery is fully discharged.

## Hygienic reprocessing

The instruction is available:

- at [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

## Maintenance

EN50 UNPLUGGED and power supply does not require regular maintenance.

- ⚠ The power supply must not be opened. (Risk of electrical shock).  
Li-Polymer batteries can be replaced in mPack UNPLUGGED, please read the chapter "changing the battery".

### Changing the battery

In the course of time, batteries gradually wear out and require longer charging times; this is normal. If you charge the batteries regularly and notice a shortening of the operating time or an extension of the charging time, it is probably time to replace them. The new Li-Pol battery can be inserted simply by opening the lid of the mPack UNPLUGGED.


- Use only original HEINE Li-Pol rechargeable batteries.

Isolate the mPack by removing it from the charger cradle or by unplugging the UNPLUGGED mains transformer. Unscrew the eight screws in the lid of the mPack UNPLUGGED with the screwdriver supplied and remove the lid, which contains the Li-Pol battery. Make sure that no metal objects such as screws fall into the unit. Protect the electronics from electrostatic discharge (ESD), by avoiding

contact with electronic components. Before replacing the lid, remove the protective cap from the three-pole battery contacts. Reattach the lid with the eight screws, making sure that no wires are damaged when doing so.

The battery should be disposed of correctly and should be completely discharged before disposal.

**⚠** Do not short-circuit the Li-Pol battery. This can occur when a metal object such as a coin or paper clip connects the two battery contacts together. This can damage the battery.

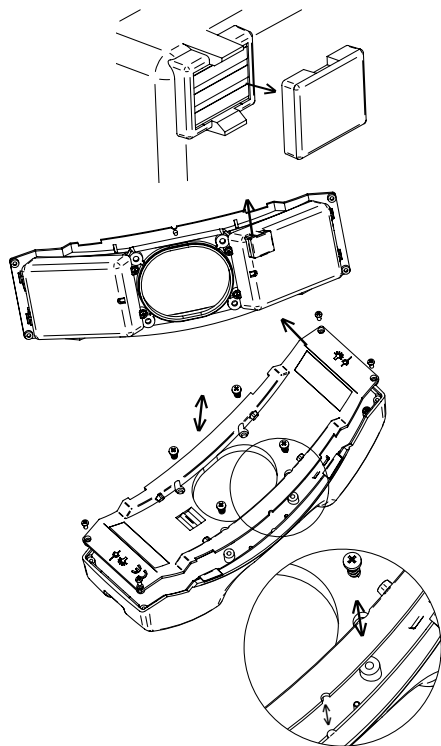Unplug the mains connector if you will not be using the unit for a long period if you are cleaning it or if a fault occurs. An unlit green power LED on the UNPLUGGED transformer and the EN50 UNPLUGGED does not guarantee a current free state.

Pay attention to the mounting instructions when wall-mounting the EN50 UNPLUGGED charging station.

The capacity of the mPack UNPLUGGED Battery can reduce with increasing use. This is a normal procedure with time and inevitable. Careful use of the battery can reduce this effect. We recommend that the instrument is use should be switched off after use and the battery recharged regardless of its state of charge.

The LED on the UNPLUGGED and EN50 UNPLUGGED transformers indicates the operational readiness of the power supply.

### General Warnings

**⚠** Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)

The device is not allowed to be entered into strong magnetic fields like MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

If the power source is transferred from a cold to a warm environment, let it rest for approximately 1 h to allow it to equilibrate to the ambient conditions.

The device should never be used or stored outdoors, since this may lead to malfunction.

### Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

 Dispose at your local collection point.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

### Service

The device has no components serviceable by the end-user.

### General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Use only an original HEINE plug-in transformer to connect the unit to the local mains supply. Make sure that the local mains voltage is the same as the nominal voltage shown on the identification plate of the HEINE plug-in transformer.

## What if...?

In the event of a fault, it is generally recommended that the mPack UNPLUGGED be connected briefly to the charger.

Fault description	LED display on mPack UNPLUGGED	Cause of problem	Remedy
Green Power LED on power unit does not light	-	- no mains supply voltage present - mains supply voltage outside permissible range	-
Instrument does not function	No LEDs on	Battery is discharged	Charge the mPack UNPLUGGED
		Electronics defect in the mPack UNPLUGGED or brightness controller	Refer to your service center. Check whether direct operation is still possible
		Ambient temperature too high	Do not expose the mPack UNPLUGGED to ambient temperatures greater than 40 °C
	LEDs on	Bulb in instrument is defective	Change the bulb
Fault on brightness controller		- turn the rotary knob on/off briefly - plug in the connecting cable again	
mPack UNPLUGGED does not charge	Fourth green LED does not light	Remaining capacity in the battery is greater than 75 %; frequent, brief recharging is to be avoided, to preserve the battery	-
	Even after a long charging time, only the orange LED is flashing	Due to a disturbance in the instrument, e.g. when changing the bulb, the LED display on the mPack UNPLUGGED no longer corresponds to the actual state of charge	If the instrument is functioning, fully discharge and recharge the battery once; the LED display will then be resynchronized
	- no mains voltage present - mains voltage does not correspond to the nominal mains voltage		-
Instrument does not function	LEDs blink	Defective bulb or bulb not correctly installed	Change the bulb or install correctly. Switch the instrument off and then on again

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<b>Electromagnetic disturbances – Requirements and tests</b>	
<p>The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.</p>	
Statement for the operational environments	<p>Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p>
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	<p>Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.</p>
	<p>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</p>
	<p>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the EN50 UNPLUGGED / mPack UNPLUGGED, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.</p>
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	See attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed
Voltage Fluctuations/Flicker	Passed

## Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
<b>HEINE EN50 UNPLUGGED</b> Mains voltage Current Output Fuse Duration of use Protection Class	100–240 V AC / 50–60 Hz 600–300 mA 9 VDC; 3.3 A Integrated overload protection Continuous II
<b>HEINE mPack UNPLUGGED</b> Input Output Fuse Weight Dimensions Cell type Nominal voltage Capacity Charging time	9 VDC; 1.5 A Variable, max. 6-8.4 V / < 15 W Integrated overload protection approx. 190 g 166 mm x 62 mm x 53 mm Lithium-polymer 7.4 V approx. 2000 mAh approx. 2 hours

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