**HEINE** QUALITY  
MADE IN GERMANY

∴ **HEINE EN50**  
**HEINE mPack**

 **HEINE Optotechnik GmbH & Co. KG**  
Dornierstr. 6 · 82205 Gilching · Germany  
E-Mail: [info@heine.com](mailto:info@heine.com) · [www.heine.com](http://www.heine.com)  
MED 1814 2021-04-10


# HEINE EN50

## HEINE mPack

DEUTSCH	4
ENGLISH	8
FRANÇAIS	12
ESPAÑOL	16
ITALIANO	20
SVENSKA	24
NEDERLANDS	28
DANSK	32
SUOMI	36
PORTUGUÊS	40

Produkt-Übersicht  
Product overview  
Vue d'ensemble du produit  
Partes del producto  
Informazioni generali sul prodotto  
Produktöversikt  
Overzicht van de producten  
Produktoversigt  
Tuotteen osat  
Visão geral do produto

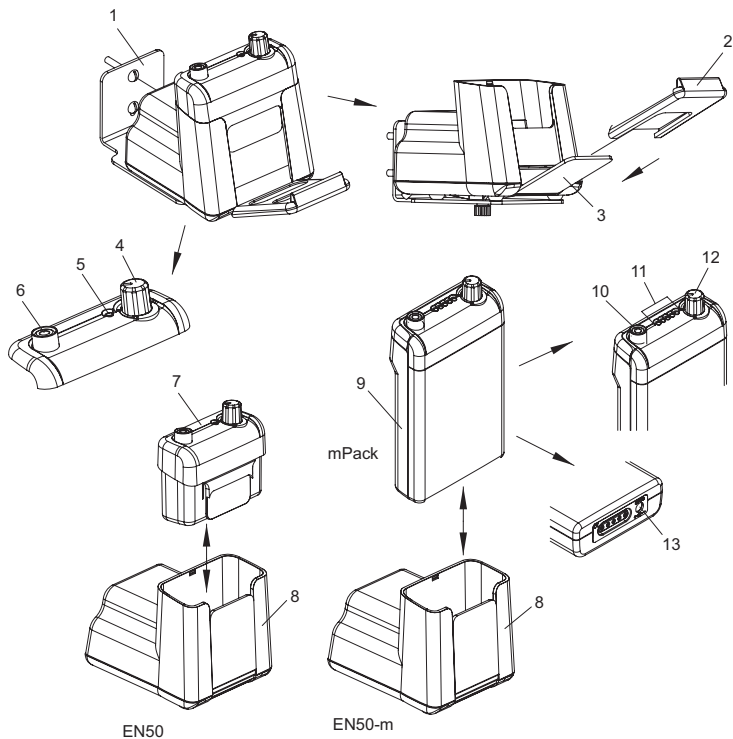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 Please read and follow these instructions for use and keep them for future reference.

### Intended Use

The HEINE EN50 Wall/Table transformer (HEINE EN50 and HEINE mPack) are designed solely for use with HEINE instruments (6V).

The product must only be used by qualified medical professionals and in professional healthcare facilities.


 **For U.S. only:**  
**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

### Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

### Installation EN50 Wall/Table Transformer

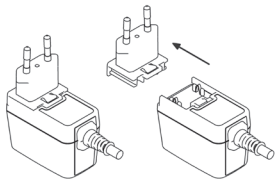#### Primary plug:

EU

USA/Japan

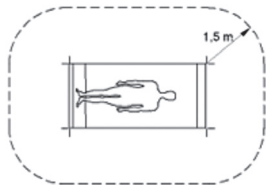
UK

Australia

The primary plug is interchangeable. This allows a world wide use. The procedure of changing the plugs is as per the pictures.

 The HEINE EN50/mPack is intended for installation in medical areas outside the patient environment (at least 1.5 metres from the patient or patient support pursuant to IEC 60601-1, see figure).


**⚠** To prevent mechanical and electrical breakdown, never install or store the charger in the following areas:

- Areas exposed to high moisture or condensation
- Areas exposed to extreme environmental conditions
- Areas subject to constant vibrations
- Areas subject to severe temperature fluctuations
- Outdoors

### Wall-mounting of the EN50

**•** Mount the bracket to the wall using the four delivered screws. Fix the EN50 with the screw on the bracket.

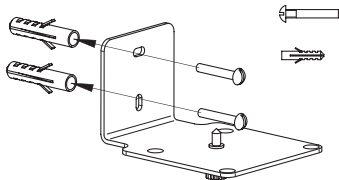Use the wall-mounting bracket (1) to fix the EN50 to the wall. Fit the instrument support (2) on to the actuator (3). The instrument light will switch off when the headband is placed onto the instrument support (2). When the headband is taken off the instrument support (2), the EN50 automatically switches the instrument light on again.

### Setting up and operation

#### HEINE EN50 operation

Connect the instrument to the cinch connector (6) and select the required brightness by means of the brightness control (4). The EN50 switches off with an audible click (4). The indicator (5) lights up as soon as the EN50 is switched on either by the control (4) or-if fitted- the headband brightness control. It switches off when the EN50 is switched off or the actuator (3) is pushed down.

**•** If the headband control HC50L is used with the EN50, the control (4) on the EN50 has no effect. The brightness is controlled in this case by adjusting the control on the headband control HC50L.

We recommend that the mPack brightness control is switched off when used with instruments fitted with a headband brightness control.

The EN50 may only be operated in combination with the control unit (7).

#### HEINE EN50-m and HEINE mPack operation

The EN50 power source can be transformed into an EN50-m by adding the mPack. This equips the instrument for both mobile and static use. The EN50 is modified with the mPack in the following way: Remove the control unit (7) of the EN50 from the basic unit (8) by pulling it upwards. Insert the mPack (9) into the opening in the basic unit (8). Plug in the instrument to the socket (10) of the mPack. If the mPack is used with the basic unit (8) (EN50-m), the following functions are identical to those of the EN50: on/off (audible click), brightness control, operation with the headband control, on/off switching by the actuator (3) when wall-mounted. Charge the mPack for two hours before use. Charge the mPack either by means of the socket (13) with the HEINE external plug-in transformer or insert the mPack into the opening in the basic unit (8). The mPack is automatically recharged even when switched off or when no instrument is connected.

**•** When the EN50 is used in combination with the HC 50L to power an LED Instrument (eg. OMEGA 500, ML4), and the LED is powered with less than 50 % of the maximum power, the green operating light on the EN50 will blink intermittently indicating low-load operation of the attached instrument.

### LED indicators (11) on the HEINE mPack

During charging the LED's operate sequentially, otherwise they remain switched on (→ stand-by). If the LEDs (11) remain switched off, the unit is non-operational or the mPack is switched off. The number of illuminated or sequentially-switched LEDs gives an indication of the state of charge of the Li-ion battery. All five LEDs illuminated indicate a fully-charged battery. If the yellow LED flashes on and off, the battery must be recharged. Switch off the mPack and recharge it by means of the HEINE mains transformer or the EN50 basic unit. The HEINE mPack clip is used to clip the HEINE mPack to a belt or trouser pocket etc.

### Hygienic reprocessing

The instruction is available:

- at [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

### Maintenance

#### Changing the battery

The Li-ion mPack only needs replacement when the battery no longer recharges to give sufficient capacity. This usually occurs after 3 to 5 years if properly used.

#### Procedure

Disconnect the plug-in transformer of the mPack from the mains or remove the mPack from the basic unit (8) of the EN50. Unscrew the lid screw on the back of the mPack and remove the lid. Remove the old battery and disconnect the cord. No tools are necessary. Insert the new Li-ion mPack battery into the mPack so that the label is visible and the battery cord is to the front control (4). Connect the plug of the battery to the printed circuit board. Take care when closing the lid that the cord is not pinched between lid and body. To close, locate the two pins on the lid in the cut-outs in the lower half of the housing. Close the lid and tighten the screw. Charge the new Li-ion mPack battery for two hours before use.

 The HEINE Li-ion mPack battery must be properly disposed.

### Service

The device has no components serviceable by the end-user.

### Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe  the relevant state-specific disposal regulations.

### General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

### General Warnings

 Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

We can only guarantee the performance of this instrument when fitted and used with original HEINE accessories and spare parts.

Follow the instructions supplied with the unit regarding installation of the instrument support and the wall-mounting bracket.

For wall-mounting, be sure to use the instrument support (2) supplied with the unit to be sure that the instrument headband is located safely.

Make sure that the local mains voltage is the same as that shown on the identification plate of the plug-in transformer.

In order to ensure correct operation, always connect the angled adaptor first to the connecting or extension cord and only then to the power source.

Use only the appropriate HEINE mPack plug-in transformer to connect the unit to the local mains supply.

If not used for long periods, unplug the mains transformer from the mains.

When moving from a cold to a warm environment, leave the power source switched off for up to one hour to allow it to adapt to the new conditions.

The device must not be installed or stored outdoors to prevent operational failures.

### **Safety Information**

 Do not expose the Li-ion mPack battery to ambient temperatures above +45 °C.

All repairs should be carried out by qualified personnel, with the exception of a battery change with mPack. When changing the battery, use only the original rechargeable battery HEINE Li-ion mPack.

Do not use the device in rooms where an explosion hazard is present.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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
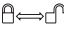
## Technical specification

<i>HEINE EN50</i> Mains voltage Safety class Current consumption Output Fuse Classification Length of cord Weight Dimensions	100–240 VAC / 50–60 Hz II max. 30 VA max. 9 V integrated overload protection Type B 200 cm approx. 905 g 85 mm x 87 mm x 95 mm
<i>HEINE mPack</i> Output Fuse Charging current limit Weight Dimensions	Variable, max. 9 V / 15 W integrated overload protection 3.7 A approx. 300 g 80 mm x 28 mm x 125 mm
<i>Ambient conditions</i> In use Storage Transport Relative humidity Air pressure	+10 °C to +35 °C +5 °C to +45 °C -20 °C to +45 °C 10 % to 75 % 700 hPa to 1060 hPa

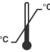
**Erläuterung der verwendeten Symbole**  
**Explanation of utilized symbols**  
**Explication des symboles utilisés**  
**Explicación de los símbolos utilizados**  
**Spiegazione dei simboli utilizzati**  
**Förklaring av symboler som används**  
**Verklaring van de gebruikte symbolen**  
**Forklaring af de anvendte symboler**  
**Käytettyjen symbolien selitys**  
**Explicação dos símbolos utilizados**

<p><b>CE</b></p>	<p>CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizinprodukterichtlinie 93/42 EWG.          The CE mark indicates that the product complies with the European medical device directive 93/42/EEC.          Le marquage CE indique la conformité à la directive européenne 93/42/CEE relative aux dispositifs médicaux.          El marcado CE indique la conformidad con la directiva europea 93/42 /CEE relativa a los productos sanitarios.          Il marchio CE indica la conformità con la direttiva europea sui dispositivi medici 93/42 CEE.          CE-märkning markerar en överensstämmelse med det europeiska direktivet för medicinska produkter 93/42 EEG.          CE-markering duidt de overeenstemming aan met de Europese Richtlijn betreffende medische hulpmiddelen 93/42 EEG.          CE-mærkningen angiver overensstemmelse med det europæiske direktiv 93/42/EØF om medicinsk udstyr.          CE-merkintä tarkoittaa, että laite vastaa eurooppalaisen lääkinnällisiä laitteita koskevan standardin 93/42 ETY vaatimuksia.          O símbolo CE identifica a concordância com a Diretriz Europeia para Dispositivos Médicos 93/42/CEE.</p>
<p><b>REF</b></p>	<p>Katalog- oder Bestellnummer          Catalogue- or order number          Numéro de catalogue ou de commande          Número de catálogo o de pedido          Codice catalogo e di dell'ordine numero          Katalog- eller Beställningsnummer          Catalogus- of Bestelnummer          Katalog- eller Ordrenummer          Luettelo- tai viitenumero          Número de catálogo ou do pedido</p>
<p><b>SN</b></p>	<p>Serienummer          Serial number          Numéro de série          Número de serie          Numero di serie          Seriennummer          Seriennummer          Seriennummer          Sarjanumero          Número de série</p>

	<p>Hersteller          Manufacturer          Fabricant          Fabricante          Produttore          Tillverkare          Fabrikant          Producent          Valmistaja          Fabricante</p>
	<p>Hersteldatum          Date of manufacture          Date de fabrication          Fecha de fabricación          Data di produzione          Tillverkningsdatum          Productiedatum          Produktionsdato          Valmistuspäivä          Data de fabricação</p>
	<p>Getrennte Sammlung von Elektro- und Elektronikgeräten          (Europäische WEEE Richtlinie)          Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions.          (European Waste Electrical and Electronic Equipment Directive, WEEE)          Tri sélectif des appareils électriques et électroniques          (Directive européenne DEEE)          Desechado separado de aparatos eléctricos y electrónicos          Directiva Europea RAEE)          Raccolta differenziata di apparecchi elettrici ed elettronici          (direttiva europea RAEE)          Separat insamling av elektriska och elektroniska apparater          (det europeiska WEEE-direktivet)          Gescheiden inzameling van elektrische en elektronische apparaten          (Europese AEEA richtlijn).          Separat indsamling af elektrisk og elektronisk udstyr          (det europæiske WEEE-direktiv)          Sähkö- ja elektroniikkalaitteille tarkoitettu erillinen keräyspiste          (eurooppalainen WEEE-standardi)          Coleção separada de aparelhos elétricos e eletrônicos          (Diretrizes Europeias WEEE)</p>
	<p>Batterien müssen einer zentralen Sammelstelle zugeführt werden.          Dispose at your local collection point.          Les batteries doivent être rapportées à un point de collecte central.          Desechar en un punto limpio de la comunidad.          Smaltimento in un centro di raccolta comunale.          Avfallshanteras på ett kommunalt insamlingsställe.          Batterijen moeten bij een recyclinginzamelpunt worden ingeleverd.          Bortskaffelse på et kommunalt indsamlingssted.          Laite on hävitettävä paikalliseen keräyspisteeseen.          Proceda à eliminação do aparelho em um ponto de coleta municipal.</p>

	<p>Zulässiger Temperaturbereich in °C für Lagerung und Transport  Temperature limits in °C for storage and transport  Plage de température admise en °C pour le stockage et le transport  Rango de temperatura permitida en °C para almacenar y transportar el producto  Temperatura ammessa in °C per conservazione e trasporto  Tillåtet temperaturintervall i °C för lagring och transport  Toegestane temperaturen in °C voor opslag en transport  Tilladt temperaturområde i °C ved opbevaring og transport  Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°C)  Gama de temperatura permitida em °C para o armazenamento e transport</p>
	<p>Zulässiger Temperaturbereich in °F für Lagerung und Transport  Temperature limits in °F for storage and transport  Plage de température admise en °F pour le stockage et le transport  Rango de temperatura permitida en °F para almacenar y transportar el producto  Temperatura ammessa in °F per conservazione e trasporto  Tillåtet temperaturintervall i °F för lagring och transport  Toegestane temperaturen in °F voor opslag en transport  Tilladt temperaturområde i °F ved opbevaring og transport  Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°F)  Gama de temperatura permitida em °F para o armazenamento e transporte</p>
	<p>Zulässige Luftfeuchtigkeit für Lagerung und Transport  Humidity limitation for storage and transport  Humidité admise pour le stockage et le transport  Humedad del aire permitida para almacenar y transportar el producto  Umidità atmosferica ammessa durante il trasporto e la conservazione  Tillåten luftfuktighet för lagring och lagring  Toegestane luchtvochtigheid voor opslag en transport  Tilladt luftfugtighed ved opbevaring og transport  Sallittu ilmankosteus kuljetuksen ja varastoinnin aikana  Umidade do ar admissível para o armazenamento e transporte</p>
	<p>Zulässiger Luftdruck für Lagerung und Transport  Pressure limitation for storage and transport  Pression atmosphérique admise pendant le transport et le stockage  Presión de aire permitida para almacenar y transportar el producto  Pressione atmosferica ammessa durante il trasporto e la conservazione  Tillåten lufttryck för lagring och transport  Toegestane luchtdruk voor opslag en transport  Tilladt lufttryk ved opbevaring og transport  Sallittu ilmanpaine kuljetuksen ja varastoinnin aikana  Pressão do ar admissível para o armazenamento e transporte</p>
	<p>Verriegelung  Lock mechanism  Verrouillage/Déverrouillage  Bloqueo/Desbloqueo  Bloccaggio/sbloccaggio  Låsning/upplåsning  Aan-/onthoppelen  Låsning/opplåsning  Lukitse/Vapauta  Trava/destrava</p>

	<p>Netzteilananschluss (9VDC)  Power supply connector (9VDC)  Connecteur-adaptateur – Alimentation en tension (9VDC)  Conector – Alimentación de tensión (9VDC)  Spina di collegamento – tensione di alimentazione (9VDC)  Anslutningskontakt – spänningsförsörjning (9VDC)  Stekker-aansluiting (9 V DC)  Netdeltiislutning (9VDC)  Liitäntäpiste – jännitteen syöttö (9 V DC)  Conector de energia (9VDC)</p>
	<p>Instrumentenanschluss  Instrument connector  Connecteur-adaptateur – Instrument  Conector – Instrumento  Spina di collegamento strumento  Anslutningskontakt – instrument  Stekkerverbinding – instrument  Instrumenttilslutning  Liitäntäpiste – instrumentti  Conector de instrumento</p>
	<p>Gleichspannung DC  Direct current DC  Tension continue  Tensión continua (CC)  Tensione continua  Likspänning  Geljikspanning  Jævnspænding DC  Tasajännite  Tensão contínua (CC)</p>
	<p>Trocken lagern!  Keep dry!  Conserver au sec !  Conservar en un lugar seco!  Evitare ambienti umidi!  Förvaras torrt!  Droog bewaren!  Opbevares tørt!  Säilytetään kuivassa paikassa!  Armazenar em ambiente seco!</p>
	<p>Gebrauchsanweisung verbindlich befolgen.  (Hintergrundfarbe: blau, Vordergrundfarbe: weiß)  Follow instructions for use!  (Background color: blue, foreground color: white.)  Suivre le mode d'emploi.  (Couleur de fond : bleu ; couleur du premier plan : blanc)  Seguir obligatoriamente las instrucciones de uso.  (Color de fondo: azul, color de primer plano: blanco)  Attenersi obbligatoriamente alle istruzioni per l'uso.  (Colore dello sfondo: blu, colore in primo piano: bianco)  Bruksanvisningen ska alltid följas.  (Bakgrundsfärg: blå, förgrundsfärg: vit)  De gebruiksaanwijzing is bindend en dient gevolgd te worden.  (achtergrondkleur: blauw, voorgrondkleur: wit)  Følg altid brugsanvisningen.  (Baggrundsfarve: Blå; forgrundsfarve: Hvid)  Käyttöohjetta on noudatettava tarkasti.  (Taustaväri: sininen, etualan väri: valkoinen)  Leia e cumpra as instruções de uso e guarde-as para consulta futura.  (fundo: azul; primeiro plano: branco)</p>

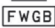	<p>Genehmigungszeichen von SIQ Approval mark of SIQ Marque d'homologation SIQ Sello de autorización de SIQ Marchio di conformità SIQ Godkännandesymbol från SIQ Toestemmingsteken van SIQ Godkendelsesmærke fra SIQ SIQ-laitoksen hyväksymismerkintä Símbolo de aprovação da SIQ</p>
	<p>Bedienungsanleitung lesen Instructions for use Mode d'emploi Manual de instrucciones Istruzioni per l'uso Bruksanvisning Gebruiksaanwijzing Brugsanvisning Käyttöohjeet Instruções de utilização</p>
	<p>Markenzeichen des Herstellers Trade mark of the manufacturer Sigle du fabricant Marca del fabricante Marchio del produttore Tillverkarens märkessymbol Merkteken van de fabricant Producentens varemærke Valmistajan tavaramerkki Símbolo da marca do fabricante</p>
<p>ta40/F</p>	<p>Bemessungs-Umgebungstemperatur / Wandler Temperaturklasse Rated ambient temperature / converter temperature class Température ambiante nominale / classe de température du transformateur Temperatura ambiente de medición / transformador de clase de temperatura Temperatura ambiente di taratura / trasformatore classe Nominell omgivningstemperatur / omvandlare temperaturklass Vaststelling omgevingstemperatuur / omzetter temperatuurklasse Dimensionerings-omgivelsestemperatur / transformer temperaturklasse Mittausympäristön lämpötila / muuntajan Lämpötilaluokka Temperatura de dimensionamento e ambiente / conversor classe de temperatura</p>
	<p>Gerät der Schutzklasse II Class II equipment Appareil de classe de protection II Aparato de clase de protección II Apparecchio di classe di protezione II Apparat i skyddsklass II Apparaat van beschermingsklasse II Apparater i sikkerhedsklasse II Suojausluokan II laite Aparelho da classe de proteção II</p>

<p>IPxx</p>	<p>IP-Code (Schutzart)  IP code (Ingress Protection Rating)  Indice IP  Índice IP  Cifre di identificazione IP  IP-siffrå  IP-kencijfers  IP-kode (beskyttelsestype)  IP-numerot  Números de identificação IP</p>
<p> V xxxx Q xxxx</p>	<p>Genehmigungszeichen für Australien  Approval mark of Australia  Marque d'homologation pour l'Australie  Licencia para Australia.  Marchio di conformità per Australia  Godkännandetecken för Australien  Toestemmingsteken voor Australië  Godkendelsesmærke for Australien  Australian hyväksymismerkintä  Símbolo de aprovação para Austrália</p>
<p><b>UDI</b></p>	<p>Unique Device Identification</p>
<p><b>MD</b></p>	<p>Medical Device</p>