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HEINE EasyClean LED

laryngoscope handles

Please read and follow these instructions for use of and keep them for future reference.

Intended Use

The internally powered HEINE laryngoscope handle is intended for the transient use with laryngoscope blades according to the "green standard" ISO 7376 for tracheal intubation. It should only be operated by qualified medical personnel within a professional healthcare facility or emergency medical services environment.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

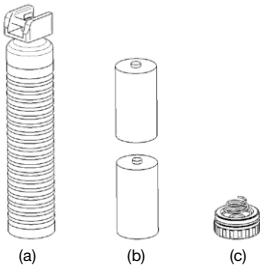
NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

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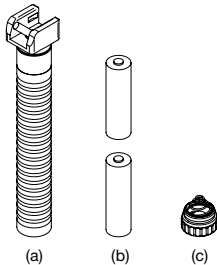
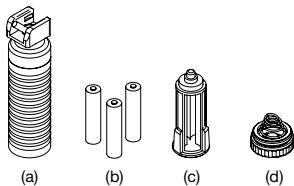
Setting up

EasyClean LED | EasyClean LED SLIM

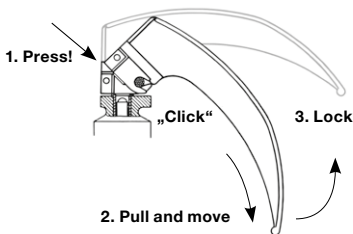
Insert the batteries (b) into the handle (a) leading with the positive terminal. Screw the bottom insert (c) into handle (a).

EasyClean LED SHORT

Insert the batteries (b), regarding the markings on the plastic, into the casing (c). Insert the casing with the contact first into the handle (a) and screw on the bottom insert (d).

Operation

Fix the blade into the working position (see fig.).

Before starting a clinical application, the state of the internal electrical power source must be tested by turning on the light source.

⚠ Only use laryngoscope blades which comply with the "green standard" ISO 7376 in conjunction with HEINE Laryngoscope handles.

The HEINE laryngoscope handle corresponds to the "green standard" according to ISO 7376. If non-HEINE laryngoscope blades are used, component compatibility and functionality are to be checked before each use!

Only trained personnel are allowed to use a laryngoscope for intubation purposes.

HEINE laryngoscopes are designed for use < 5 minutes.

Check the correct operation of the device before each use! Do not use the device if you notice any moisture in the device or damage to the device. (e.g. rough surfaces, edges, damaged O-rings etc).

Before each use, check the condition of the connector for damages, wear and tear. If defects exist, do not use the laryngoscope handle and replace it with one that is in working order.

We strongly recommend a functional backup of laryngoscopes in an emergency situation.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

Routinely inspect the condition of the batteries and replace them if necessary.

Changing the batteries

Unscrew the bottom insert from the handle and replace the batteries.

In order to ensure a secure electrical connection, use only batteries from renowned manufacturers:

EasyClean LED: LR14 (Size C)

EasyClean LED SLIM: LR6 (Size AA)

EasyClean LED SHORT: LR03 (Size AAA)

Check the correct function of the device after each battery change.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety

information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

If you don't use the device for a longer period of time please remove the batteries in advance.

General Warnings

 Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not shine directly in the eyes to avoid dazzle, especially if the pupils are dilated. Do not use the laryngoscope for eye reflex testing.

Do not use the device for transillumination of tissue.

Disposal

Perform a hygienic reprocessing before disposal.

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the laryngoscope handle, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Not applicable
Voltage Fluctuations Flicker	Not applicable

Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 80 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage (without batteries)	-40 °C to +70 °C up to 95 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport (without batteries)	-40 °C to +70 °C up to 95 % rel. humidity 500 hPa to 1060 hPa
Nominal voltage	EasyClean LED: 2 x Battery 1.5 V (Size C / LR14) EasyClean LED SLIM: 2 x Battery 1.5 V (Size AA / LR6) EasyClean LED SHORT: 3 x Battery 1.5 V (Size AAA / LR03)
Nominal current	max. 260 mA
Class	Internally powered
Classification according to IEC 62471	Group 2
Protection class	IP68

Erläuterung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati
Förklaring av symboler som används
Verklaring van de gebruikte symbolen
Forklaring af de anvendte symboler
Symbolforklaring
Käytettyjen symbolien selitys
Explicação dos símbolos utilizados

CE

CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizinprodukteverordnung (EU) 2017/745.
 The CE mark indicates that the product complies with the medical device regulation (EU) 2017/745.
 Le marquage CE indique que le produit est conforme ou au règlement (UE) 2017/745 relatif aux dispositifs médicaux.
 La marca CE indica que el producto cumple el Reglamento europeo sobre productos sanitarios (UE) 2017/745.
 Il marchio CE indica la conformità con il regolamento sui dispositivi medici (UE) 2017/745.
 CE-märkning markerar en överensstämmelse med förordningen om medicinska produkter (EU) 2017/745.
 CE-markering duidt de overeenstemming aan met de verordening betreffende medische hulpmiddelen (EU) 2017/745.
 CE-mærkningen angiver overensstemmelse med forordningen om medicinsk udstyr (EU) 2017/745.
 CE-merket angir at produktet er i samsvar med forskriften om medisinsk utstyr (EU) 2017/745.
 CE-merkintä tarkoittaa, että laite lääkinnällisiä laitteita koskevan asetuksen (EU) 2017/745 kanssa.
 O símbolo CE identifica a concordância com o regulamento de Dispositivos Médicos (EU) 2017/74.

REF

Katalog- oder Bestellnummer
 Catalogue- or order number
 Numéro de catalogue ou de commande
 Número de catálogo o de pedido
 Codice catalogo e di dell'ordine numero
 Katalog- eller Beställningsnummer
 Catalogus- of Bestelnummer
 Katalog- eller Ordrenummer
 Katalog- eller bestillingsnummer
 Luettelo- tai viitenumero
 Número de catálogo ou pedido

SN

Seriennummer
Serial number
Numéro de série
Número de serie
Numero di serie
Seriennummer
Seriennummer
Seriennummer
Seriennummer
Sarjanumero
Número de série

Hersteller
Manufacturer
Fabricant
Fabricante
Produttore
Tillverkare
Fabrikant
Producent
Produsent
Valmistaja
Fabricante

Hersteldatum
Date of manufacture
Date de fabrication
Fecha de fabricación
Data di produzione
Tillverkningsdatum
Productiedatum
Produktionsdato
Produksjonsdato
Valmistuspäivä
Data de fabricação

Getrennte Sammlung von Elektro- und Elektronikgeräten (Europäische WEEE Richtlinie)
Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE)
Tri sélectif des appareils électriques et électroniques (Directive européenne DEEE)
Desechado separado de aparatos eléctricos y electrónicos (Directiva Europea RAEE)
Raccolta differenziata di apparecchi elettrici ed elettronici (direttiva europea RAEE)
Separat insamling av elektriska och elektroniska apparater (det europeiska WEEE-direktivet)
Gescheiden inzameling van elektrische en elektronische apparaten (Europese AEEA richtlijn).
Separat innsamling af elektrisk og elektronisk udstyr (det europæiske WEEE-direktiv)
Produkter med dette symbolet skal ikke avhendes sammen med vanlig husholdningsavfall, men krever separat kassering i henhold til lokale bestemmelser. (European Waste Electrical and Electronic Equipment Directive, WEEE)
Sähkö- ja elektroniikkalaitteille tarkoitettu erillinen keräyspiste (eurooppalainen WEEE-standardi)
Colecção separada de aparelhos elétricos e eletrônicos (Diretrizes Europeias WEEE)

Zulässiger Temperaturbereich in °C für Lagerung und Transport
Temperature limits in °C for storage and transport
Plage de température admise en °C pour le stockage et le transport
Rango de temperatura permitida en °C para almacenar y transportar el producto
Temperatura ammessa in °C per conservazione e trasporto
Tillåtet temperaturintervall i °C för lagring och transport
Toegestane temperaturen in °C voor opslag en transport
Tilladt temperaturområde i °C ved opbevaring og transport
Temperaturbegrensning i °C for oppbevaring og transport
Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°C)
Limite de Temperatura em °C para armazenamento e transporte

Zulässiger Temperaturbereich in °F für Lagerung und Transport
Temperature limits in °F for storage and transport
Plage de température admise en °F pour le stockage et le transport
Rango de temperatura permitida en °F para almacenar y transportar el producto
Temperatura ammessa in °F per conservazione e trasporto
Tillåtet temperaturintervall i °F för lagring och transport
Toegestane temperaturen in °F voor opslag en transport
Tilladt temperaturområde i °F ved opbevaring og transport
Temperaturbegrensning i °F for oppbevaring og transport
Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°F)
Limite de Temperatura permitida em °F para armazenamento e transporte

Zulässige Luftfeuchtigkeit für Lagerung und Transport
Humidity limitation for storage and transport
Humidité admise pour le stockage et le transport
Humedad del aire permitida para almacenar y transportar el producto
Umidità atmosferica ammessa durante il trasporto e la conservazione
Tillåten luftfuktighet för transport och lagring
Toegestane luchtvochtigheid voor opslag en transport
Tilladt luftfugtighed ved opbevaring og transport
Fuktighetsbegrensning for oppbevaring og transport.
Sallittu ilman kosteus kuljetuksen ja varastoinnin aikana
Umidade do ar admissível para o armazenamento e transporte

Zulässiger Luftdruck für Lagerung und Transport
Pressure limitation for storage and transport
Pression atmosphérique admise pendant le transport et le stockage
Presión de aire permitida para almacenar y transportar el producto
Pressione atmosferica ammessa durante il trasporto e la conservazione
Tillåten lufttryck för lagring och transport
Toegestane luchtdruk voor opslag en transport
Tilladt lufttryk ved opbevaring og transport
Trykkbegrensning for oppbevaring og transport
Sallittu ilmanpaine kuljetuksen ja varastoinnin aikana
Pressão do ar admissível para o armazenamento e transporte

Vorsicht Bruchgefahr!
Fragile, handle with care!
Fragile ! Manipuler avec soin
Atención. Frágil.
Attenzione: pericolo di rottura!
Försiktigt! Risk för brott
Voorzichtig, kans op breuk!
Forsigtig, risiko for brud!
Ømtålig, behandles forsigtig!
Varo särkymisvaaraa!
Frágil, manuseie com cuidado!

Trocken lagern!
Keep dry!
Conserver au sec !
Conservar en un lugar seco!
Evitare ambienti umidi!
Förvaras torrt!
Droog bewaren!
Opbevares tørt!
Hold tørt!
Säilytetään kuivassa paikassa!
Armazenar em ambiente seco!

Gebrauchsanweisung verbindlich befolgen.
(Hintergrundfarbe: blau, Vordergrundfarbe: weiß)
Follow instructions for use!
(Background color: blue, foreground color: white.)
Suivre le mode d'emploi.
(Couleur de fond : bleu ; couleur du premier plan : blanc)
Seguir obligatoriamente las instrucciones de uso.
(Color de fondo: azul, color de primer plano: blanco)
Attenersi obbligatoriamente alle istruzioni per l'uso.
(Colore dello sfondo: blu, colore in primo piano: bianco)
Bruksanvisningen ska alltid följas.
(Bakgrundsfärg: blå, förgrundsfärg: vit)
De gebruiksaanwijzing is bindend en dient gevolgd te worden.
(achtergrondkleur: blauw, voorgrondkleur: wit)
Følg altid brugsanvisningen.
(Baggrundsfarve: Blå; forgrundsfarve: Hvid)
Følg bruksanvisningen!
(Bakgrunnsfarge: blå, forgrunnsfarge: hvit.)
Käyttöohjetta on noudatettava tarkasti.
(Taustaväri: sininen, etualan väri: valkoinen)
Siga as instruções de uso!
Cor de fundo: azul, cor de primeiro plano: branco)

UDI

Unique Device Identification

MD

Medical Device

 **HEINE Optotechnik GmbH & Co. KG**
Dornierstr. 6, 82205 Gilching, Germany
www.heine.com

V-200.00.119